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Model: KT100AM

Wstęp

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Model KT100AM

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

Multifunctional height-adjustable toilet chair with steel construction, equipped with folding armrests upholstered in soft material. Non-slip feet guarantee stability on slippery surfaces. The set includes a five-litre waste container with a practical lid, and a removable toilet seat with lid.

Intended use

The medical device is intended to facilitate the performance of hygiene-related at home and in hospital, especially for people with reduced mobility. It can be used both in the shower and in the bath, providing stability and comfort while bathing. The product allows satisfying physiological needs safely and hygienically in a seated position, whether over the toilet or elsewhere.

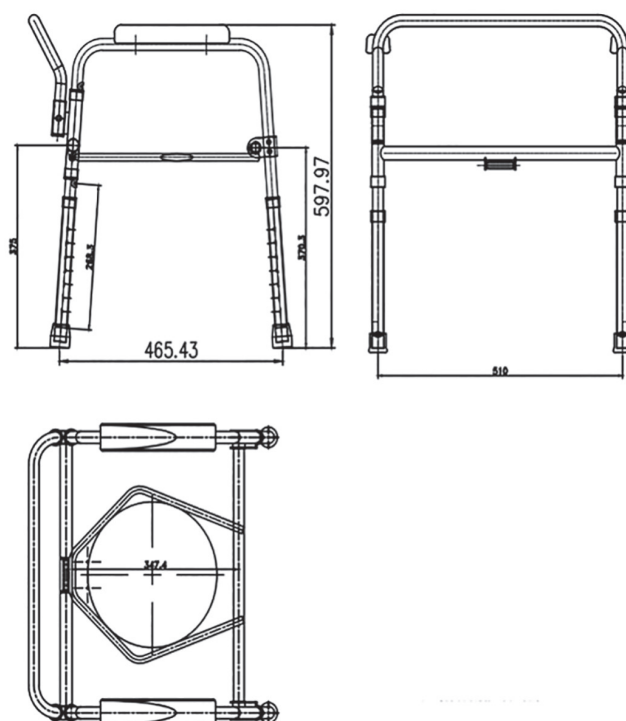
Contraindications

People with balance problems, coordination disorders, spinal instability or contraindications to sitting should avoid using the product due to the risk of falling and aggravating their condition.

The set includes: Side rails with handles, curved rear frame, front frame, rear frame, container holder, toilet seat with flap, sanitary bucket, legs with rubber caps, latches.

Assembly

1. Remove all components from the packaging.
2. Build the base: Unfold the chair frame by pulling the side pieces apart and connecting them with a crossbar. Ensure that the crosspiece latches are correctly fixed in the holes of the side pieces.
3. Secure the toilet lid: Attach a special railing to the back of the chair. It ensures that the toilet lid will not close accidentally. The snaps will make it easier to install the barrier.

Dimensions of the assembled product in millimetres (mm)

	The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.
	Maximum permissible user weight – 120kg.
	Any use other than for the intended purpose is considered inappropriate ☐ and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety rules

1. The chair is designed for use on flat and level surfaces.
2. Using a chair on uneven ground that does not provide stability can lead to tipping over and injury.
3. Standing on a chair is prohibited.
4. The maximum permissible load on the chair is 120kg.

5. Do not use the chair if you notice any visible or perceptible damage (e.g. cracks in the frame), as this could lead to an accident.

Adjustment and use

To adjust the height of the toilet chair, individually adjust each leg by pressing the latch on the bottom of the chair. Select the desired height by moving the leg frame up or down, then release the latch to lock the setting.

1. Make sure all legs are of equal height. Check that the legs are stable before use.
2. Place the bucket between two metal barriers that are connected by two crossbars.
3. Place the lid on top of the bucket and press down on it to make sure it is well seated.
4. Tighten the toilet seat in two places against the rear transverse frame on both sides of the bucket, then fix the toilet lid to the rear transverse frame.
5. Check that the plastic fixing clips are correctly attached and that the lid and toilet seat are securely fastened to the frame.

Maintenance

- Regular cleaning of the chair should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the chair thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, gasoline and any solvents, as these can cause damage to the frame, visible defects and lead to a loss of warranty.
- It is a good idea to check the condition of structural components at least once every three months, including fixing bolts, as well as pipes for any cracks. In addition, it is advisable to inspect the rubber cap regularly; if it is found to be worn, it should be replaced with a new one.

Storage and transport conditions

The chair should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer

Product content: Material: seat and back: high-density polyethylene, foam; frame and legs: powder-coated steel; supports: polypropylene; legs: thermoplastic.

Colour: White/grey.

Technical data: Height: 55 cm; Width: 51 cm; Length: 74 cm; Seat width: 46cm; Seat depth: 45 cm; Product weight: 23.6 kg

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ACTION S.A.