ENG STETHOSCOPE LD. *Instruction Manual.*

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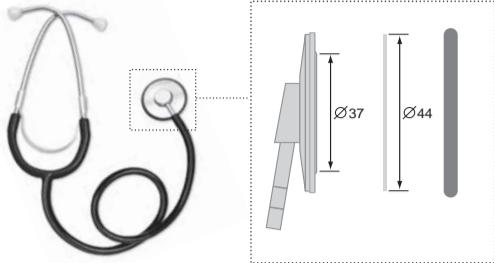
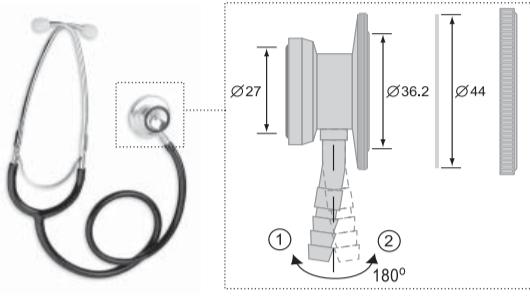
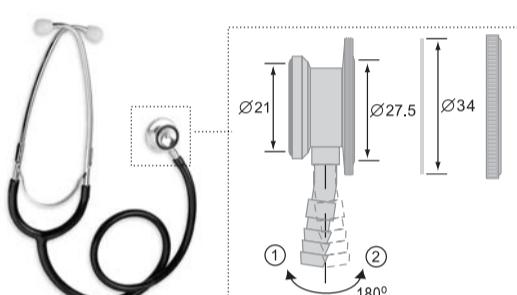
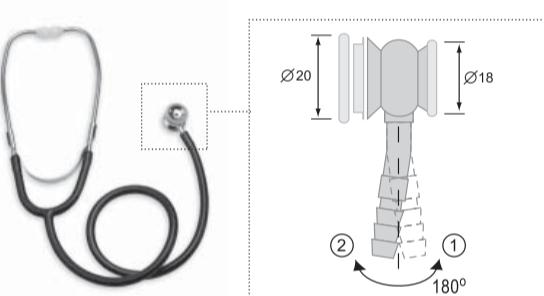
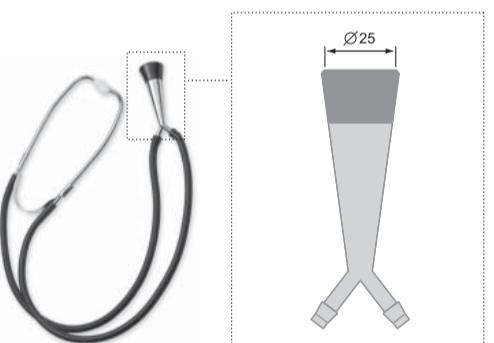
ENG

LD PROF-PLUS

LD Prof-Plus stethoscope is a Korotkoff's tone phonendoscope consisting of a head that is shaped like a plastic diaphragm, as well as connecting tube, binaural tubes, and ear ends.

Complete set:

- A light-metal stethoscope head (LD-S025) with a 44 mm diaphragm (LD-S023) – 1 piece.
- A medical connection tube (LD-S020) of 56 cm length (optional colors: black, grey, red, blue, purple, green, yellow) – 1 piece.
- Binaural tubes (LD-S019) – 1 piece.
- Ear ends (LD-S066) – 1 set.

Overall dimensions (when packed), mm: 100 x 210 x 30.

Net weight, g: 76.

LD PROF-I

LD Prof-I stethoscope is a stethophonendoscope intended for medical and paramedical personnel that consists of a dual head with one side shaped like a plastic diaphragm and another side – like an open-end bell, as well as connecting tube, binaural tubes, and ear ends.

Complete set:

- A light-metal stethoscope dual head (LD-S017) with a large diaphragm and open-end bell – 1 piece.
- The diaphragm (LD-S023) has a diameter of 44 mm. The bell is fitted with a contact rubber ring. The diameter of the bell is 27 mm. Turn the head around its axial valve by 180 degrees to switch over between the bell and diaphragm. The bend of the axial head indicates to an active side of the head. To switch over to the diaphragm bring the bend to position (1) and to switch over to the bell bring the bend to position (2).
- A medical connection tube (LD-S020) of 56 cm length (optional colors: black, grey, red, blue, purple, green, yellow) – 1 piece.

Binaural tubes (LD-S019) – 1 piece.

Ear ends (LD-S066) – 1 set.

Overall dimensions (when packed), mm: 100 x 210 x 30.

Net weight, g: 135.

LD PROF-II

LD Prof-II stethoscope is a special pediatric stethophonendoscope consisting of a dual head with one side shaped like a plastic diaphragm and another side – like an open-end bell, as well as connecting tube, binaural tubes, and ear ends.

Complete set:

- A light-metal stethoscope dual head (LD-S028) with a small diaphragm and open-end bell – 1 piece.
- The diaphragm (LD-S024) has a diameter of 34 mm. The bell is fitted with a contact rubber ring. The diameter of the bell is 21 mm. Turn the head around its axial valve by 180 degrees to switch over between the bell and diaphragm. The bend of the axial head indicates to an active side of the head. To switch over to the diaphragm bring the bend to position (1) and to switch over to the bell bring the bend to position (2).
- A medical connection tube (LD-S020) of 56 cm length (optional colors: black, blue, green, yellow) – 1 piece;

Binaural tubes (LD-S019) – 1 piece;

• Ear ends (LD-S066) – 1 set.
Overall dimensions (when packed), mm: 100 x 210 x 30.

LD PROF-III

LD Prof-III stethoscope is a special neonatal stethophonendoscope consisting of a dual head with one side shaped like a plastic diaphragm and another side – like an open-end bell, as well as connecting tube, binaural tubes, and ear ends.

Complete set:

- A light-metal stethoscope dual head (LD-S037) with a small diaphragm and bell – 1 piece. The diaphragm (LD-S040) has a diameter of 20 mm and bell – 18 mm. Turn the head around its axial valve by 180 degrees to switch over between the bell and diaphragm. The bend of the axial head indicates to an active side of the head. To switch over to the diaphragm bring the bend to position (1) and to switch over to the bell bring the bend to position (2).
- A medical connection tube (LD-S020) of 56 cm length (optional colors: black, blue, green, yellow) – 1 piece;
- Binaural tubes (LD-S019) – 1 piece;
- Ear ends (LD-S066) – 1 set.

Overall dimensions (when packed), mm: 100 x 210 x 30.

Net weight, g: 85.

LD PROF-IV

LD Prof-IV stethoscope is a special fetoscope consisting of a head that is shaped like a bell, as well as connecting tube, binaural tubes, and ear ends.

Complete set:

- Light-metal cone-shaped stethoscope head (LD-S038) with a plastic bell – 1 piece. The bell has a diameter of 25 mm.
- Medical connecting tubes (LD-S039) – two sound-conducting latex tubes of 56 cm length (optional colors: black, blue, purple, green) – 1 piece.
- Binaural tubes (LD-S019) – 1 piece.
- Ear ends (LD-S066) – 1 set.

Overall dimensions (when packed), mm: 100 x 210 x 30.

Net weight, g: 115.

MAINTENANCE, STORAGE, SERVICE AND DISPOSAL

- Protect the stethoscope from high humidity, direct sun exposure and shock loads.
- Do not keep or use the stethoscope close to heaters and open fire.
- Keep the stethoscope out of contamination.
- Make sure that the stethoscope does not contact with any aggressive solutions.
- When required, repair this device at authorized service centers only.
- For disposal of the device follow the local regulation applicable at your region. No special disposal provisions are stipulated by the producer.
- Operating conditions: +10°C to +40°C; relative humidity: 85% or below. Storage and transportation conditions: -34°C to +50°C; relative humidity: 85% or below.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

WARRANTY

The purchased device is warranted for 24 months from the date of sale. Faults detected during the warranty period will be repaired free of charge by a specialized service center within 21 business days from the date of device delivery to the service center. The costs for the defective device transportation are borne by Little Doctor Europe Sp. z o. o. (door-to-door service), except for faults that are not covered by the warranty. The party submitting the claim must properly prepare the device for shipment, protecting it from damage during transportation and enclose a completed warranty card (with the seller's stamp, date of sale and seller's signature) and proof of purchase (invoice or receipt). The warranty does not cover damages resulting from inadequate protection of the device during transportation. The warranty period is extended for the time the device is under repair. Post-warranty repair of the device is performed at the user's expense.

The warranty does not cover the following: consumables and accessories for individual use (tubes, nebulizers, masks, mouthpieces, etc.); products with mechanical, thermal or chemical damage, or damage caused by violations of the rules specified in the operating instructions; products with signs of modification, opening and/or repair by an unauthorized service center (by an individual); products that have been connected to unrecommended power supplies or used with unrecommended consumables; products that have been operated with non-repaired defects; products that are defective through no fault of the manufacturer, such as: actions of third parties, natural phenomena and natural disasters, animals, insects, foreign objects or liquids entering the device, etc.; internal and external contamination, scratches, cracks, scrapes and other mechanical damage to the equipment occurring during its operation; products with illegible or altered serial numbers or a damaged warranty seal; batteries, glass parts, light bulbs, as well as operations such as adjusting, cleaning and other maintenance of the product.

The buyer has the right to replace the device with a serviceable one if an authorized service center finds a manufacturing defect that cannot be eliminated or after 3 unsuccessful repairs of the same malfunction.

MANUFACTURER AND AUTHORIZED REPRESENTATIVES

SYMBOLS:

	Compliance with Directive MDR (EU) 2017/745
	Important: Read the manual
	Authorized Representative in the EU
	Manufacturer

The revision date of this Instruction Manual is shown on the last page as XX-XXXX-YYMM-NN, where YY is the year, MM is the month, and NN is the revision number.

This unit manufacturing is certified according to international standard ISO 13485. Stethoscope comply with the requirements of European Directive MDR (EU) 2017/745.

Manufactured under control and for Little Doctor International (S) Pte. Ltd. (7500A BEACH ROAD #11-313 THE PLAZA SINGAPORE 199591. Postal Address: 3 Coleman Street #03-24 Singapore 179804.

Manufacturer: Little Doctor Electronic (Nantong) Co. Ltd. (No. 8, Tongxing Road Economic & Technical Development Area, 226010 Nantong, Jiangsu, People's Republic of China).

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