
lionelo
care

User manual
Instrukcja obsługi
Bedienungsanleitung
Инструкция по эксплуатации
Manuale d'uso
Manuel de l'Utilisateur
Manual de usuario
Handleiding


Naudojimo instrukcija
Návod k obsluze
Használati utasítás
Manual utilizare
Bruksanvisning
Bruksanvisning
Brugsvejledning
Käyttöopas

Fidi Plus


Breast pump
Laktator
Muttermilchpumpe
Молокоотсос
Tiralatte
Tire-lait
Sacaleches
Borstkolf
Pienktraukis
Odsávačka mateřského mléka
Mellszívó
Pompă de sân
Bröstpump
Brystpumpe
Brystpumpe
Rintapumppu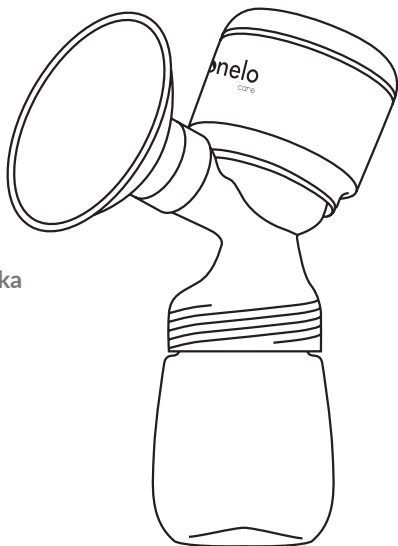

www.lionelo.com

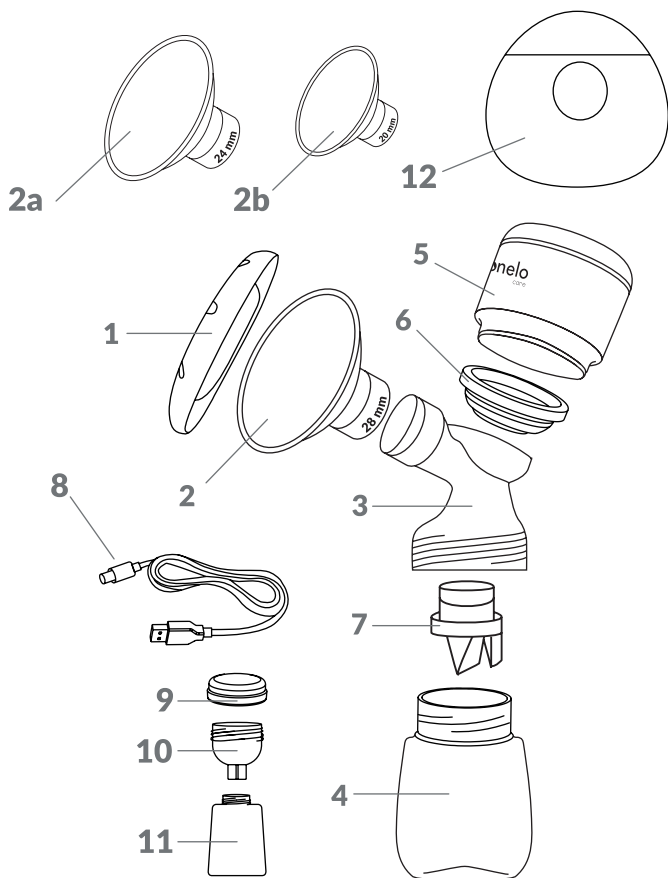A

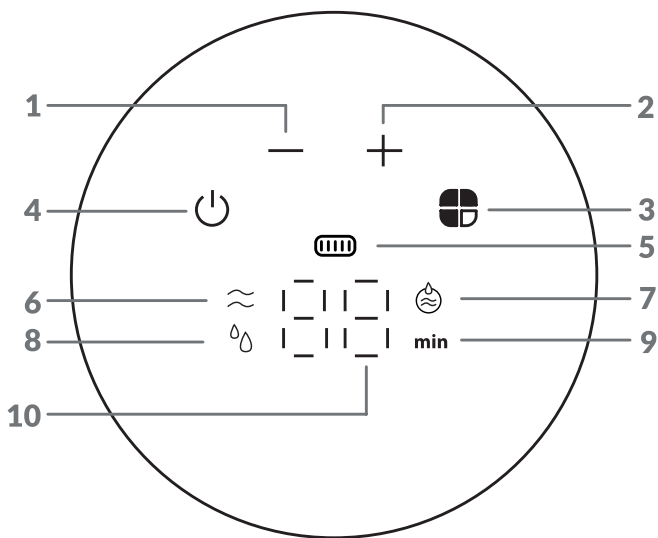
B

1


2

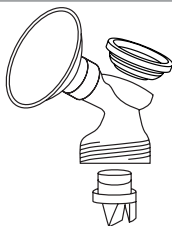3





4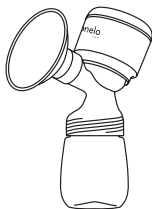


5

C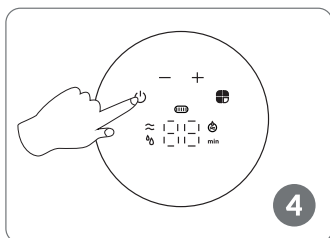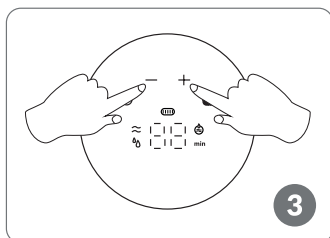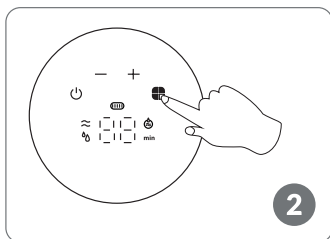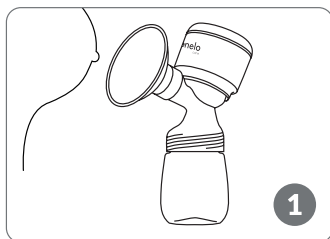

D

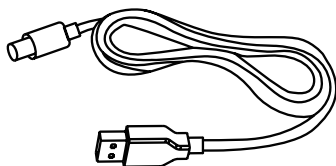E

Dear Customer!

In case of any questions or comments on the purchased product, please contact us:

help@lionelo.com

Producer:

BrandLine Group Sp. z o. o.

A. Kręglewskiego 1, 61-248 Poznań, Poland

IMPORTANT INFORMATION

Please read this manual before using the product to familiarize yourself with its functions and use it as intended.

Failure to follow the safety instructions and use not in accordance with the instructions may cause harm to health.

Keep this instruction manual for future reference.

1. Do not repair or modify the device. These operations can only be performed by an authorized service center.
2. This device is not a toy. Do not allow children to play with it. Keep the device out of the reach of children. Cleaning and maintenance should not be performed by children without supervision.
3. The device may be used by children at least 8 years old and persons with reduced physical, sensory or mental capabilities, as well as persons without relevant knowledge and experience in the use of the device, if they are supervised or have been previously instructed in the safe use of the device and the risks involved.
4. Do not start the device if you notice any damage. This applies both to the device itself and its accessories, such as the USB cable.
5. Do not use the device if it begins to malfunction, has been dropped or submerged in water, becomes excessively hot, discoloration or bulging appears, or if other abnormal phenomena occur. Contact the manufacturer's authorized service center.
6. Never leave the device plugged into a power outlet unattended. Only charge the device with a 5V DC 1A output charger.

7. Use the device away from heat sources, high temperatures, hot surfaces, direct sunlight, sources of sparking, open flames, oils and sharp edges.
8. Do not use chemicals to clean the device. Do not wash the main motor unit with water.
9. Turn off the device after each use.
10. Do not immerse the entire device in water or other liquids. Do not use near water.
11. Do not disinfect the device with a steam sterilizer.
12. Do not use the device improperly.
13. Do not use the device or touch the power supply with wet or damp hands.
14. Before cleaning or moving the device, turn it off and disconnect it from the power source.
15. Before and after each use, wash and disinfect all parts that will adhere to the body and come into direct contact with milk.
16. Thoroughly dry each component before use.
17. For hygienic reasons, the device should be used by one person.
18. If using the breast pump causes discomfort or pain, stop using the device immediately.
19. Do not use the device throughout your pregnancy.
20. Do not use the device if you are tired or while sleeping.
21. If breast milk does not appear for more than 5 minutes while using the device, stop using the breast pump and consult a doctor.

Precautions for batteries

1. Do not disassemble, hit, crush or throw into fire.
2. Do not leave in high temperature environments.
3. If severe swelling of the battery occurs, do not continue using it.
4. Do not attempt to replace the battery yourself, as this may cause damage to the battery and cause overheating, fire and injury.
5. Do not disassemble the battery, some lithium battery components are flammable, harmful and may cause injury, fire, cracking or explosion.
6. It is forbidden to disassemble, open, cut or scratch the battery with metal.
7. Use only a standard adapter that meets charging standards, otherwise charging is dangerous.

8. Do not charge the battery for a long period of time.
9. Lithium batteries should only be replaced by the company or authorized service providers, and replacing them with a battery of the wrong model involves the risk of explosion. Turn off the device before removing the battery.
10. Dispose of batteries in accordance with national environmental regulations.

The product complies with the requirements of European Union directives. In accordance with Directive 2012/19/EU, this product is subject to separate collection. The product should not be disposed of with municipal waste as it may pose a risk to the environment and human health. Return the used product to the recycling centre for electrical and electronic equipment.

Device diagram (fig. A)

- | | |
|-------------------------------|----------------------------------|
| 1. Protective cover | 6. Silicone diaphragm |
| 2. Silicone funnel (28 mm) | 7. Suction valve |
| 2a. Funnel (24 mm) | 8. USB cable |
| 2b. Funnel (20 mm) | 9. Milk bottle cap |
| 3. T-piece of the breast pump | 10. Adapter for milk storage bag |
| 4. Milk bottle | 11. Milk storage bag |
| 5. Main body | 12. Milk collector |

Touch screen diagram (fig. B)

- | | |
|---------------------------------|------------------------------------|
| 1. Reduction of intensity level | 6. Massage mode |
| 2. Increase of intensity level | 7. Mixed mode |
| 3. Switching the operating mode | 8. Milk pumping mode |
| 4. On/off switch | 9. Time indicator |
| 5. Battery level indicator | 10. Mode intensity level indicator |

Breast pump modes

1. Massage mode – 9 levels. Massage simulates the initial sucking phase of the baby during breastfeeding. The device automatically switches from massage mode to pumping mode after two minutes.
2. Pumping mode – 9 levels. This mode simulates the deeper sucking phase of the baby during breastfeeding.
3. Mixed mode – 9 levels. Combination of massage and pumping modes

to simulate the final sucking phase of the baby. It is used for pumping leftover milk.

Installation of the breast pump (fig. C)

NOTE

Make sure all parts of the device in contact with milk are disinfected with boiling water. Wash your hands before assembly.

1. Place the silicone funnel (2, fig. A) on the breast pump T-piece (3, fig. A). Make sure that the collar of the funnel is properly hooked (1, fig. C). You can also choose one of the two 24 or 20 mm funnels (2a or 2b, fig. A) to insert into the silicone funnel (2, fig. A).
2. Place the silicone membrane (6, fig. A) on the breast pump T-piece (3, fig. A). Make sure that the collar of the diaphragm is properly hooked and does not slip when using the breast pump (2, fig. C).
3. Insert the suction valve (7, fig. A) into the lower opening of the breast pump T-piece (3, fig. A). Make sure the valve is fully inserted (3, fig. C).
4. Attach the milk bottle (4, fig. A) to the lower hole of the breast pump T-piece (3, fig. A). Make sure the bottle is tightened securely (4, fig. C).
5. Connect the main body (5, fig. A) to the breast pump T-piece (3, fig. A) at the location of the silicone membrane (6, fig. A). Make sure that the body is installed correctly (5, fig. C).
6. If you want to take milk directly into the breast milk bag for storage for later, attach the adapter (10, fig. A) to the milk storage bag (11, fig. A), and then connect the whole thing to the breast pump T-piece (3, fig. A).

Method of use (fig. D)

NOTE

Charge the device before the first use.

Wash your hands before use. Make sure all parts of the breast pump are properly installed according to the installation instructions.

During use, maintain a normal sitting position and relax.

1. Press the breast pump tightly against your breast to avoid sucking in air (fig. 1).
2. Turn on the device by holding the switch for 2 seconds. The device will automatically turn on the massage mode when it starts. After 2 minutes

of operation in stimulation mode, the device will automatically switch to pumping mode (fig. 2).

3. To change the mode during use, press the mode switch button (3, fig. B).
4. The mode intensity level indicator displays the current operating level of the device. Press the decrease and increase intensity level buttons to switch mode intensity levels from 1 to 9 (fig. 3).
5. When using the breast pump, the breast milk collector (12, fig. A) allows you to collect milk while it is spontaneously flowing out of the other breast.
6. When, at the end of pumping, it's no longer as effective, switch to the mixed mode to pump the leftover breast milk.
7. Press the switch to stop the device (pause mode). Press again to resume its operation. If the pause mode is maintained for a longer time, the device will automatically turn off after 5 minutes.
8. To turn off the device press and hold the switch for about 2 seconds (fig. 4). The device will automatically turn off after 30 minutes of operation and no action taken.

The product has a memory function. The device automatically remembers the operation mode and intensity level of the mode after switching off and changing the operation mode.

The time indicator (9, fig. B) displays the current usage time of the device. If the device is stopped (pause mode), the current usage time will be displayed until the device resumes operation or is turned off.

Storage and heating of milk

Storage

1. After collecting the milk, immediately close the container tightly and put it in the refrigerator.
2. Do not store milk at room temperature (16-29°C).
3. In the refrigerator (4°C), the milk can be stored for 3 to 5 days.
4. The milk can be stored in the freezer (-16°C) for up to 6 months.
5. Do not store milk on the refrigerator door.

Defrosting and reheating

1. To freeze milk, fill the container $\frac{3}{4}$ full, as milk expands in volume when frozen.
2. To preserve the nutritional value of milk defrost it naturally in the refrigerator

or under warm running water with a maximum temperature of 37°C.

3. Before feeding, shake the milk container gently to mix the milk evenly.
4. Do not defrost frozen milk in the microwave or boiling water to avoid losing nutritional value and burning your baby.
5. Never re-freeze milk that has already been defrosted once and store it either at room temperature or in the refrigerator.

Charging (fig. E)

1. Connect the DC plug of the cable to the device's power socket on the side of the main body, and connect the other end with the USB plug to the USB adapter (not included).
2. During charging, the battery level indicator flashes green. When it stops flashing, it indicates that the device is fully charged.
3. The device automatically detects the battery level. When the battery level is high, the battery level indicator lights up green. When more than half of the battery is used up, the battery level indicator lights up yellow. When the battery level is low, the battery level indicator starts to glow red, and when the indicator starts to blink, the device will turn itself off after a while.

NOTE

You can use the breast pump while the battery is charging, but extreme caution is recommended. However, please note that frequent use of the device while it is charging may result in reduced battery life.

Use a DC5V/1A adapter for charging.

Troubleshooting

Problem	Solution
The device does not switch on	• The device's battery may have discharged or entered a protective state caused by long storage. Connect the device to the adapter to activate the battery. • Fully charge the battery before using the device for the first time and after long-term storage.
Insufficient suction power	• Check that the silicone funnel of the device fits snugly to the breast. • Check whether all parts of the device are correctly and fully installed. • If the room temperature is below 15°C, leave the device running for about 30 seconds before using it.

Problem	Solution
The device displays the message "E1"	• If the adapter used to charge the device has a voltage higher than 5V, it will display the message "E1" and will be protected from over voltage and cannot be charged. • After using the 5V adapter to charge the device, it will resume standard charging.
Pain or discomfort	• Adjust the intensity level or change the mode of operation. • Change the position or alignment of the breast. • In the event of unremitting pain or discomfort, consult a doctor.

Cleaning and disinfection of the device

NOTE

Do not clean or disinfect the main body or power cord of the device. These parts do not come into contact with food. They can be wiped clean with a soft cloth. Wash all parts in contact with the body and milk separately to avoid drying out food residue and prevent the growth of bacteria.

1. Before the first use

Before using the device for the first time, wash with a cleaning agent and then sterilize with boiling water all its parts except the main body. Please follow the cleaning and disinfection instructions in steps 2 and 3 below.

2. After each use

1. Disassemble the device into its individual parts and rinse with cool water all parts in contact with the body and milk to remove food residues.
2. Soak all separated parts in water with mild detergent for 5-10 minutes. Then wash them with a clean cloth or gentle brush and rinse them separately under clean running water.
3. Leave all parts of the device to dry in a clean place.
4. Store the device parts in a clean bag or container for the next use.

NOTE

Clean the parts that have been in contact with the body once a day and after each use. The device does not require maintenance. Be careful not to damage the device when cleaning it.

3. Daily disinfection

1. After use, disassemble the breast pump and clean the main device with a dry and soft cloth.
2. Before using it for the first time, disinfect all components that will come into contact with milk (milk bottle, T-piece, valve, diaphragm, silicone funnel and collector).
3. To disinfect, put all components in boiling water for 10 minutes. Make sure all components are immersed in the water and none touches the dry part of the pot, as the components may melt.
4. Before the next use, rinse all elements under running water to remove food residue.
5. Leave all items to dry completely on a clean dish cloth or dish dryer.

NOTE

Do not use sharp tools to clean silicone components, as this may result in damage to them.

Be careful not to damage the suction valve and silicone diaphragm when cleaning. If they are damaged, the device will not work properly.

Remove the suction valve from the device carefully. When cleaning it, rinse in clean water without inserting your finger or any objects into it, as this may result in damage.

The photos are for reference only, the actual look of the products may differ from the ones shown in the pictures.

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