
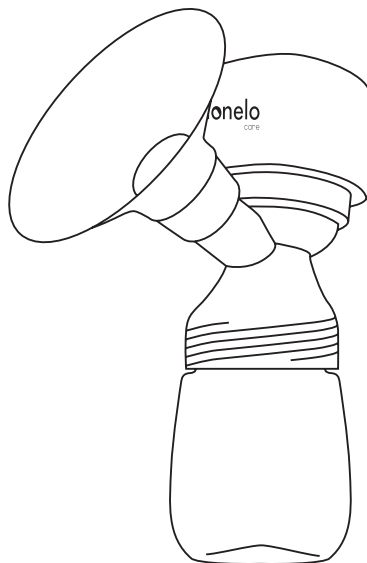
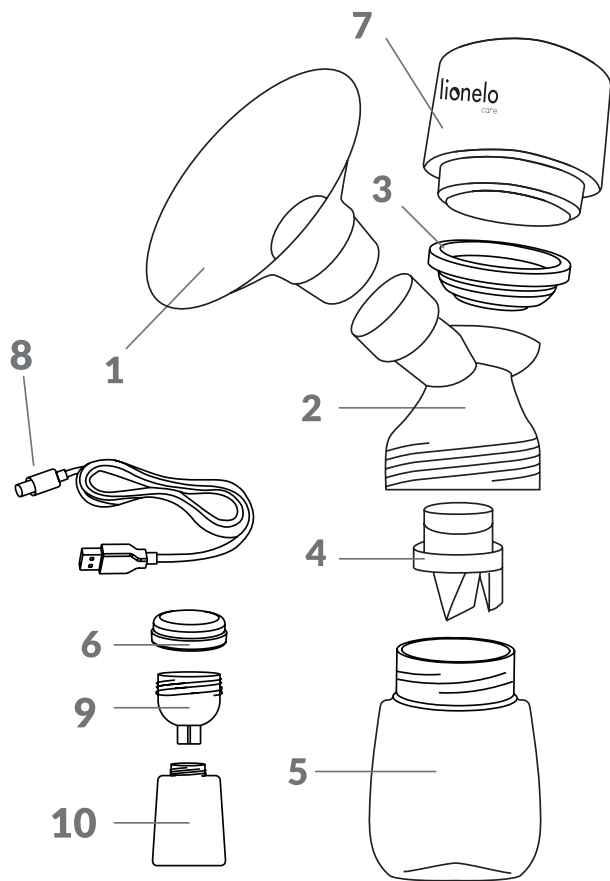
# Fidi One

Breast pump  
Laktator  
Muttermilchpumpe  
Молокоотсос  
Tiralatte  
Tire-lait  
Sacaleches  
Borstkolf  
Pienktraukis  
Odsávačka mateřského mléka  
Mellszívó  
Pompă de sân  
Bröstpump  
Brystpumpe  
Brystpumpe  
Rintapumppu

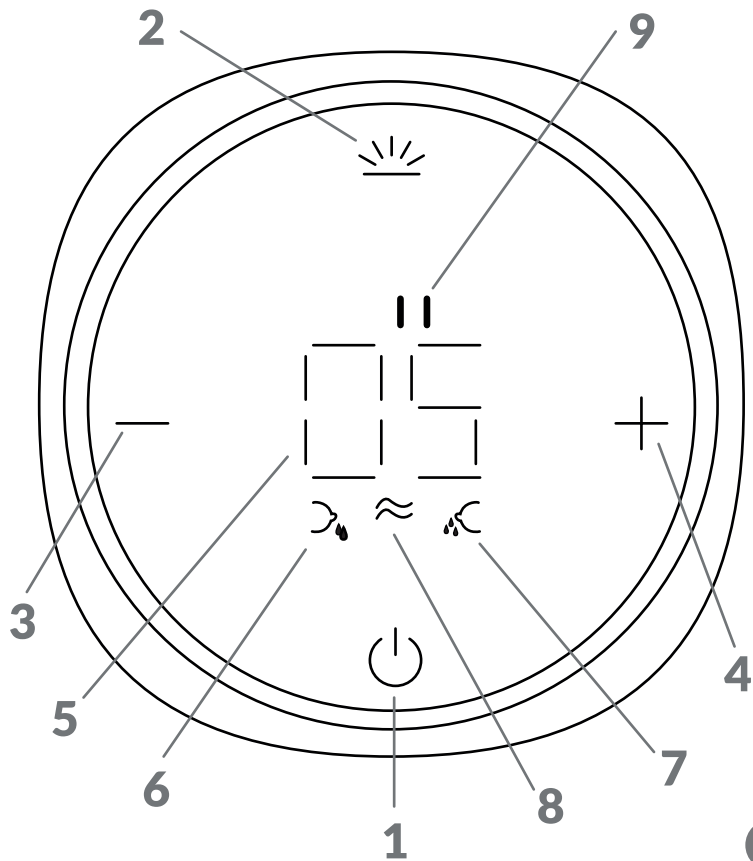A


B

1

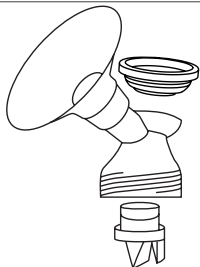
2

3

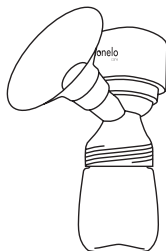4

5

C

**Dear Customer!**

If you have any comments or questions about the product you have purchased, please contact us:

**help@lionelo.com**

**Manufacturer:**

BrandLine Group Sp. z o. o.

A. Kręlewskiego 1, 61-248 Poznań, Poland

**IMPORTANT INFORMATION**

Please read this manual before using the product to familiarize yourself with its functions and use it as intended.

Failure to follow the safety instructions and use not in accordance with the instructions may cause harm to health.

Keep this instruction manual for future reference.

1. Do not repair or modify the device. These operations can only be performed by an authorized service center.
2. This device is not a toy. Do not allow children to play with it. Keep the device out of the reach of children. Cleaning and maintenance should not be performed by children without supervision.
3. The device may be used by children at least 8 years old and persons with reduced physical, sensory or mental capabilities, as well as persons without relevant knowledge and experience in the use of the device, if they are supervised or have been previously instructed in the safe use of the device and the risks involved.
4. Do not start the device if you notice any damage. This applies both to the device itself and its accessories, such as the USB cable.
5. Do not use the device if it begins to malfunction, has been dropped or submerged in water, becomes excessively hot, discoloration or bulging appears, or if other abnormal phenomena occur. Contact the manufacturer's authorized service center.
6. Never leave the device plugged into a power outlet unattended. Only charge the device with a 5V DC 1A output charger.

7. Use the device away from heat sources, high temperatures, hot surfaces, direct sunlight, sources of sparking, open flames, oils and sharp edges.
8. Do not use chemicals to clean the device. Do not wash the main motor unit with water.
9. Turn off the device after each use.
10. Do not immerse the entire device in water or other liquids. Do not use near water.
11. Do not disinfect the device with a steam sterilizer.
12. Do not use the device improperly.
13. Do not use the device or touch the power supply with wet or damp hands.
14. Before cleaning or moving the device, turn it off and disconnect it from the power source.
15. Before and after each use, wash and disinfect all parts that will adhere to the body and come into direct contact with milk.
16. Thoroughly dry each component before use.
17. For hygienic reasons, the device should be used by one person.
18. If using the breast pump causes discomfort or pain, stop using the device immediately.
19. Do not use the device throughout your pregnancy.
20. Do not use the device if you are tired or while sleeping.
21. If breast milk does not appear for more than 5 minutes while using the device, stop using the breast pump and consult a doctor.

The product complies with the requirements of European Union directives.

In accordance with Directive 2012/19/EU, this product is subject to separate collection. The product should not be disposed of with municipal waste as it may pose a risk to the environment and human health. Return the used product to the recycling centre for electrical and electronic equipment.

## Device diagram (fig. A)

- |                               |                                 |
|-------------------------------|---------------------------------|
| 1. Silicone funnel            | 6. Milk bottle cap              |
| 2. T-piece of the breast pump | 7. Main body                    |
| 3. Silicone diaphragm         | 8. USB cable                    |
| 4. Suction valve              | 9. Adapter for milk storage bag |
| 5. Milk bottle                | 10. Milk storage bag            |

## Touch screen diagram (fig. B)

- |                                      |                             |
|--------------------------------------|-----------------------------|
| 1. On / off / change of suction mode | 6. Mixed mode               |
| 2. Backlight on / off switch         | 7. Milk pumping mode        |
| 3. Reduction of intensity level      | 8. Massage mode             |
| 4. Increase of intensity level       | 9. Battery charge indicator |
| 5. Mode intensity level indicator    |                             |

## Breast pump modes

1. Massage mode – 9 levels. Massage simulates the initial sucking phase of the baby during breastfeeding.
2. Pumping mode – 9 levels. This mode simulates the deeper sucking phase of the baby during breastfeeding.
3. Mixed mode – 9 levels. Combination of massage and pumping modes to simulate the final sucking phase of the baby. It is used for pumping leftover milk.

## Installation of the breast pump (fig. C)

### NOTE

Make sure all parts of the device in contact with milk are disinfected. Wash your hands before assembly.

1. Place the silicone funnel (2, fig. A) on the breast pump T-piece (3, fig. A). Make sure that the collar of the funnel is properly hooked (1, fig. C).
2. Place the silicone membrane (4, fig. A) on the breast pump T-piece (3, fig. A). Make sure that the collar of the diaphragm is properly hooked and does not slip when using the breast pump (2, fig. C).
3. Insert the suction valve (5, fig. A) into the lower opening of the breast pump T-piece (3, fig. A). Make sure the valve is fully inserted (3, fig. C).
4. Attach the milk bottle (6, fig. A) to the lower hole of the breast pump T-piece (3, fig. A). Make sure the bottle is tightened securely (4, fig. C).
5. Connect the main body (8, fig. A) to the breast pump T-piece (3, fig. A) at the location of the silicone membrane (4, fig. A). Make sure that the body is installed correctly (5, fig. C).
6. If you want to take milk directly into the breast milk bag for storage for later, attach the adapter (10, fig. A) to the milk storage bag (11, fig. A), and then connect the whole thing to the breast pump T-piece (3, fig. A).

## Use

### NOTE

Charge the device before the first use.

Before use, wash your hands and make sure that all parts of the breast pump are properly installed according to the installation instructions.

Control the level of milk collected. The amount of milk in the bottle must be small enough so that it does not reach the suction valve.

1. Relax and sit in a comfortable chair. You can use a pillow to support your back.
2. Press the breast pump tightly against your breast to avoid sucking in air.
3. Turn on the device by holding down the switch. The device will automatically turn on the massage mode when it starts. Press the decrease and increase intensity level buttons to select the desired massage level (between 1 and 9).
4. When the milk starts flowing, press the switch briefly to change the mode to pumping. Press the decrease and increase intensity level buttons to select the desired massage level (between 1 and 9).
5. When, at the end of pumping, it's no longer as effective, switch to the mixed mode to pump the leftover breast milk.
6. Each time you change modes, the mode intensity level indicator displays the current level of the device for that mode.
7. Press the backlight switch briefly to turn on the backlight. Press a second time to increase the intensity of the light. When pressed a third time, the backlight will turn off.
8. To turn off the device, press and hold the switch and gently unhook the main body from the device's T-piece, then unscrew the bottle from it as well.

The device will automatically turn off after 30 minutes of operation and no action taken.

The product has a memory function. When you turn off and change the mode, the device automatically remembers the intensity level of the mode from the previous use.

## Storage and heating of milk

### Storage

1. After pumping milk, immediately close the container tightly and put it in the refrigerator or freezer.
2. In the refrigerator the milk can be stored for 48 hours, while in the freezer it can be stored for 3 months.

## NOTE

If you want to freeze milk, fill the container  $\frac{3}{4}$  full, as milk expands in volume when frozen. Do not store milk on the refrigerator door.

## Heating

1. To preserve the nutritional value of milk, defrost it naturally in the refrigerator or under warm running water with a maximum temperature of 37°C.
2. After defrosting, heat milk in a special milk warmer or warm water. Do not heat milk in a microwave or boiling water to avoid losing nutritional value.
3. Before feeding, shake the milk container gently to mix the milk evenly.

## NOTE

Do not defrost frozen milk in the microwave or boiling water to avoid losing nutritional value and burning your baby.

Never re-freeze milk that has already been defrosted once and store it either at room temperature or in the refrigerator.

## Charging

1. Connect the plug of the USB-C cable to the device's power socket on the side of the main body, and the other end with the USB plug to the USB adapter (not included).
2. During charging, the battery level indicator will flash. When it stops flashing and shows all dashes, it means the device is fully charged.

## NOTE

You can use the breast pump while the battery is charging, but extreme caution is recommended. However, please note that frequent use of the device while it is charging may result in reduced battery life.

For charging, use a 5V DC 1A adapter.

## Troubleshooting

| Problem                       | Solution                                                                                                                                                                                                                                                                                                                     |
|-------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| The device does not switch on | <ul style="list-style-type: none"><li>• The device's battery may have discharged or entered a protective state caused by long storage. Connect the device to the adapter to activate the battery.</li><li>• Fully charge the battery before using the device for the first time and after long-term storage.</li></ul>       |
| No suction                    | <ul style="list-style-type: none"><li>• Check that the device is installed correctly.</li><li>• Check that the suction valve is not damaged.</li><li>• Make sure that the silicone diaphragm and the silicone funnel are accurately installed on the device's T-piece.</li></ul>                                             |
| Insufficient suction power    | <ul style="list-style-type: none"><li>• Check that the silicone funnel of the device fits snugly to the breast.</li><li>• Check whether all parts of the device are correctly and fully installed.</li><li>• If the room temperature is below 15°C, leave the device running for about 30 seconds before using it.</li></ul> |
| Pain or discomfort            | <ul style="list-style-type: none"><li>• Adjust the intensity level or change the mode of operation.</li><li>• Change the position or alignment of the breast.</li><li>• In the event of unremitting pain or discomfort, consult a doctor.</li></ul>                                                                          |

## Cleaning and disinfection of the device

### NOTE

Do not clean or disinfect the main body or power cord of the device. These parts do not come into contact with food. They can be wiped clean with a soft cloth.

Before using the device for the first time, wash with a cleaning agent and then sterilize with boiling water all its parts except the main body.

1. After each use, disassemble the device into its separate parts and rinse with cool water all parts in contact with the body and milk to remove food residues.
2. Wash all separated parts in warm water with mild detergent, then rinse thoroughly under clean running water.
3. Leave all parts of the device to dry in a clean place.
4. Store the device parts in a clean bag or container for the next use.
5. Use a special pan or boil in boiling water for 5 minutes to sterilize all device parts in contact with milk and body.

### NOTE

Do not use sharp tools to clean silicone components, as this may result in damage to them.

Be careful not to damage the suction valve and silicone diaphragm when cleaning. If they are damaged,

the device will not work properly.

Remove the suction valve from the device carefully. When cleaning it, rinse in clean water without inserting your finger or any objects into it, as this may result in damage.

*The photos are for reference only, the actual look of the products may differ from the ones shown in the pictures.*

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