CS

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THANK YOU FOR PURCHASING A HILDING MEDICAL DEVICE / PRODUCT.

Congratulations on your choice!

GENERAL INFORMATION ABOUT THE USE OF MATTRESSES

- The medical device/ product is intended for sleep and rest.
- The medical device/ product shall retain its characteristics:
 - at a temperature range from 10 to 27°C,
 - in an environment with a relative humidity not higher than 70%.
- Putting it on the surface in the form of a flexible or wooden frame, in which the spacing between the slats is 3-4 cm and consists of at least 18 slats will ensure adequate ventilation, support and prevent deformation of the medical device / product. It is unacceptable to place a medical device / product directly on the floor or on a uniform plate. This arrangement may result in the accumulation of moisture in the lower zone of the medical device/ product due to the lack of adequate ventilation.
- We recommend a wide selection of slatted base from the Hilding collection. For spring mattresses, we recommend elastic slatted base, for foam mattresses, in addition to elastic ones, we also offer manually or mechanically adjustable slatted bases. Safe and correct use of the slatted base is conditioned by its support along the entire length on both sides of the bed frame and the central strip running from the front of the bed to the headboard.
- Medical devices/products must be protected from liquids, temperatures above 50°C and open flames.
- Medical devices/products may have a specific smell of the components used in their manufacture process, particularly during the initial period of use. A specific smell does not mean that the medical device/product is harmful to your health.
- Individual units of the medical device / product may differ in width and length from the nominal dimensions by 1%.
- During the use of the mattress, permanent deformations with a depth of up to 10% of the mattress height may occur on its surface,

constituting a completely natural consequence of using the mattress.

- In the case of medical devices/products with a foam or latex core, the foam core or latex may be glued to receive the required size. The manufacturer guarantees that the layout of the comfort zones in this case will not be significantly changed and this will not affect the standard of use in a meaningful way.
- Covers intended for washing have special zippers (except for mattresses of non-standard size), sewn around the mattress and labels with information about the method of washing. The washing information on the label must be followed. The cover after washing may shrink up to 5% from the nominal dimension. Before putting the cover back on the medical device/ product, it must be properly stretched.
- Removing a cover marked as non-washable will void the warranty.
- Medical devices/products may produce sounds that are the result of the technological solutions and materials used in them.
- Jumping on the mattress or prolonged pressure on one point (e.g., frequent sitting on the edge of the mattress/ bed) may cause damage to the springs and/or deformation of the foam cores.
- To ensure better hygiene and longer maintenance of cleanliness of the main mattress, we recommend using a mattress topper. The top mattress, commonly called "Topper", located directly on the main mattress not only protects it from dirt, but also increases the feeling of comfort during sleep.
- The safe shelf life of mattresses packed in rolled form is 6 months from the date of purchase. After this period, Hilding Anders Polska reserves the right to refuse a complaint for failure to maintain the correct dimensions of the mattress after unpacking.

BEFORE THE FIRST USE

- Medical devices/products reach the correct shape and size within 3-4 days after unpacking

and unfolding. During this time, the medical device/product can be used.

- Products with a latex core should be shaken before use for the correct positioning of the mattress core.
- When moving a medical device/product, the following rules should be followed:
 - the medical device/product must be moved by at least 2 persons,
 - if model is equipped with handles, use them for moving,
 - do not pull/push the medical device/ product on the floor,
 - grab the mattress in the central part, not by the corner.

Do not place the medical device/product vertically, as falling over may be dangerous for people within its reach.

- Arrange the mattress in the bed on the appropriate frame, check that the length and width of the frame correspond to the size of the mattress. Only a bed with size fitting to the size of the mattress allows its proper use.
- Initially, a new mattress may seem too hard or too soft for you. You need not more than one month for your body to get used to the mattress and for the mattress to adapt to the shape of your body. To ensure full comfort and fully enjoy your new mattress, a properly selected pillow is also recommended.

MATTRESS CARE:

- Each mattress has information about the method of taking care on the label. The use of gentle detergents and following the recommended washing temperature influence the correct preservation of the appearance of the cover.
- In the case of washable covers, we recommend washing it at least every 6 months, for hygiene and comfort of use.
- In order to get rid of mites and dust, we recommend vacuuming the mattress / bed with a traditional vacuum cleaner using an

upholstery brush.

- We do not recommend cleaning the mattress cores in any way.
- We do not recommend the use of a carpet-beater and steam cleaners while taking care of a medical device/product.
- We do not recommend exposing the medical device / product to external weather conditions.
- It is recommended to rotate the mattress every 6 months (does not apply to one-sided mattresses and the ones on the wooden frame) and move it over by 180° in order to extend its life.

PACKAGING:

- do not walk on the foil, as this may cause slipping and falling,
- do not put the foil on the head – the risk of suffocation,
- it is recommended to keep the label from the packaging in order to identify the medical device/product properly.

ENVIRONMENT:

The material from which this product was made is recyclable. Check the waste management policies in your area and the addresses of places where items can be recycled.

THE BUYER IS OBLIGED TO READ THESE INSTRUCTIONS FOR PROPER USE – APPLIES TO MATTRESSES:

Rumba Elips/Velur, Pasodoble Velvet/ Tencel, Salsa Velvet/Tencel, Makarena Merced/Elips, Fandango Medi-Cover

GENERAL TERMS AND CONDITIONS OF USE

Before using the product, please read this Instructions for Use carefully. All recommendations and guidelines given in the Instructions should be followed in order to use the product properly. The device must be used in accordance with the intended purpose of the manufacturer.

INTENDED PURPOSE

- preventing the formation of bedsores: Rumba Elips/Velur, Salsa Velvet/Tencel
- supporting the rehabilitation process of posture defects: Pasodoble Velvet/Tencel, Fandango Medi-Cover
- preventing the developmental defects of locomotor system (musculoskeletal system): Makarena Merced/Elips

MATTRESS CORE MAINTENANCE

The mattress should not be placed on surfaces such as the floor or solid plate, which prevent ventilation. Mattresses should be rotated at least once every 6 months (excluding one-sided mattresses), this increases the service life and improves sleep hygiene. Mattresses are ventilated after production, but they may have their own specific smell, which disappears over time. A mattress without a clear marking about the possibility of washing should not be dry cleaned, wet washed or cleaned with steam. Mattresses should not be dried in the sun; the mattress should be regularly ventilated. Any dirt should be cleaned with a wet cloth. In case of flooding or getting the mattress wet, dry it in a horizontal position, without turning it over. Water (moisture) should evaporate constantly in one direction. The mattress should be placed on a base

that guarantees air access from the bottom. If the mattress cover is removable, it must be removed and dried separately.

MAINTENANCE OF THE COVER

The mattress cover is a removable cover. In the middle part of the cover there are zippers sewn in from 4 sides for easy removal. Before washing, the cover must be closed at least on 3 sides. The cover should be washed at a temperature in accordance with the washing instructions attached to the cover, with the use of delicate detergents, do not chlorinate, do not dry clean. Do not dry mechanically. Dry in an upright position. Do not iron. A label containing care tips is sewn into each mattress. In order not to damage the mattress cover through improper care - these tips should be followed.

Date of last update: 24.10.2022

Any serious incident related to a device shall be reported to the manufacturer or to the competent authority of the EU Member State in which the user or the patient resides.

LEGEND

catalogue number

manufacturer

the product meets the requirements of (EU) 2017/745 Regulation of the European Parliament and of the Council

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