INSTRUCTIONS FOR USE


Elbow crutch with flip-up forearm support

Product code: FlipTim

1. Characteristics

The elbow folding crutch consists of a comfortable, durable handgrip, special adjustable flip-up forearm support and an aluminum shaft finished with a high-quality rubber tip. Thanks to the length adjustment and folding function, it is very convenient for daily use. The crutch has an ergonomic shape of the cuff and angle adjustment, which ensures optimal distribution of the pressure on the handgrip.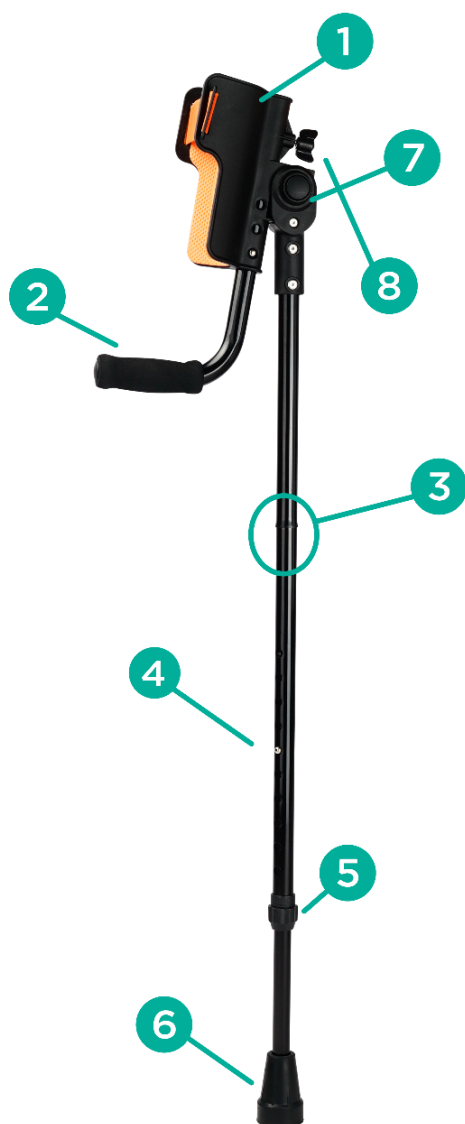

- 1 forearm support
- 2 handgrip
- 3 folding point
- 4 height adjustment
- 5 adjustment lock
- 6 rubber tip
- 7 forearm support angle adjustment
- 8 handgrip length adjustment

AFTER FOLDING

2. Application

The forearm crutch is an ancillary equipment which is used to increase the area of support in case of patients with reduced mobility. It is also recommended to patients who need help and support while walking. It can be used as an aid in keeping balance after injuries of lower limbs or trunk.

Contraindications: coordination and balance disorders, advanced rheumatic changes, injuries and dysfunctions of upper limbs which may hinder the user to support on the crutch.

3. Assembly and adjustment

- To make transportation easier, the crutch has a folding function. Before adjusting the length, the crutch should be unfolded by connecting the two parts of the shaft.
- To adjust the length of the crutch, unscrew the adjustment lock.
- To adjust the crutch, press the locking pin and select appropriate position of the inner tube. Lock the position with the locking pin. The crutch is adjustable in 2.5 cm increments.
- Once the length adjustment is complete, screw the crutch adjustment lock.
- To set the proper length of the crutch, stand upright and bend upper limb in elbow joint at the angle of 20°.
- The crutch should be adjusted individually to the user so that the handgrip is at hip joint level and the cuff is at the height of upper part of the forearm.
- Before use, ensure that the crutch is stable and the locking pin fully protrudes through adjustment hole of the crutch.

NOTE! In case of doubts concerning the technique of moving with the use of the crutches, consult a person who has appropriate knowledge and qualifications.

HANDGRIP LENGTH ADJUSTMENT

- The crutch has a handgrip length adjustment. To adjust the length, unscrew the knob on the back of the forearm support and push in the locking pin, then select appropriate length. Lock the pin and tighten the screw.

ANGLE ADJUSTMENT OF THE SUPPORT

- The crutch has an adjustable angle of forearm support at the forearm. The maximum angle of adjustment is 90°.
- To adjust the angle of support press the button under forearm support.

4. Technical data

Overall width	21 cm
Height adjustment	97-119 cm
Handgrip length adjustment	27-32 cm
Forearm support width	10,5 cm
Size after folding	70 x 23,5 cm
Maximum capacity	136 kg
Weight	1,60 kg
Leg	aluminium, black

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

- Clean the crutch using only water and mild detergents, wipe dry after each use.
- **Do not use:** strong detergents, abrasive agents, bleach, petrol nor solvents (it may cause frame damage, visible defects, and in consequence – loss of warranty rights).
- It is recommended to check rubber tip regularly. Replace worn tip immediately.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.

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