INSTRUKCJA UŻYWANIA/INSTRUCTIONS FOR USE/NÁVOD NA POUŽITIE/ NÁVOD K POUŽITÍ/GEBRAUCHSANWEISUNG

SUPPORT 103_2013

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EN SUPPORT - gel toe crest with a ring. **Application:** for feet with deformed toes, nail thickenings, and skin thickening on the tips of the toes. **Performance characteristics of the device:** It adapts to the shape of the foot and fills the space under deformed toes, which increases the contact surface with the ground and ensures a more even distribution of foot pressure on the ground when walking. This reduces the formation of nail thickenings and skin thickening on the tips of the toes. The ring holds the foot orthosis in the correct position. **Instructions for use and cleaning:** Apply to cleansed and dried foot. The product must not be applied directly to wounds. Place it under the toes while putting the ring on the middle toe. Please note that the product is available for the right and left foot. Wash in lukewarm water with a mild soap, leave to dry completely and use talcum powder if necessary. Store in a dry and clean place and protect from direct sunlight. **Precautions and warnings:** People with diabetes and blood supply disorders should consult a doctor before using the product. If redness / irritation occurs, discontinue use and consult a doctor, and report information to the manufacturer/distributor. Keep out of reach of children.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user is established.

Date of issue of the instructions for use: the second issue of 22.09.2021.

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