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INSTRUCTIONS FOR USE

Issue number and date: 1 / 11.2024

**Shower chair with backrest and armrests
Model KP300AM**

Introduction

Thank you for choosing our product. We are confident that it will meet your expectations. We encourage you to read this document before using the product for the first time. These instructions are intended to provide the necessary information on the safe use of the medical device. The manual provides detailed guidance on the device's functions and capabilities. Retaining the document allows this information to be easily found in the future, facilitating correct use.

Description of the medical device

Shower chair with backrest and armrests (seat 40cm) - height-adjustable, is a product for users with limited mobility for support when performing hygiene-related activities. Made of PVC (polyvinyl chloride), the chair is fitted with height-adjustable aluminium legs. Rubber foot tips are designed with safety in mind, effectively eliminating the risk of slipping on wet surfaces.


Intended use

The medical device is intended to facilitate the performance of hygiene-related at home and in hospital, especially for people with reduced mobility. It can be used both in the shower and in the bath, providing stability and comfort while bathing.

Contraindications

People with balance problems, coordination disorders, spinal instability or contraindications to sitting should avoid using the product due to the risk of falling and aggravating their condition.

Set composition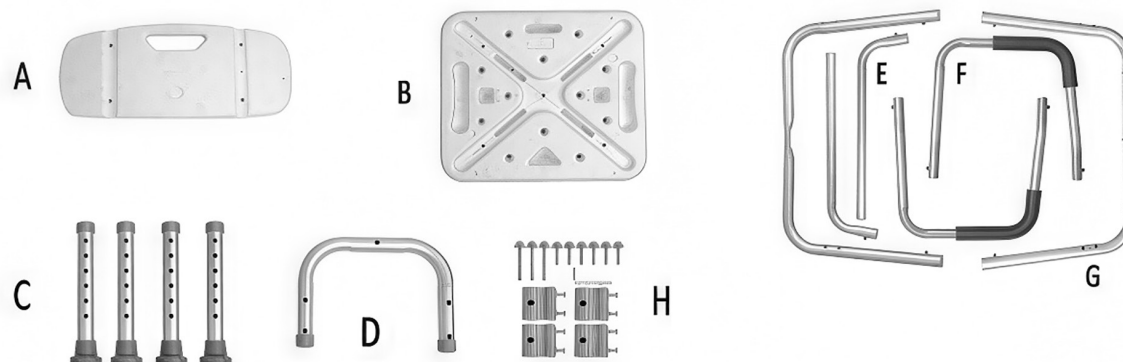

Item number	Item description	Quantity
A	Backrest	1 pc.
B	Seat	1 pc.
C	Height-adjustable chair legs	4 pcs.
D	U-shaped frame	1 pc.
E	Backrest supports	2 pcs.
F	Armrests	2 pcs.
G	Frame parts for legs	2 pcs.
H	Bolts / brackets	3 pcs / 6 pcs. / 4 pcs.

Assembly


1. Place the seat (B) on a flat surface so that the screw holes on the bottom face upwards.

2. The next step is to place the section of the frame for the legs, which has a hole at the bottom (G), with the screw holes in the screw holes.

Then place another section of the leg frame on top of the section that has already been placed.

3. Place the U-shaped frame (D) on the stacked frame sections. Fix the frame section and the U-frame with 3 long and 2 short screws at the bottom of the seat.

4. Now attach the height-adjustable chair legs (C) to the frame section. Slide the legs onto the frame section until they lock into the height adjustment knobs.
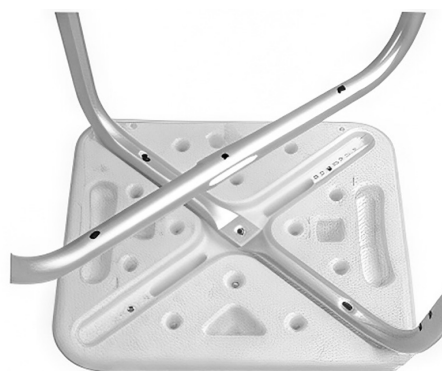


5. In the next step, insert the two backrest brackets (E) into the open ends of the U-shaped frame and fix the backrest (A) with four short screws.

6. Now insert the armrests (F) into the circular brackets made of powder-coated aluminium, making sure that they are slightly tilted outwards. The brackets are on the outside of all parts of the leg frame.

As soon as you hear a click, the armrest is locked into the bracket.

7 Once assembly is complete, check that the legs, backrest and armrests are securely fastened.

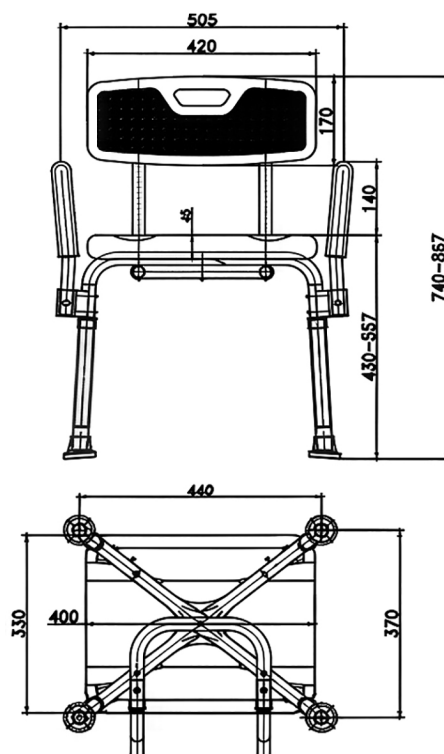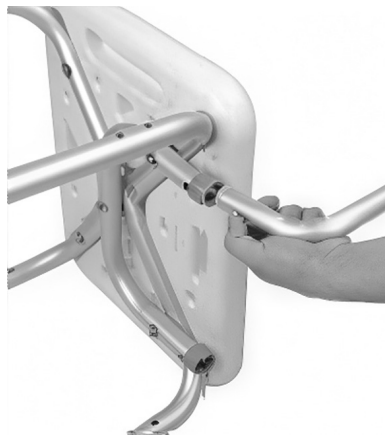Dimensions of the assembled product in millimetres (mm)

	The user or their carer should consult a healthcare professional if they are unsure whether the medical device is appropriate for the patient in question.
	Maximum permissible user weight – 110kg.
	Any use other than for the intended purpose is considered inappropriate and DANGEROUS. The manufacturer and/or distributor shall not be liable for damage caused by improper, incorrect and/or unreasonable use.

Safety and usage rules

1. The chair is designed for use on flat and level surfaces.
2. Using a chair on uneven ground that does not provide stability can lead to tipping over and injury.
3. Standing on a chair is prohibited.
4. The maximum permissible load on the chair is 110kg.
5. Do not use the chair if you notice any visible or perceptible damage (e.g. cracks in the frame), as this could lead to an accident.

Maintenance

- Regular cleaning of the chair should be carried out with water, a sponge and mild detergents intended for plastics, such as soap or a mild cleaning agent approved by the National Institute of Hygiene. It is important to dry the chair thoroughly after each use.
- Avoid the use of: strong detergents, abrasives, bleach, gasoline and any solvents, as these can cause damage to the frame, visible defects and lead to a loss of warranty.
- It is a good idea to check the condition of structural components at least once every three months, including fixing bolts, as well as pipes for any cracks. In addition, it is advisable to inspect the rubber cap regularly; if it is found to be worn, it should be replaced with a new one.

Storage and transport conditions

The chair should be stored in a dry and well-ventilated place, protected from moisture and extreme temperatures to prevent damage to the materials. Contact with chemicals that may affect its durability and performance should be avoided. During transport, the product should be secured in such a way as to prevent displacement and damage. Use appropriate protective packaging.

Disposal

Disposal of the product must be carried out in accordance with the applicable local environmental regulations.

Product life

The expected product life is up to 5 years if used in accordance with the information in the instructions for use. The product can be used after this period if it is undamaged and meets the specified requirements for safe use.

Warranty

- The product is covered by a 24-month warranty from the date of purchase, against any defects in material or workmanship.
- The warranty covers the free replacement and/or repair of parts found to be manufacturing defects.
- The warranty does not cover parts subject to normal wear and tear.
- The warranty is no longer valid if the product has been tampered with/modified, if the defect is the result of misuse or if the damage cannot be attributed to the manufacturer.

NOTE

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is residing.

Explanation of symbols

	Conformity of the device with EU Regulation 2017/745 on medical devices		Medical device
	Take a look at the instructions for use		Maximum user weight
	Model		Batch/series number
	Date of production		Warning
	Unique identification code of the device		Medical device manufacturer

Product content: Material: seat and backrest: high-density polyethylene, foam; frame and legs: powder-coated aluminium; supports: polypropylene; legs: thermoplastic.

Colour: White/grey.

Technical data: Product weight: 3.5 kg; Backrest dimensions: 41 x 17 x 2.5 cm; Seating area: 40 x 33 cm; Maximum user weight: 110 kg. Adjustable seat height: 43-56 cm in 6 levels; Overall dimensions: 49 (width) x 42 (depth) x 74-87 (height) in cm.

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ACTION S.A.