6

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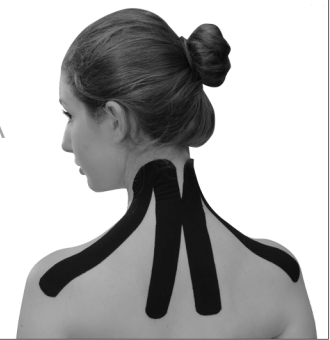
Nanjing 3H Medical Products Co., Ltd.
Add: No.5 Zhushan Road, Gaochun district,
Nanjing, Jiangsu, China 210000
Tel: +86-600-056-0111

Lotus NL B.V.
Add: Koningin Julianaplein 10, Ie Verd, 2595AA,
The Hague, Netherlands.
Tel: +0031626669008

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THERAPY TAPE

+ Muscle tissue Injury

When muscle tissue been damaged, nerve received signal, correct change, lead action deformation, nerve adapt to change the nerve center damage gradually, lead acondformation. After long time, cause muscleatrophy tissue injury, repeated a vicious circle.

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6

THERAPY TAPE

II. Instructions of 3 basic therapy method

1. Structural

STRETCH THE TAPE NOT THE TISSUE

1. Never maximally stretch the tape.
2. Don't start off with maximum correction.
3. Never stretch the the ends of the tape.
4. Amount of tape stretch and tissue shortening depends on clinical rationale
5. Do not touch the adhesive
6. Back to normal position after application, no crinkle on the tape

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Applikacje

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Neck and shoulder pain
Ból szyi i barków

1. Never maximally stretch the tape.
2. Don't start off with maximum correction.
3. Never stretch the the ends of the tape.
4. Amount of tape stretch and tissue shortening depends on clinical rationale
5. Do not touch the adhesive
6. Back to normal position after application, no crinkle on the tape

17

THERAPY TAPE

Basic Shapes and Cut

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1. Basic Shapes: X, I, Y, W
2. Cut and then round all of the edges
3. Lengh depends on injury

23

24

THERAPY TAPE

Neuro-Structural Taping Technique

Neurosensory, Microcirculatory - Structural

Instructions for use

Amount Of Stretch On Tape

NEUROSENSORY 0 1 2 3 4 5 6 7 8 STRUCTURAL

1. Neurosensory Application: no stretch of the tape (peel off the release paper and apply after its resiliency)
2. Microcirculatory Application: stretch by 10%-20%
3. Structural Application: stretch by 20%-50%

Catalog / Spis treści

I. Rapid diagnostic methods 9
II. General specification of 3 Taping Techniques 11
III. Taping Application cases 17
IV. Key Application Guidelines 25

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1.

2.

THERAPY TAPE

+ Prevent the injured area from vicious circle

1. Stimulation of the injured area and abate the pain / Neurosensory taping
2. Structural adjustment of the injured area / Neurosensory taping
3. Facilitate the circulation, quicken the recovery of the injured area / Neurosensory taping

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14

THERAPY TAPE

1. Never maximally stretch the tape.
2. Don't start off with maximum correction.
3. Never stretch the the ends of the tape.
4. Amount of tape stretch and tissue shortening depends on clinical rationale
5. Do not touch the adhesive
6. Back to normal position after application, no crinkle on the tape

19

THERAPY TAPE

IV. Key Application Guidelines

I. Ready:
1. Identify the Clinical Objective.
2. Choose the appropriate Kinesiology Taping Product.
3. Choose pre-cut strips (X, I, Y, W).
4. Never maximally Stretch the tape.
5. Structural application doesn't start off with maximum correction.
6. No Stretch at the start and end of each section
Amount of tape stretch and tissue.
7. shortening depends on clinical rationale.
8. Don't touch the adhesive.

II. IFU:
1. Clean, dry, and hair free skin.
2. No stretch at the start and end of each section.
3. Do not touch the adhesive.
4. Rub the tape to activate each section.
5. To remove, peel tape in the same direction as hair growth.
6. Other notes: this taping technique need to coordinate with other rehabilitation methods, such as therapy, massage or exercise.

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THERAPY TAPE

I. Quick diagnose

1. Structural/Acute muscle strain/acute injury / trauma Chronic posture change/Chronic structural change/Like joint sprain/Muscle contusion(part of bar back and stiff neck belongs to structural)
2. Neurosensory/Most part of chronic pain or Acute neurological damage Like: lumbar muscle degeneration spinal disease,etc
3. Microcirculatory/Large bruises contusion, large damage

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III. Microcirculatory

STRETCH THE TISSUE AND THE TAPE

1. Never stretch the the ends of the tape.
2. Amount of tape stretch and tissue shortening depends on clinical rationale.
3. Do not touch the adhesive.

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