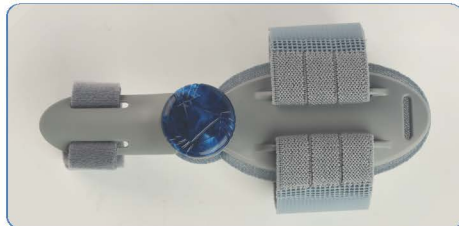
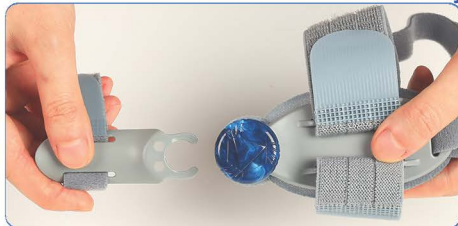
INSTRUKCJA

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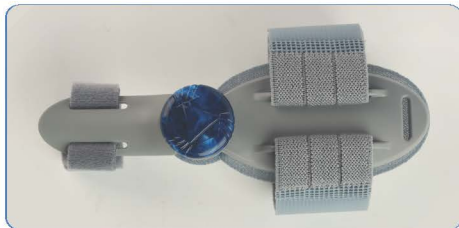
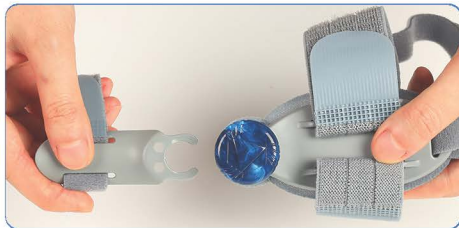
Use Steps

fit4med.

The big toe fixing plate can be inserted horizontally in the direction of the arrow.

Loosen the straps, put on and secure the silicone pad to the foot bone.

Adjust the arch strap to fix the sole of the foot.

Adjust the big toe strap.

Adjust the knob to the appropriate tension.

USER INSTRUCTIONS:

Begin with 30 minutes per day and increase gradually until you can wear the splint daily 2 - 3 hours while walking or nightly while sleeping. Recommend wearing the splint 4 - 8 weeks or until realignment is achieved. Thereafter, wear 2 - 3 times per week for maintenance. Wear barefoot, or splint may fit in wide-fitting slippers or shoes. The splint is designed to fit most foot sizes and can be worn on either the right or left foot. In case of pain, redness, irritation, or any other adverse symptoms, discontinue use of the splint and consult a physician

Indications for Use:

- Correction and alleviation of symptoms of hallux valgus (bunion) in early to moderate stages of deformity.
- Prevention of further progression of the big toe deformity.
- Reduction of pain and discomfort associated with pressure on the big toe joint.
- Stabilization and improvement of toe alignment during walking and rest.
- Support of conservative therapy as a complement to orthopedic treatment.
- Intended for daily use both while standing and during sleep.

Contraindications

There are no known hypersensitivity or allergic reactions associated with this product when used correctly. Discontinue use of Bunion corrector in case of the following conditions:

- skin diseases or injuries in the affected area,
- especially inflammation, as well as scarring with swelling,
- redness or increased skin temperature,
- gout,
- rheumatoid arthritis,
- tenderness to pressure,
- polyneuropathy, or swelling of the leg.

Helpful informations:

Bunion Corrector can be worn while barefoot. In order to avoid soiling, wear a sock over the splint and foot. Bunion Corrector can be worn in shoes that are non-restrictive, provide a wide opening towards the front of the shoe, and are a comfortable fit. Bunion Corrector should not be worn while swimming, on the beach, or in salt water. Salt and sand may cause damage to the splint. In case of improper use, product liability is precluded.

Hand wash in warm water

Do not bleach

Do not iron

Do not dry clean

Do not tumble dry