# USER MANUAL

## Aluminium tripod cane with long legs

Product code: JMC-C 2680SL

### 1. Characteristics

The tripod cane is made of aluminium-steel frame. It has height adjustable ergonomic, chromed handle with soft foamed handgrip and powder coated three leg base.

#### Elements of the cane:

- A. handgrip
- B. height adjustment – locking pin
- C. rubber tips

### 2. Application

The tripod cane is ancillary equipment designed for patients with reduced mobility who need help and stable support while walking. The tripod is also recommended to elderly people as an aid in keeping balance. The design with three legs provides better stability than regular canes.

**Contraindications:** coordination and balance disorders, advanced rheumatic changes, injuries and dysfunctions of the upper limbs which may hinder the user to support on the tripod cane.

### 3. Adjustment

To adjust the crutch, press the metal pin and select the appropriate position of the inner tube. Lock the tubes in place using the metal pin. The crutch is adjustable in 2.5 cm increments

- To set the proper length of the tripod, stand upright and bent upper limb in the elbow joint at the angle of 30°.
- The tripod should be adjusted individually to the user so that the handgrip is at the hip joint level.
- Before use, ensure that the crutch is stable and that the locking pin is fully protruding through the adjustment hole.

**NOTE!** In case of doubts concerning the technique of moving with the use of tripod, consult a person who has appropriate knowledge or qualifications.

#### 4. Technical data

Type	adjustable
Leg colour	silver
Base colour	black
Handgrip	black, foamed
Height adjustment	77-98 cm
Maximum capacity	100 kg
Tripod weight	0,6 kg
Base dimensions	24,5 cm x 29,5 cm

#### 5. Notes

**Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.**

#### 6. Maintenance and cleaning

- Clean the tripod cane using only water and mild detergents.
- **Do not use:** strong detergents, abrasive agents, bleach, petrol nor solvents (it may cause frame damage, visible defects, and in consequence - loss of warranty rights).
- It is recommended to check rubber tips regularly. Replace worn tips immediately.

#### 7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

#### 8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

#### 9. Labels

	Reference number		Manufacturer
---	------------------	---	--------------

	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: [www.timago.com](http://www.timago.com). Thank you for choosing Timago!

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