VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

www.vevor.com/support

Bathroom Wheelchairs

MODEL: HFK-3331

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Bathroom Wheelchairs

MODEL: HFK-3331


NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:


Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.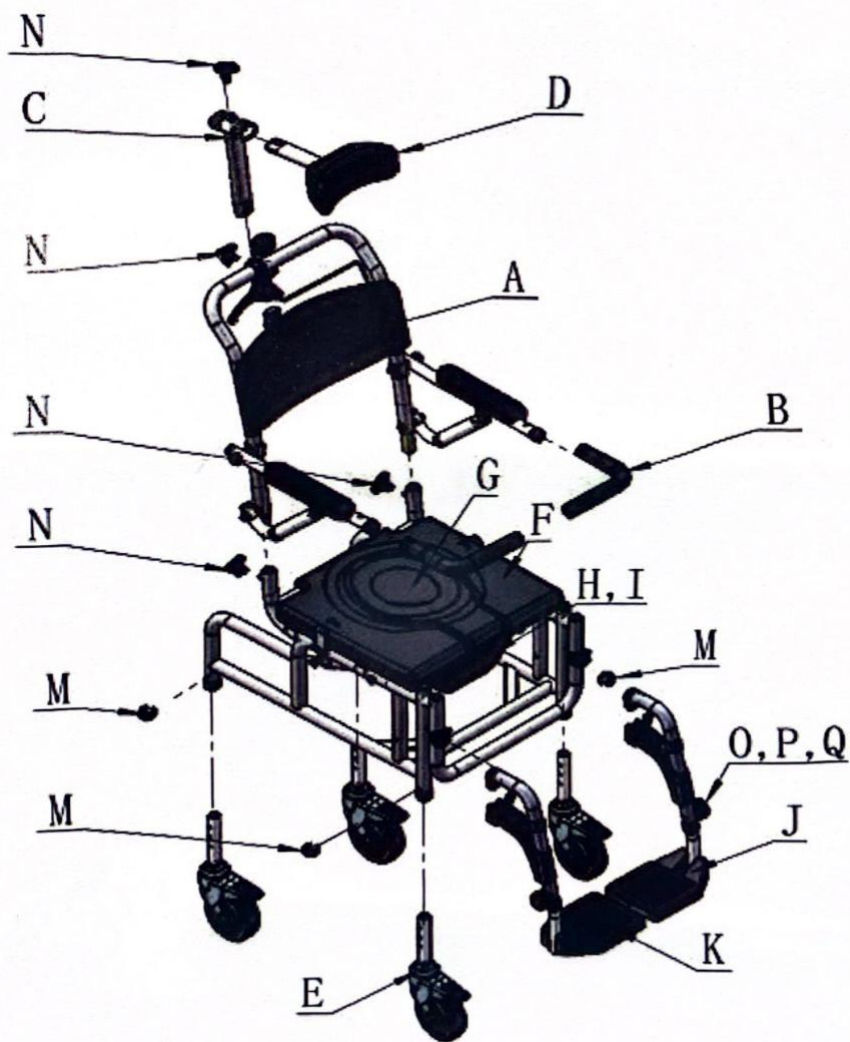

Note:When you open the packing box, please take care to protect all the spare parts in the box. If assembled on the ground, put the paper or carpet on the ground first to prevent scratching. Please pay attention to distinguish the spare parts, and then put them well.

Part List			
No.	Picture	Name	QTY.
A		Main Frame	1
B		Protective Handle	2
C		PU Head Pad Adjustable Set	1
D		PU Head Pad Set	1
E		Caster	4
F		Seat Pad Set	1
G		Seat Lid	1

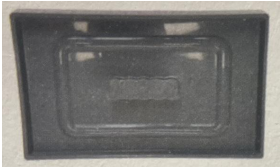H		Bucket Lip	1
I		Bucket	1
J		Left Footrest	1
K		Right Footrest	1
L		Winder (Installed on Main Frame)	1
M		C-shaped Buckle (Installed on Main Frame)	4
N		M8 Screw	4
O		M6 Screw (Installed on Footrest)	2
P		Shim-Screw (Installed on Footrest)	2

Q		Screw (Installed on Footrest)	2
R		Blet	1

Assembly Description

Figure 1

Step 1: As shown in Figure 1, insert the backrest assembly in the tube hole of the seat frame, and force them down to the end. Then lock the backrest with the plum knob.

Figure 2

Step 2: The four casters are aligned to the corresponding holes in the tube of the sub-frame and locked with a C-locking button as shown in Figure 2 .

Figure 3

Step 3: Lock the left and right detachable footrests against the front tube hole of the seat frame, as shown in Figure 3.

Figure 4

Step 4: Insert the two protective guard handles into the armrest tubes, allowing the pinball to be aligned with the hole position, as shown in Figure 4.

Figure 5

Step 5: Insert the PU pillow adjustment component into the oval tube of the backrest and locked with the triangle knob, as shown in figure 5.

Figure 6

Step 6: Insert the PU pillow into the oval tube of the PU pillow adjustment component and locked with the triangle knob as shown in figure 6.

Post-installation inspection:


- 1) Please refer to the manual diagram to check whether the installation is correct.
- 2) Verify that all accessories are fully assembled.
- 3) Check whether there is any looseness. If so, be sure to re-tighten it.
- 4) Check whether the appearance of all parts is skewed and make adjustments.

Partial component description :

- 1) To adjust the PU pillow height and distance: loosen the triangle knob to adjust the distance and then lock it .

2) If you need to wash your hair, you must press the brakes on the casters to ensure that the chair is fixed and then press the control switch on your back. Adjust the seat angle according to your needs as shown below: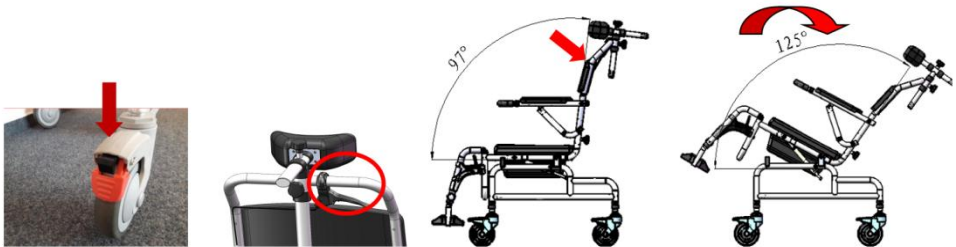

3) When the seat is reset, press the steel balls on the left and right sides of the armrest protection tube, and then press it back into place.

Product Maintenance:

1. Do not scratch or scratch the surface of the product to prevent material damage.
2. Daily cleaning and maintenance with the neutral synthetic detergent, wipe with wet cloth, then wipe with dry cloth.
3. Do not contact with strong acid or alkali to prevent corrosion.

Product Storage:

Packaged commode chairs should be stored at the temperature not exceeding -10°C~40°C and a relative humidity not exceeding 80%. No corrosive gases, the room is well ventilated and clean. During transportation, it should be protected from moisture, light, and heat sources.

Manufacturer: Shanghaimuxinmuyeyouxiangongsi

Address: Shuangchenglu 803nong11hao1602A-1609shi, baoshanqu, shanghai 200000 CN.

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP
----	-----

YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147,
Centurion House, London Road,
Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
----	-----

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Importerad till AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australien
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730
E-CrossStu GmbH