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GB English – Bed Safety Rail Installation Instructions

Installation:

Preparing the installation area

1. Ensure that the bed surface where the rail will be attached is stable and clean.
2. Choose the appropriate side of the bed – the rail should be installed in a position that provides the best support for the user.

Attaching the mounting brackets

1. Place the metal mounting brackets against the edge of the bed frame where you plan to install the rail.
2. Mark the screw holes with a pencil to ensure precise drilling.

Securing the rail

1. Position the rail on the brackets so that the mounting holes on both align.
2. Attach the rail to the brackets using the included screws – tighten them gradually to avoid misalignment.
3. Ensure the rail is firmly secured – it should not move or wobble.

Testing stability

1. Gently pull the rail to check its strength.
2. Sit on the bed and lean on the rail – it should provide stable support.
3. If you notice any looseness, tighten the screws.

Done! Your rail is now securely installed and ready for use.

Usage Tips:

- ✓ Regularly check that the screws are tightly secured for maximum safety.
- ✓ If needed, the rail can be removed by unscrewing the screws and detaching the brackets.
- ✓ Clean the rail with a soft, damp cloth to maintain its aesthetics and hygiene.

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