


MRI-Use ConRad Thermal Blanket

Stop Heat Loss

How to Use the MRI Safe ConRad® Thermal Blanket

Directions for Use – Prevent Phase 1 heat loss

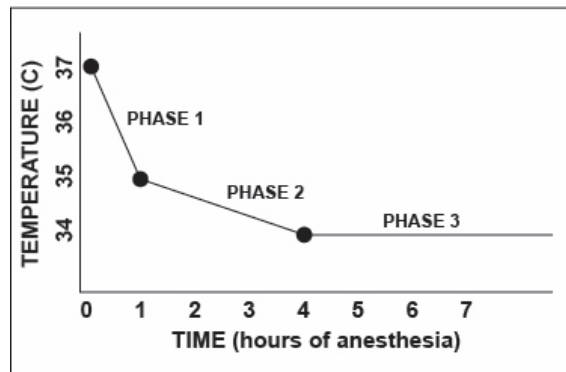

- 1) Place the patient on a solid surface warming pad and covered with the MRI ConRad Thermal Blanket in the pre-oxygenation phase prior to induction of anesthesia.
- 2) Keep the patient on the warming pad and covered with the MRI ConRad Thermal Blanket as much as possible while applying patient monitors.
- 3) Keep the patient covered with the MRI Safe ConRad Thermal Blanket while moving patient to MRI
- 4) Once at MRI scanner, remove the MRI Safe ConRad Thermal Blanket, wrap the patient with a regular fleece blanket then replace the MRI Safe ConRad Thermal Blanket so that it covers the entire body during the scan.
- 5) Make certain that an insulating layer, such as a regular blanket or another MRI ConRad Thermal Blanket, is underneath and between the patient and the cold underlying MRI surface.
- 6) The MRI ConRad Thermal Blanket may be cleaned with soap and water or a germicidal solution.