



Global Men's Home Care Device

KAI2AR





**To become
a more
attractive man.**

**When you
run towards
success.**




KAISAR is a product that irradiates light to the important parts of men.

**Red and
near infrared
wavelengths**

Penile care


The corpus
cavernosum in the
penis is the part that
creates the rigidity.

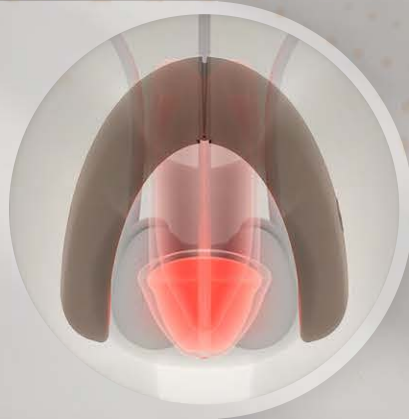

**Red and
near infrared
wavelengths**

Testicular care

The testicles are also
the site of sperm
production and male
hormones.

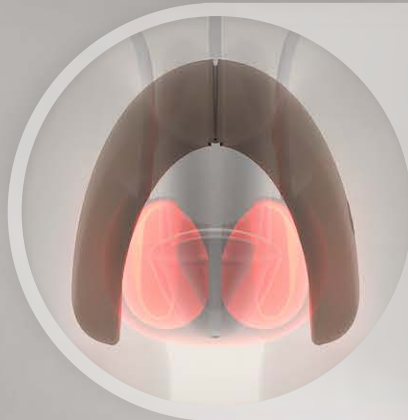

Ancient wisdom, history of light



In ancient Rome,

public baths were everywhere.
For the ancient Romans, baths were
"solariums," and many Romans
would gather there to sunbathe.



There are many examples


of how humans consciously introduced sunbathing into
their lives starting thousands of years ago.

In ancient Rome, solariums were installed in bathhouses
from around the 2nd and 3rd centuries B.C., and solariums called
solar rooms were also installed in each household.

ref: 宇都宮光明, 「太陽を浴びたは、医者はいらない: ガンをも防ぐビタミンDが、
病気に負けない心と体を作る!」, ワニブックス, 2010

LIGHT THERAPY

based on red/near infrared light(NIR)

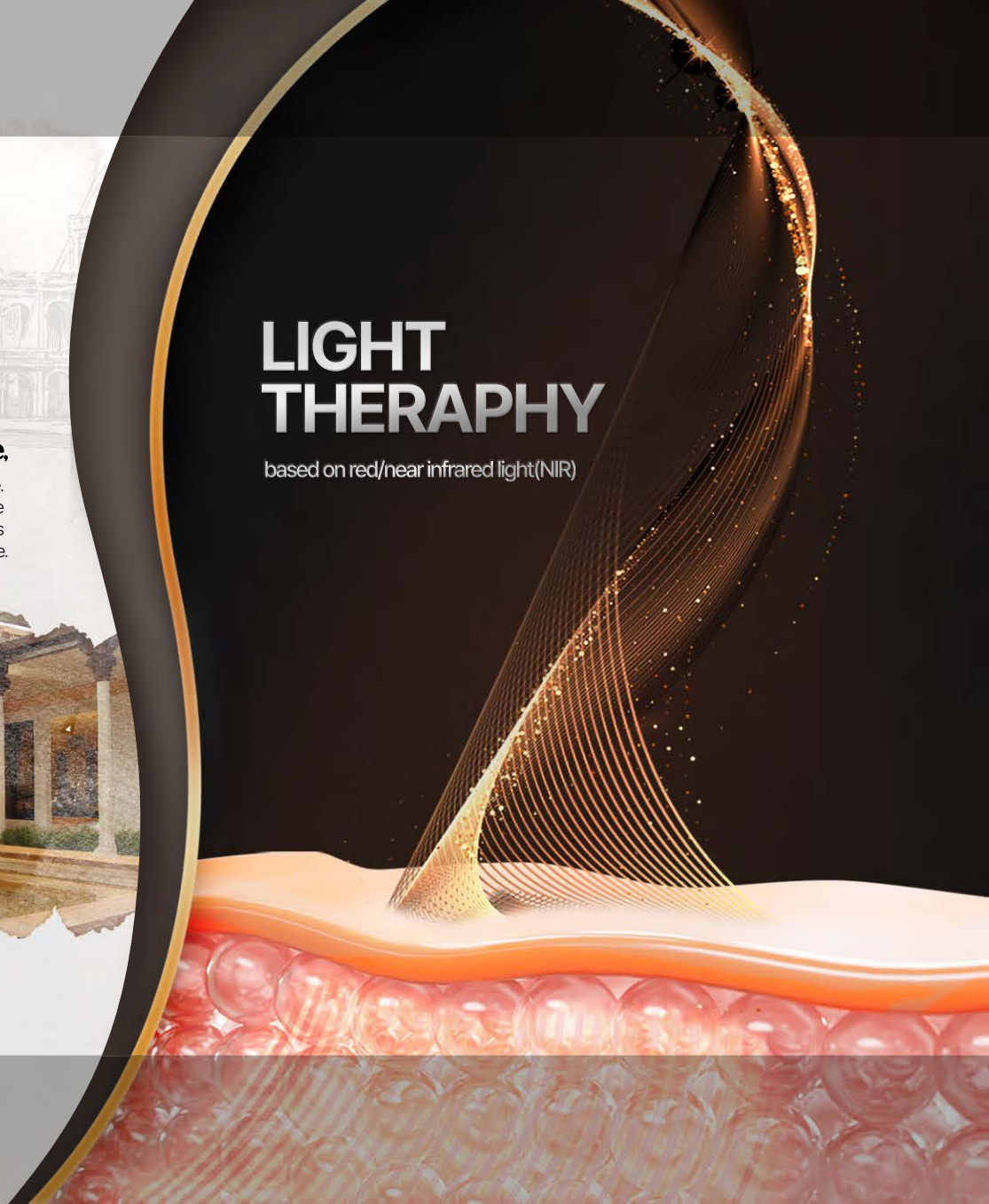

WHAT IS PHOTOBIOMODULATION?

1

The human body consists of cells.

Cells are made up of thin membranes and the inside is composed of various organs.



The thin membrane surrounding the cells

has channels that control the passage of various substances, and they either allow or block substances through them.

2

3

Light is a kind of waves.



The color looks different depending on the length of the wavelength and has a different amount of energy for each wavelength range.

The light that our eyes see

as red has a wavelength of 600 nm to 750 nm.

Although invisible to our eyes, near infrared rays (NIR)


have a wavelength of 750nm to 1400nm.

4

5

There is a chromophore (cytochrome c oxidase) that absorbs

red wavelengths in mitochondria. Cytochrome c oxidase triggers the activity of various molecules such as nitric oxide, ATP, calcium ions, and ROS.

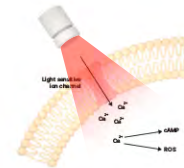

In the case of the near-infrared region,

the Ca²⁺ level increases through the activation of ion channels, and ROS and cAMP interact with calcium ions.

6

[ref] Dorpe, Claudia, et al. "Photobiomodulation underlying mechanism and clinical applications." Journal clinical medicine 9.6 (2020): 1724.

This information is for understanding the product and is not related to the product.

EASY CARE with KAISAR!

#One-touch

#Wireless

#Regardless of body size



Point 01

Easy to use at home with one touch button

Automatic shutdown
after 15 minutes of operation
with vibration notification

Point 02

Light but sturdy and stable design

Luxurious hinge structure
and curved shape design



Point 03

Wireless devices that do not require a separate device

Charging anywhere with USB-C type



Point 04

Regardless of body size due to hinge

Ergonomic design can be worn
by any male regardless of size





Glory to your life

Reliable KAISAR R&D system



01

.

We maintain a quality management system through strict production control.



02

.

It is a company specializing in PBM research, such as carrying out support projects for university hospitals.



03

.

We continue to research the men's healthcare field.



04

.

6 Domestic applications
2 PCT applications
2 Madrid international applications

Product Features



Check 1

Patent/PCT/trademark
application for wearing device



Check 2

Thorough inspection
and quality control



Check 3

15 minutes a day
Easy Daily Care



Check 4

Light care optimized
for male penis and testicles




Check 5

Ergonomically designed
for men to use


How to use

1

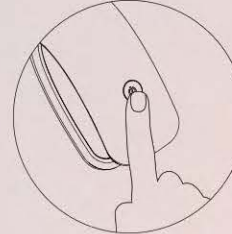

Press the power button to activate the Product.
The white LED on the power button
lights up and the product is operating.

2

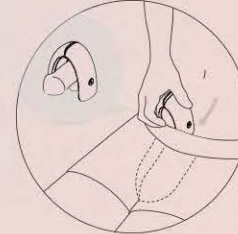


With the product running, place the product
inside the underwear so that it is placed on
the main part of the man.

3




Caring for 15 minutes.
It is recommended to use for
15 minutes a day, 5 times a week.

4

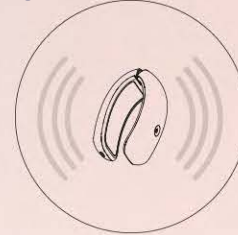

It automatically shuts off after 15 minutes,
and when it shuts down, you can know
that it has shut down through vibration.

Product configuration

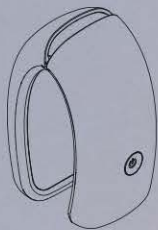

Main unit




USB-C type charging cable



User manual

Name of Each Part

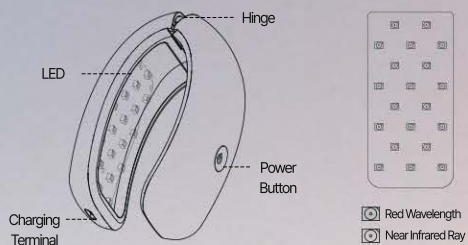

In the case of near-infrared rays, they are invisible even during operation.
Please use it with confidence as it is not a malfunction of the device.

Product Specification

Product name	KAISAR
Device name	Health care device for men's home care
Model name	HOMME FORCE KR001
Charging power	DC 5V USB-C type charger
Built-in battery	Lithium polymer 3.7V / 600mAh
Number of LEDs	40 (24 RED; 16 NIR)
Size (mm)	(W)48 x (D)62 x (H)96
Weight (range of error: ±5g)	Less than 104g(±5g)
Manufacturer / Country of origin	DARAM Inc. / Made in Korea

CAUTION

- Do not subject the device to impact or place excessive weight on it.
Do not arbitrarily separate or modify the inside of the device.
- This product may be vulnerable to excessive moisture or sweat. Use in as dry a condition as possible.
- Remove any precious metals from the parts that come into contact with the device before use.
- Chemicals such as alcohol, benzene, and acetone are not used.
- Please keep out of reach of children and be careful not to operate the device.
- Never use force to spread the body beyond the set angle.

Glory to your life



Free shipping on
all products



Search [kaisar_korea](#)
on Instagram



Official website
www.kaisar.co.kr



Global Men's Home Care Device

KAI2AR

