

## USER'S MANUAL OF MODEL 400639

**Dealer:** This manual **MUST** be given to the user of the Shower Chair/Bench.

**USER:** Before using this Shower Chair/Bench, read this manual and save for future reference.

---

### GENERAL WARNINGS

Rubber tips **MUST** be in contact with tub floor at all times.

Always inspect the shower chair/bench to ensure that is properly positioned and stable before using. **DO** not use if when it is wobbly or unstable.

Check rubber tips for rips, tears, crack or wear. If any of these conditions exist, replace rubber tips immediately.

The Shower Chair/Bench has a weight limitation of 100 kg.

---

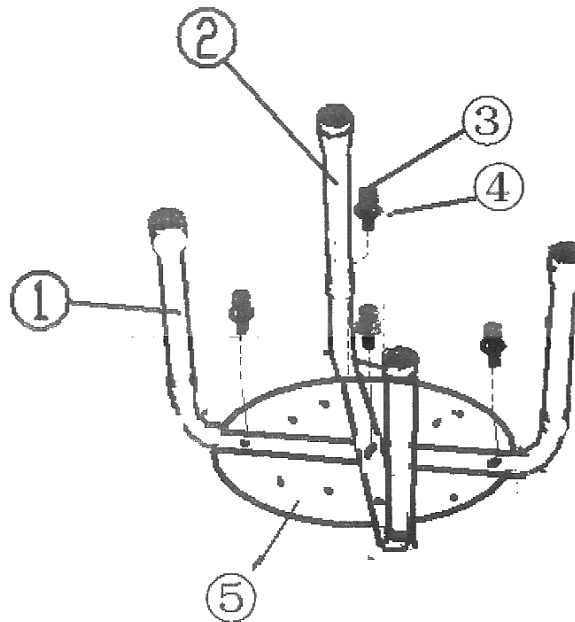


Figure 1



### ASSEMBLING

1. Put seat upside down on the ground /floor.
2. Align seat legs on the seat accordingly the number indicated on the legs and seat.
3. Install mounting screw through vaulted washer, top leg, bottom leg and tighten on the seat. Refer to Figure 1.

### ADJUSTING LEG EXTENSION

1. Select a hole which suitable to you on the lower leg frame.
2. Depress the snap button and slide the adjustable legs up or down to the desire height.
3. Ensure that the snap button fully protrude the tube's hole.
4. REPEAT STEP 2. - 3. for the other legs

**Make sure all four legs are adjusted to the same height so that the shower chair/bench sit level.**

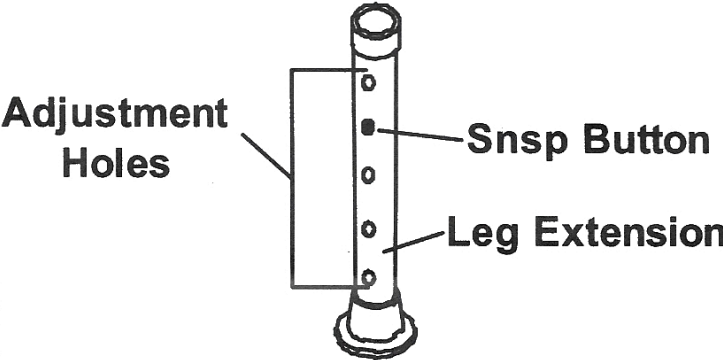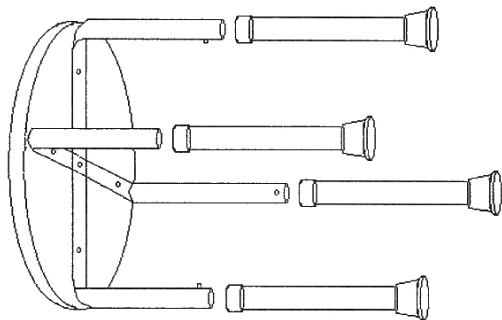

Item	Description	Unit
①	Bottom leg 1	1
②	Bottom leg 2	1
③	Screw M6*35	5
④	Vaulted washer	5
⑤	Seat	1
⑥	Leg extension	4
⑦	Instruction manual	1

Hardware Package :

BUY ON  
[www.cablematic.com](http://www.cablematic.com)

**NOTE: MAKE SURE ALL FOUR LEGS ARE ADJUSTED TO THE SAME HEIGHT AND ENSURE THE SEAT OPENED TO FULL EXPANSION BEFORE USE.**