

Dealer: This manual **MUST** be given to the user of the Shower Chair/Bench.

USER: Before using this Shower Chair/Bench, read this manual and save for future reference.

GENERAL WARNINGS

Rubber tips **MUST** be in contact with tub floor at all times.

Always inspect the shower chair/bench to ensure that is properly positioned and stable before using. **DO** not use if when it is wobbly or unstable.

Check rubber tips for rips, tears, crack or wear. If any of these conditions exist, replace rubber tips immediately.

The Shower Chair/Bench has a weight limitation of 110 kg.

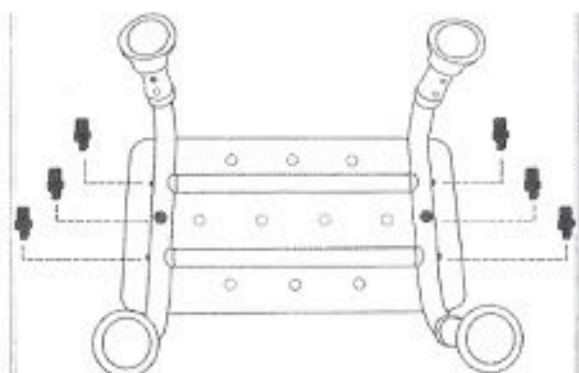


Figure 1

ASSEMBLING


1. Put seat upside down on the ground /floor.
2. Align seat legs on the seat accordingly the number indicated on the legs and seat.
3. Install mounting screw through vaulted washer, top leg, bottom leg and tighten on the seat. Refer to Figure 1.

BUY ON
www.cablematic.com

ADJUSTING LEG EXTENSION

1. Select a hole which suitable to you on the lower leg frame.
2. Depress the snap button and slide the adjustable legs up or down to the desire height.
3. Ensure that the snap button fully protrude the tube's hole.
4. REPEAT STEP 2. - 3. for the other legs

Make sure all four legs are adjusted to the same height so that the shower chair/bench sit level.

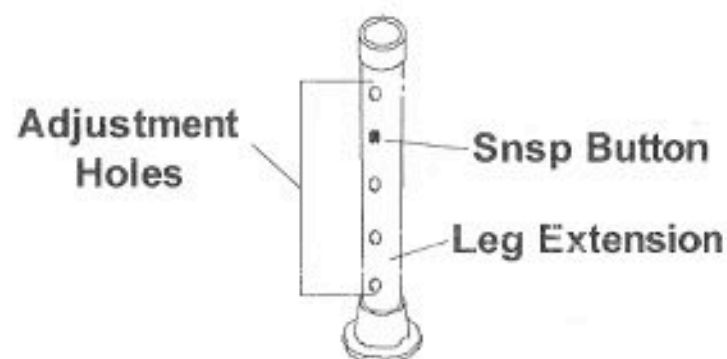

Hardware Package :

NOTE: MAKE SURE ALL FOUR LEGS ARE ADJUSTED TO THE SAME HEIGHT AND ENSURE THE SEAT OPENED TO FULL EXPANSION BEFORE USE.

Item	Description	Unit
①	Bottom leg	2
②	Adjustable leg	4
③	Screw M6*40	6
④	Vaulted washer	5
⑤	Seat	1