

## USER'S MANUAL OF MODEL 500406

**Dealer:** This manual **MUST** be given to the user of the Shower Chair/Bench.

**USER:** Before using this Shower Chair/Bench, read this manual and save for future reference.

### GENERAL WARNINGS

Rubber tips **MUST** be in contact with tub floor at all times.

Always inspect the shower chair/bench to ensure that is properly positioned and stable before using. **DO** not use if when it is wobbly or unstable.

Check rubber tips for rips, tears, crack or wear. If any of these conditions exist, replace rubber tips immediately.

The Shower Chair/Bench has a weight limitation of 100 kg

### ASSEMBLING

1. Put seat upside down on the ground /floor.
2. Align seat legs on the seat accordingly the number indicated on the legs and seat.
3. Install mounting screw through vaulted washer, top leg, bottom leg and tighten on the seat. Refer to Figure 1.

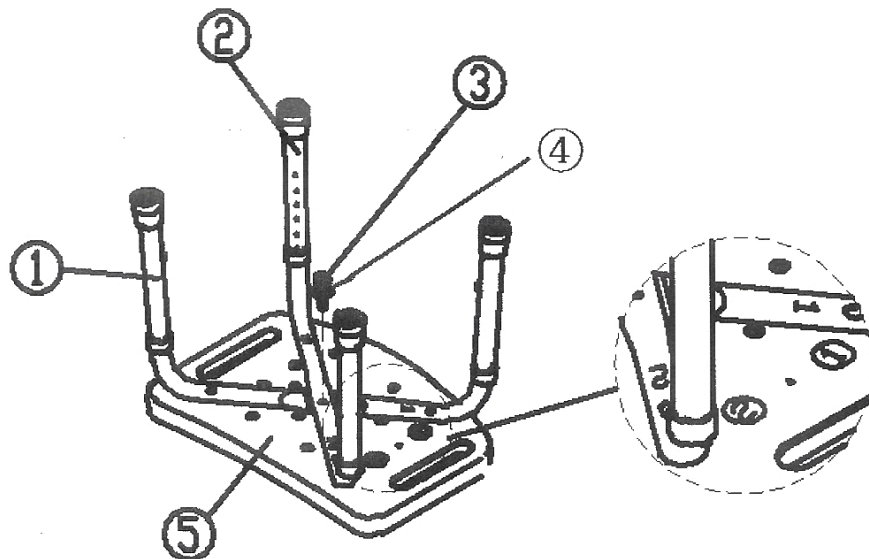


Figure 1

4. Align back support tubes above the seat legs. Refer to Figure 2.
5. Install mounting screw through top leg or bottom leg and tighten on the seat. Refer to figure 3.

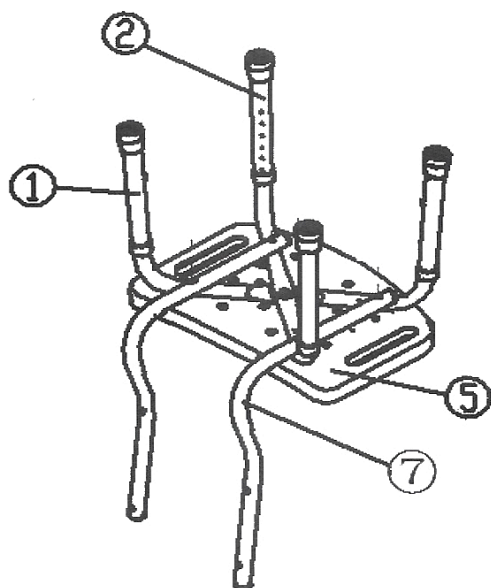


Figure 2

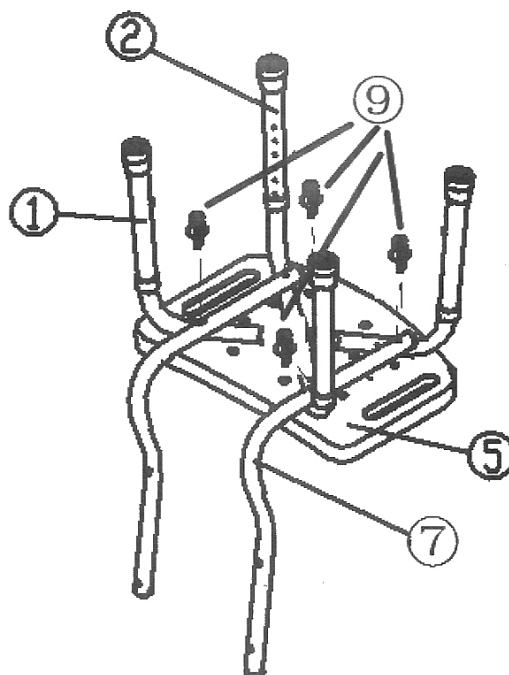


Figure 3

6. Align the back to back support tubes. Loosely install the four mounting screws through vaulted washers, the back support tubes and into the back. Refer to figure 4.
7. Once all mounting screws are insert, tighten the screws firmly, but do not force. If tightening screw too firmly may damage the units.

Hardware Package :

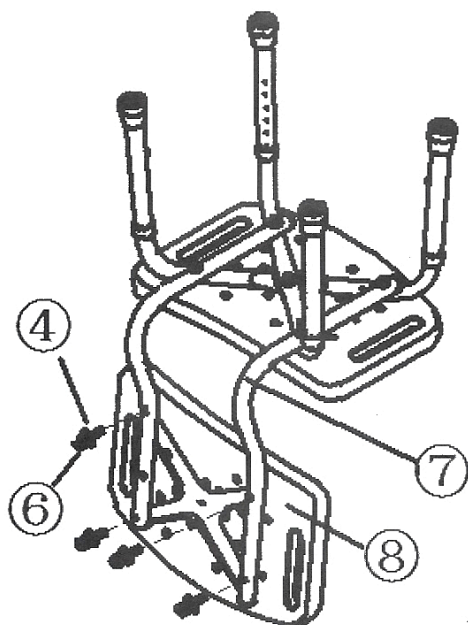


Figure 4

Item	Description	Unit
①	Bottom leg 1	1
②	Bottom leg 2	1
③	screw M6*40	1
④	Vaulted washer	5
⑤	Seat	1
⑥	Short screw M6*30	4
⑦	Back support tube	2
⑧	Back	1
⑨	Long screw M6*60	4

**NOTE:** MAKE SURE ALL FOUR LEGS ARE ADJUSTED TO THE SAME HEIGHT AND ENSURE THE SEAT OPENED TO FULL EXPANSION BEFORE USE.