


INSTALLATION INSTRUCTIONS

FOR Shower Seat Models C&D

Before installing, please read the instructions carefully in order to install and use this product safely.

Note:

The structure and installation method for Model C is the same as for Model D.

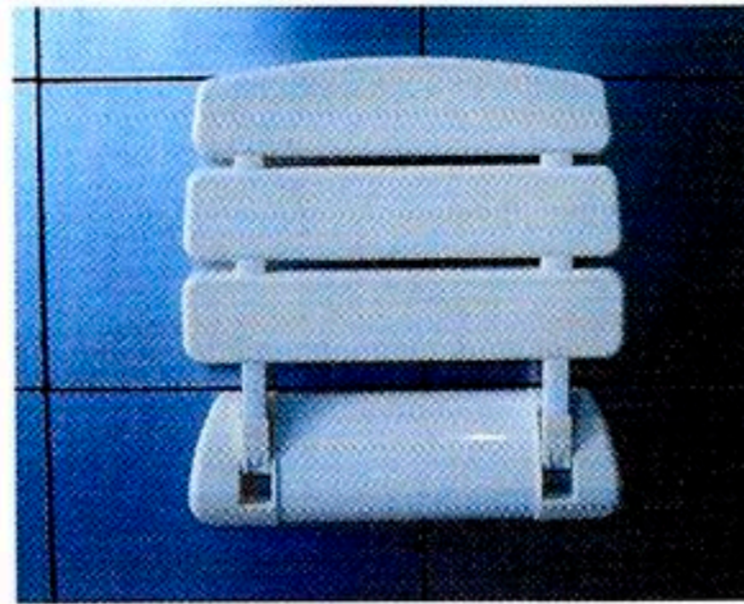


MODEL C




MODEL D

Spare parts List:

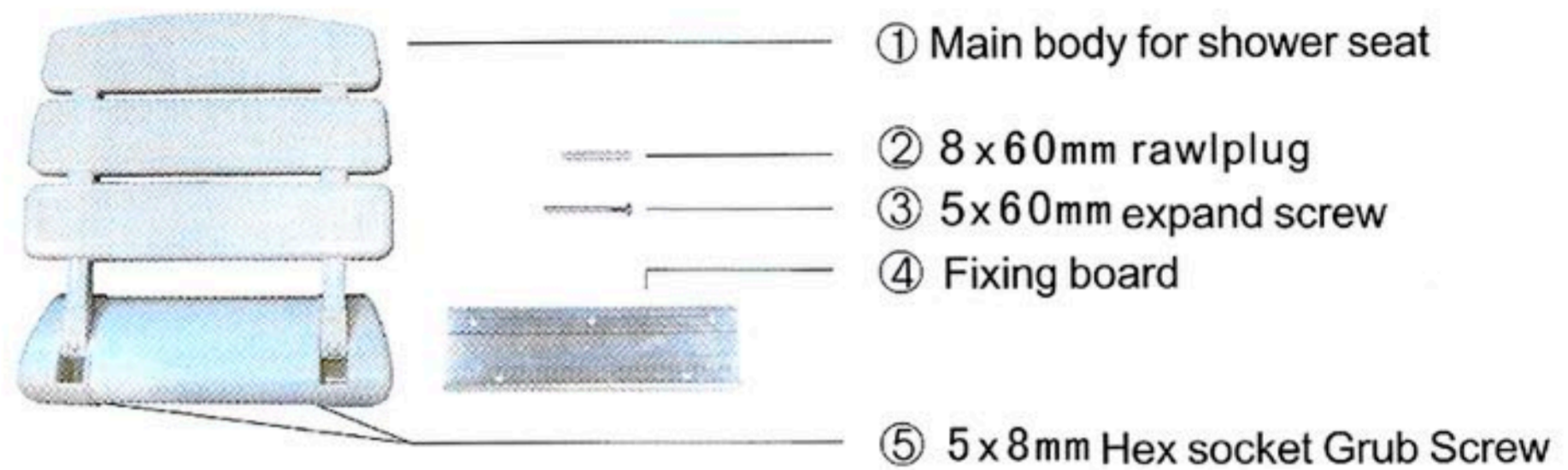

Installing steps:

- A. keep the plane of Fixing board [04] towards the wall and place it on the wall level (Note: the edge with three holes of Fixing board [04] should be up) and the recommended upright height from the bottom of Fixing board [04] to Floor is 400-500mm. Mark down the screw holes and take down the Fixing board [04] so as to drill five holes with the depth of 60mm and diameter of 8 mm and place the rawlplug [02] into the holes.
- B. Put the Fixing board [04] onto the original place once again and screw down the expand screw [03] and fasten them.
- C. Button the Main body for shower seat [01] downwards absolutely from the upper part of Fixing board [04] until the inner side on the bottom of Main body for shower seat [01] touches the wall and keep the pedestal made of aluminium alloy material on the Main body for shower seat [01] and the two ends of Fixing board [04] orderly.
- D. Screw down two pcs of 5*8mm Hex socket Grub Screw [05] inserted on the underside of Main body for shower seat [01] with the supplied Hex socket spanner.

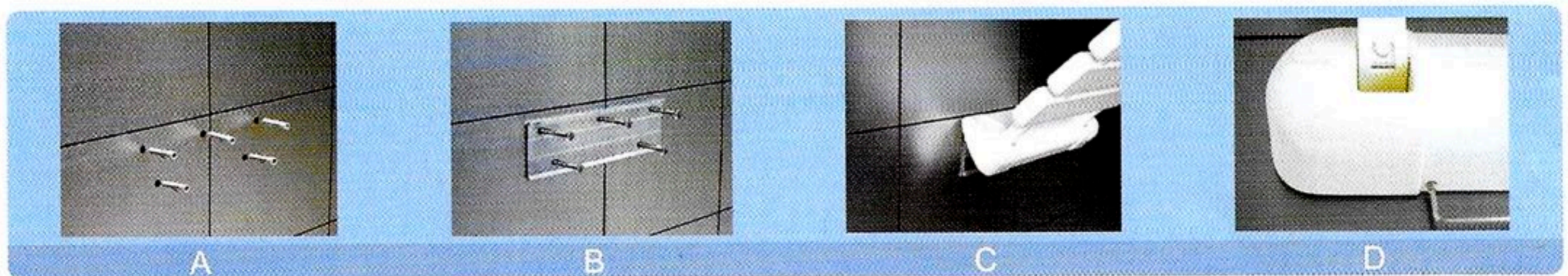

Dimensional drawing:



Packing List:

Serial No.	Name of components	Quantity	Remark
01	Main body for shower seat	1	
02	8*60mm rawlplug	5	
03	5*60mm expand screw	5	
04	Fixing board	1	
05	Hex socket spanner	1	
06	Instructions	1	

Important:

1. Safe Working Load **160** kilogram.
2. If the Supplied Screws/Raw plugs are not Suitable for Installation you must select appropriate ones.