bébéconfort® 

Dual thermometer : forehead and ear

3200 0142

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EN : Instructions for use

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EN : DUAL THERMOMETER : FOREHEAD / EAR

This thermometer is a high quality product incorporating the latest technology and tested in accordance with international standards. With its unique technology, this thermometer can provide a stable, heat-interference-free reading with each measurement. The instrument performs a self-test every time it is switched on to always guarantee the specified accuracy of measurements. This thermometer is intended for the periodic measurement and monitoring of human body temperature in the home. It is intended for use on people of all ages.

This thermometer has been clinically tested and proven to be safe and accurate when used in accordance with its operating instruction manual

Please carefully read this guide before using the thermometer and keep it in a safe place for reference.

- ① Measuring Sensor / Removable Cap
- ② START Button
- ③ Display
- ④ ON/OFF Button
- ⑤ Battery Compartment Cover
- ⑥ All Segments displayed
- ⑦ Memory
- ⑧ Ready for Measurement
- ⑨ Measurement complete
- ⑩ Surface Temperature Indication
- ⑪ Low Battery Indication
- ⑫ Changing between Celsius and Fahrenheit
- ⑬ Recall Mode
- ⑭ Recall the last 12 Readings
- ⑮ Measured Temperature too high
- ⑯ Measured Temperature too low
- ⑰ Ambient Temperature too high
- ⑱ Ambient Temperature too low
- ⑲ Error Function Display
- ⑳ Blank Display
- ㉑ Flat Battery
- ㉒ Replacing the Battery

1. THE ADVANTAGES OF THIS THERMOMETER

Multiple Uses (Wide Range Measurement)

This thermometer offers a wide measurement range feature from 0°C to 100.0°C (32.0°F to 212.0°F), meaning the unit can be used as a forehead or an ear thermometer to measure body temperature or it can be used to measure surface temperature of the following :

- Milk surface temperature in a baby's bottle
- Surface temperature of a baby's bath
- Ambient temperature

Probe Cover Free

This thermometer is more user-friendly and more cost-effective since a probe cover is not required.

Measurement in a matter of seconds

The innovative infrared technology allows measurement of ear temperature in only 1 second and forehead temperature in only 3 seconds.

Accurate and reliable

The unique probe assembly construction, incorporating an advanced infrared sensor, ensures that each measurement is accurate and reliable.

Gentle and Easy to Use

- The ergonomic design enables simple and easy use of the thermometer.
- This thermometer can even be used on a sleeping child, thereby causing no disruption.
- This thermometer is quick, therefore child-friendly.

Auto-Display Memory

The last reading is automatically displayed for 2 seconds when the unit is switched ON.

Multiple Reading Recall

temperature variations. entering the recall mode, enabling efficient tracking of Users will be able to recall the last 12 readings when.

Safe and Hygienic

- No risk of broken glass or mercury ingestion.
- Completely safe for use on children.
- Cleaning of the probe can be done with an alcohol-moistened cotton tissue, making this thermometer completely hygienic for use by the whole family.

Fever Alarm

10 short beeps and a red LCD backlight alert the patient that he/she may have a temperature above 37.5 °C.

2. IMPORTANT SAFETY INSTRUCTIONS

- Never immerse this instrument in water or other liquids. For cleaning please follow the instructions in the «Cleaning and Disinfecting» section.
- Do not use the instrument if you think it is damaged or notice anything unusual.
- Never open the instrument.
- Earwax in the ear canal may cause a lower temperature reading. Therefore it is important to ensure the subject's ear canal is clean.
- A basic physiological effect called vasoconstriction can occur in the early stages of fever, resulting in a cool skin effect. The recorded temperature using a forehead thermometer can, therefore, be unusually low.
- If the measurement result is not consistent with the patient's finding or unusually low, repeat the measurement every 15 minutes or double check the result by another core body temperature measurement.
- This instrument comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section!
- Ensure that children do not use the instrument unsupervised; some parts are small enough to be swallowed.
- Protect it from:
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold
- If the instrument is not going to be used for a prolonged period the battery should be removed.

WARNING : Use of this instrument is not intended as a substitute for consultation with your physician. This instrument is NOT waterproof so do not immerse in liquids.

3. HOW THIS THERMOMETER MEASURES TEMPERATURE

This thermometer measures forehead and ear temperature.

Forehead measurement

This thermometer measures infrared energy radiated from the skin above the eyebrow area and the surrounding tissue. This energy is collected through the lens and converted to a temperature value. Temperature readings obtained by scanning above the eyebrow area will provide the greatest accuracy.

Ear measurement

This thermometer measures infrared energy radiated from the eardrum and the surrounding tissue. This energy is collected through the lens and converted to a temperature value. The measured reading obtained directly from the eardrum (Tympanic Membrane) ensures the most accurate ear temperature. Measurements taken from the surrounding tissue of the ear canal generate lower readings and may result in misdiagnosis of fever.

To avoid an inaccurate measurement

1. Switch on the thermometer by pressing the ON/OFF button ④
2. After one beep is heard (and the temperature scale icon is flashing), straighten the ear canal by gently pulling the middle of the ear back and up.
3. Place the probe ① firmly into the ear canal, press the START button ② and keep the probe in the ear until the thermometer beeps to signal the completion of the measurement.

Measurement of a liquid/ambient temperature

- To measure a household temperature (surface, liquid or ambient), remove the cap from the end of the thermometer
- Any temperature measured outside the 32°C-42.2°C range is automatically recognised by the thermometer as being a household temperature
- Measurement distance: 5cm maximum from the surface to be measured

Measurement of the temperature of a liquid

- The temperature on the surface of a liquid can be very different from that in the middle of the bottle, the bath or the soup. To avoid any accidents, you must thoroughly stir all liquids before proceeding to the measurement.
- In the case of measurement through a container (bottle for example), take into account the inertia of the container.
- Measure the temperature as quickly as possible to avoid the formation of condensation on the thermometer lens.
- Never submerge the thermometer in the liquid; it is not waterproof.

Ambient temperature

- Measure the temperature in the middle of the room away from heating sources (radiator, fireplace), from cold (window) and away from draughts.
- Do not point the lens towards a strong light source.

4. CONTROL DISPLAYS AND SYMBOLS

- All segments displayed ⑥ : Press the ON/OFF button ④ to turn on the unit; all segments will be shown for 2 seconds.
- Memory ⑦ : The last reading will automatically be shown on the display for 2 seconds.
- Ready for measurement ⑧ : When the unit is ready for measurement, the «°C» or «°F» icon will keep flashing while the mode icon (ear or forehead) will be displayed.
- Measurement complete ⑨ : The reading will be shown on the display ③ with the «°C» or «°F» icon and the mode icon steady. The unit is ready for the next measurement as soon as the «°C» or «°F» icon is flashing again.
- Surface temperature indication ⑩ : A crossed-ear icon will appear on the display ③ when the reading in the ear mode is outside the range 32.0 - 42.2°C (89.6 - 108.0°F).
- Low battery indication ⑪ : When the unit is turned on, the battery icon will keep flashing to remind the user to replace the battery.

5. CHANGING BETWEEN FOREHEAD AND EAR MODE

To change from forehead to ear mode, take off the removable cap ① from the thermometer tip. The device will immediately switch to ear mode. The ear icon will be shown on the display and the «°C» or «°F» icon will flash to indicate that the device is ready for measurement.

To change back to the forehead mode, place the removable cap ① on the thermometer tip and the device will immediately switch back to forehead mode. The forehead icon will be shown on the display and the «°C» or «°F» icon will flash to indicate that the device is ready for measurement.

6. DIRECTIONS FOR USE

1. Press the ON/OFF button ④. The display ③ is activated to show all segments for 2 seconds.
2. The last measurement reading will automatically be shown on the display for 2 seconds with the «M» icon ⑦.
3. When the «°C» or «°F» icon is flashing, a beep sound is heard and the thermometer is ready for measurement ⑧.

Forehead measurement

- Place the probe gently onto the area approximately 1cm above the center of the eyebrow. If the eyebrow area is covered with hair, sweat or dirt, please remove the obstacle beforehand to improve the reading accuracy.
- Press the START button once and release it. Start to move the instrument steadily from 1 cm above the center of the eyebrow to the temple area. A long beep sound will emit after 3 seconds to acknowledge the completion of measurement. If the probe movement is too slow to reach the temple area before the long beep sound is heard, repeat the measurement as described above, but simply move the probe a little quicker this time.
- Read the recorded temperature from the LCD display.

NOTE:

- Patients and thermometer should stay in steady-state room condition for at least 30 minutes.
- Don't take a measurement while or immediately after nursing a baby.
- Don't use the thermometer in high humidity environments.
- Patients should not drink, eat, or exercise before/while taking the measurement.
- Before placing the thermometer probe onto the measurement area, remove dirt, hair, or sweat.
- Don't move the measurement device from the measuring area before hearing the termination beep.
- Always take the temperature in the same location, since temperature readings may vary according to location.
- 10 short beeps and a red LCD backlight alert the patient that he/she may have a temperature above 37.5 °C.

Ear measurement

- Straighten the ear canal by pulling the ear up and back to give a clear view of the eardrum.
- For children under 1 year; pull the ear straight back.
- For children of 1 year to adult; pull the ear up and back. Also refer to the short instructions at the front!
- While gently pulling the ear, insert the probe snugly into the ear canal (max. 1 sec.) and immediately press the START button ②. Release the button and wait for the beep sound that confirms the end of measurement.
- Remove the thermometer from the ear canal. The display shows the measured temperature ⑨.

NOTE:

- To ensure accurate readings, wait at least 30 sec. after 3-5 continuous measurements.
- Accumulation of ear wax on the probe can result in less accurate temperature readings or cross infection between users. Therefore, it is essential that the probe is clean before each measurement. For cleaning, follow the instructions in the «Cleaning and Disinfecting» section.
- After cleaning the measuring sensor ① with alcohol, wait 5 minutes before taking the next measurement, in order to allow the thermometer to reach its operating reference temperature.
- For an infant, it is best to have the child lying flat with his/her head sideways so the ear is facing upwards. For an older child or adult, it is best to stand behind and slightly to the side of the patient.
- Always take the temperature in the same ear, since the temperature readings may be different from ear to ear.
- Wait for a few minutes to take the ear temperature after sleeping.
- In the following situations it is recommended that three temperatures in the same ear be taken and the highest one taken as the reading:

1. New born infants in the first 100 days.
2. Children under three years of age with a compromised immune system and for whom the presence or absence of fever is critical.
3. When first-time users are learning how to use the thermometer until they have familiarized themselves with the instrument and obtain consistent readings.
4. If the measurement is surprisingly low.

7. CHANGING BETWEEN CELSIUS AND FAHRENHEIT

This thermometer can display temperature measurements in either Fahrenheit or Celsius. To switch the display between °C and °F, simply turn OFF the unit, press and hold the START button ②. This thermometer can display temperature measurements in either Fahrenheit or Celsius. To switch the display between °C and °F, simply turn OFF the unit, press and hold the START button ② for 5 seconds; after 5 seconds, the current measurement scale («°C» or «°F» icon) will be flashing on the display ⑬. Change the measurement scale between °C and °F by pressing the START button ②. When the measurement scale has been chosen, wait for 5 seconds and the unit will automatically enter the «ready for measuring» mode.

8. HOW TO RECALL 12 READINGS IN MEMORY MODE

This thermometer can recall the last 12 readings.

- **Recall mode** ⑭: Press the START button ② to enter Recall mode when power is off. The memory icon «M» will flash.
- **Reading 1 - the last reading** ⑮: Press and release the START button ② to recall the last reading. Display 1 alone with memory icon.
- **Reading 12 - readings in succession**: Press and release the START button ② consecutively to recall the last 12 readings in succession. Pressing and releasing the START button ② after the last 12 readings have been recalled will resume the above sequence from reading 1.

9. ERROR MESSAGES

- **Measured temperature too high** ⑯: Displays «H» when measured temperature is higher than 100.0°C (212.0°F) in the ear mode or 42.2°C (108.0°F) in the forehead mode.
- **Measured temperature too low** ⑰: Displays «L» when measured temperature is lower than 0°C (32.0°F) in the ear mode or 34.0°C (93.2°F) in the forehead mode.
- **Ambient temperature too high** ⑱: Displays «H» in conjunction with the «▲» when ambient temperature is higher than 40.0°C (104.0°F).
- **Ambient temperature too low** ⑲: Display «L» in conjunction with the «▼» when ambient temperature is lower than 5.0°C (41.0°F) in the ear mode or 16°C (60.8°F) in the forehead mode.
- **Error function display** ⑳: The system has a malfunction.
- **Blank display** ㉑: Check if the battery has been loaded correctly. Also check polarity («+» and «-») of the battery.
- **Flat battery indication** ㉒: If the steady battery icon is the only symbol shown on the display, the battery should be replaced immediately.

10. CLEANING AND DISINFECTING

Use an alcohol swab or cotton tissue moistened with alcohol (70% Isopropyl) to clean the thermometer casing and the measuring probe. Ensure that no liquid enters the interior of the thermometer. Never use abrasive cleaning agents, thin-ners or benzene for cleaning and never immerse the instrument in water or other cleaning liquids. Take care not to scratch the surface of the probe lens and the display.

11. BATTERY REPLACEMENT

This instrument is supplied with one lithium battery, type CR2032. Replace with a new CR2032 battery when the flashing battery symbol appears on the display ⑳. Remove the battery cover by sliding it in the direction shown. Remove the battery and replace with a new one ㉓.

Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

12. TECHNICAL SPECIFICATIONS

Measurement range: Ear Mode: 0-100.0 °C (32-212.0 °F). Forehead Mode: 34-42.2 °C (93.2-108.0 °F).

Resolution: 0.1 °C / °F.

Measurement accuracy: Laboratory: ±0.2 °C, 32.0 - 42.2 °C. (±0.4 °F, 89.6 - 108.0 °F).

Display: Liquid Crystal Display, 4 digits plus special icons.

Acoustic:

- The unit is turned ON and ready for the measurement: 1 short beep.
- Complete the measurement: 1 long beep (1 sec.) if the reading is less than 37.5°C (99.4°F), 10 short «beep» sounds, if the reading is equal to or greater than 37.5°C (99.4°F).
- System error or malfunction: 3 short «bi» sounds.

Memory:

- Auto-Display the last measured temperature.
- 12 readings recall in the Memory Mode.

Backlight:

- The display light will be GREEN for 4 seconds, when the unit is turned ON.
- The display light will be GREEN for 5 seconds, when a measurement is completed with a reading less than 37.5°C (99.4°F).
- The display light will be RED for 5 seconds, when a measurement is completed with a reading equal to or higher than 37.5°C (99.4°F).

Operating temperature: Ear Mode: 5-40.0 °C (41-104.0 °F). Forehead Mode: 16-40.0 °C (60.8-104.0 °F); 15 - 95 % relative maximum humidity

Storage temperature: -25°C to +55°C (-13°F to 131°F); 15 - 95 % relative maximum humidity

Automatic Switch-off: Approx. 1 minute after last measurement has been taken.

Battery: CR2032 Battery (X1) 3V - at least 1000 measurements.

Dimensions: 107 x 50 x 34 mm.

Weight: 55g (with battery), 52g (w/o battery).

13. WARRANTY

This product is warranted for a two-year period from the purchase date. However, this warranty does not apply to damages, in particular, resulting from non-compliance with this guide, incorrect manipulations, shocks or inappropriate installation of the battery. It does not apply either to the battery or the package.

This device complies with the requirements of the Medical Device Directive 93/42/EEC. Technical alterations reserved.

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MANDATAIRE : MICROLIFE AG Espenstrasse 139,
IMPORTATEUR : MAGNIEN SAS
DOREL FRANCE S.A.S. - Z.I. - 9 bd du Poitou, BP 905, 49309 Cholet Cedex, FRANCE
DOREL SWITZERLAND S.A. - Chemin de la Colice 4, 1023 Crissier, SWITZERLAND /
DOREL HISPANIA S.A.U. - Edificio Barcelona Moda Centre, Ronda Maiols, 1, Planta 4ª,
Microlife Corporation Taipei - TW