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EN : Recommendations for use

Cleaning the baby's nose with the Bébé Confort nasal aspirator is recommended at least once a day, particularly before a feed. In the event of infections with a very blocked nose, cleaning the nose is recommended up to six times a day to relieve congestion.

1. Reversible baby nasal aspirator

• Newborn for 0 to 4 months

Assembling the machine

- Insert a new disposable nozzle in the blue nose piece,
- Insert the free end of the disposable nozzle in the flexible suction tube.

Method of use

- Lay the baby on his back,
- Place the blue nose piece at the entrance to the child's nostril,
- Suck through the mouth piece,
- Do the same for the other nostril.

After use

- Separate the 2 nozzles,
- Rinse the blue nose piece,
- Discard the disposable nozzle.

• Baby aged 4 months and over

Assembling the machine

- Insert a new disposable nozzle in the blue nose piece,
- Insert the free end of the blue nose piece in the flexible suction tube.

Method of use

- Lay the baby on this back,
- Place the disposable nozzle at the entrance to the child's nostril,
- Suck through the mouthpiece,
- Do the same for the other nostril.

After use

- Separate the 2 nozzles,
- Discard the disposable nozzle.

2. Disposable nozzle

Method of use

- Insert a new disposable nozzle in the blue nose piece of the baby nasal aspirator,
- After cleaning the baby's nose, remove and discard the disposable nozzle.

To use the baby nasal aspirator refer to the Paragraph «1. Reversible baby nasal aspirator» depending on the age of the child.

Keep out of reach and sight of children. Store away from heat.

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