SUMMARY

ENGLISH

RAYPLICKER™ DEVICE - QUICK START	3
RAYPLICKER™ DEVICE - DIRECTIONS FOR USE	4
RAYPLICKER™ SOFTWARE - QUICK START	5

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
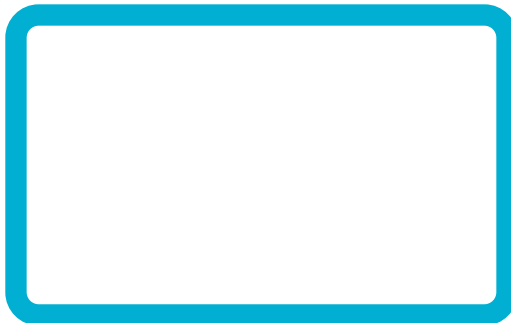
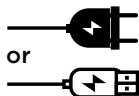
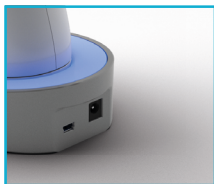
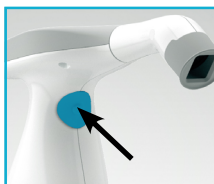
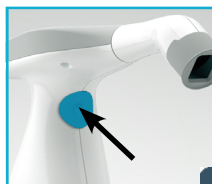
5

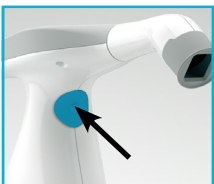
Make the acquisition

- Do not move during the acquisition (1 sec)
- Best practices p.4

6

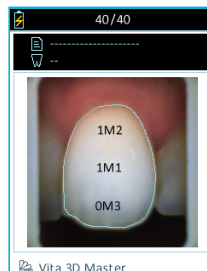Displaying data


Power source

Lifespan: 5 years
Access/replacement: only BOREA

Further information on www.BOREA.dental

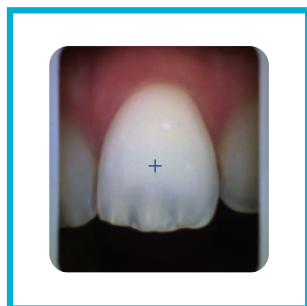Best practices

- ✓ Grey edges parallel to screen edges
- ✓ Mouth opened and breathing through the nose
- ✓ Saliva limitation
- ✓ Centering the tooth, without angle nor inclination
- ✓ Tip under the lip
- ✓ Tip in contact with adjacent teeth
- ✓ Tip in contact with the gingiva

Incorrect practices

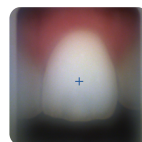
Tip incorrectly positioned

Mouth closed

Condensation

Excessive pressure on the gingiva

Tooth centered incorrectly

Wrong angle

Wrong inclination

Visible lip

Wrong support

Software downloadable on:

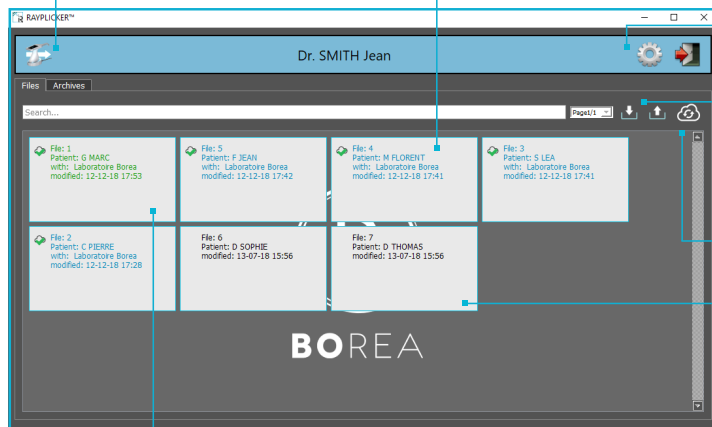
www.borea.dental/Download

I. MAIN PAGE

Import files from
Raypicker™

Order sent to the lab
(Blue File)

Settings

Import / Export

Cloud actualization

Untreated order
(Black file)

Finished order
(Green file)


II. ANALYSIS TAB

Referent tooth selection

Color mapping display

Image treatment tool:

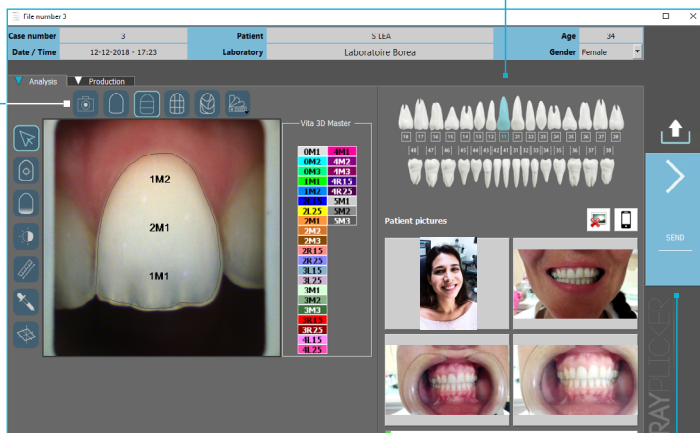- Tooth border
- Translucency
- B&W
- Ruler
- Cie Lab Analysis
- Colorimetric coordinates


Send the file

III. PRODUCTION TAB

Production tooth selection

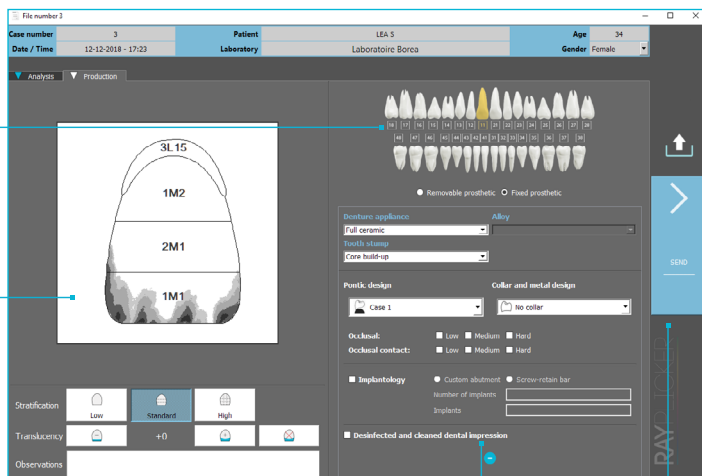Scheme of the production tooth

Information about the production tooth

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
MOBILE APPLICATION - QUICK START


Raypicker™ Pics App Solution

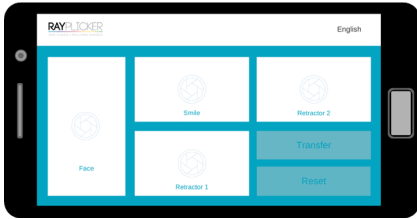Download the Raypicker™ application on Google Play or Apple Store

1

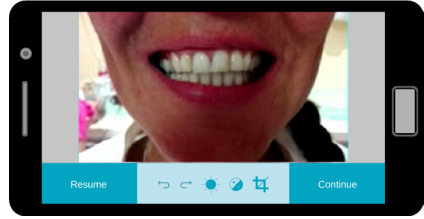
First, open the Raypicker™ application and take patient pictures.

2


Bottom menu bar allows pictures enhancements :

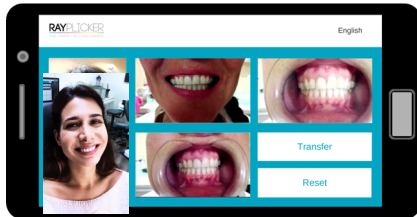Brightness
Contrast
Cropping

3

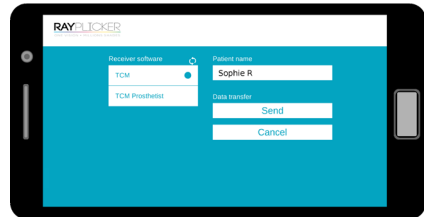
When complete, press «transfer».


4

Write a patient name and then select a software receiver* before sending.
Pictures will be automatically downloaded into the Raypicker™ software by wi-fi.

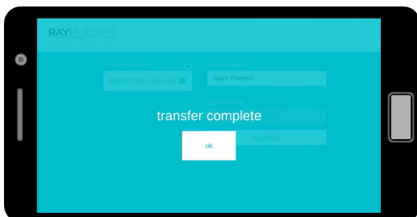*Important: Make sure both your devices (phone and software) are connected at the same time to the same wi-fi network.

5

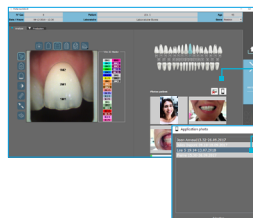Click here to import pictures.

Select the file corresponding to the patient. Pictures will be added automatically.

6

Import contextual picture to your file on the software.

BOREA

ESTER Technopole
6 allée Duke Ellington,
87100 LIMOGES, FRANCE
+33 (0)9 83 71 71 61
contact@borea.dental

www.BOREA.dental