

Cardio Companion Directions

Select Lead II on monitor

Always set monitor on 3 lead regardless of 5 lead set

Cat/Exotics/Small Dogs

Use Green catheter 12 French, 36" long

3 lead connection

Red wire on 1

White wire on 2

Black wire on 3

5 lead connection

Red wire on 1

White wire on 2

Black wire on 3

Green wire leave disconnected

Brown wire leave disconnected

Dog

Use Blue catheter 18 French, 42" long

3 lead connection

Red wire on 1

White wire on 2

Black wire on 3 for small dogs

Black wire on 4 for large dogs

5 lead connection

Red wire on 1

White wire on 2

Black wire on 3 for small/medium dogs

Black wire on 4 for medium/large dogs

Green wire leave disconnected

Brown wire leave disconnected
