

Avante Vetrec NXT

Telemetry Monitoring System



QUICK REFERENCE GUIDE

Entering new patient:

1. From telemetry main screen, **click DATABASE** (located in the middle bottom portion of screen)
2. Within database screen **click NEW** (located in the middle of screen)
3. **Input the patient info** (ie ID #, name and etc.)
4. Then **click CREATE** (blinking red)

** Now your client is created in the database*

5. **Click RETURN** to take you back to the main screen
6. **Click PATIENT SETUP** within the channel you would like to place the patient on
7. **Click on the PATIENT YOU WANT**
8. Next click on **ADMIT ON CHANNEL 2** for instance if you want to admit them on that channel 2
9. **Click YES** to confirm (Pops up automatically)
10. **Click START ADMISSION** (Pops up automatically)

Telemetry Troubleshoot Hotline:



(855) 373-0733