



Points to take note when using DT-8806H Non-Contact Forehead Infrared Thermometer

- This forehead infrared thermometer must be used in an ambient temperature range between 10°C to 40°C, and relative humidity $\leq 85\%$.
- Keep the forehead infrared thermometer away from water, heat and direct sunlight.
- For first use or inserting new batteries, wait for 10 minutes for the forehead infrared thermometer to warm-up and to test the room temperature before turn on the device.
- The forehead infrared thermometer must be left in the same environment (indoor or outdoor) for 15 to 20 minutes before using.
- Before taking temperature, making sure there is no hair, perspiration, cosmetic or cap covered the forehead.
- Ensure the “mode button” is switch to Body mode for forehead measurement.
- Aim toward the forehead as shown on Fig 1 diagram, keep the measuring distance between 5cm to 15cm.

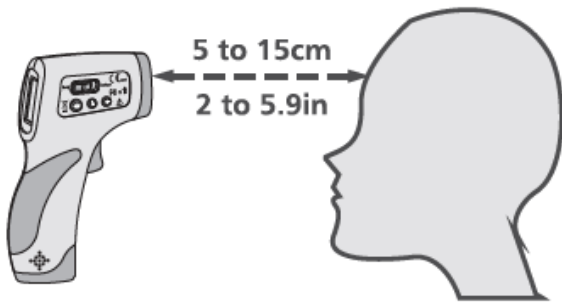


Fig 1. Forehead Method

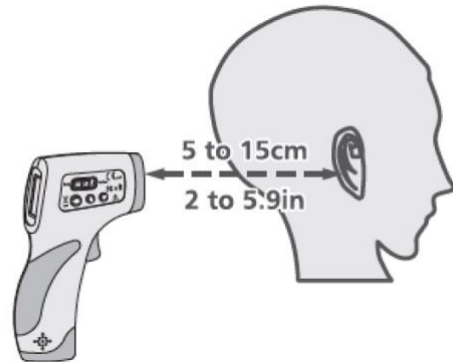


Fig 2. Behind Ear Lobe Method

- Make sure the place when you take forehead temperature is not directly airflow, and not direct sunlight. For outdoor environment, it is best under the shade.
- Another way of taking the body temperature is behind the ear lobe as shown in Fig 2 diagram. Making sure there is no hair, perspiration, cosmetic or cap covered.
- The default alarm threshold temperature is set at 38°C.
- Do not drop or knock the device, and do not use if damaged.
- Clean the glass lens with cotton bud with lightly moistened 70% alcohol.
- Remove the battery from the device if it is not required to use for a long time.
- Use of this thermometer is not intended as a substitute for the consultation with your physician.
- Should a problem occur with your device, please contact GMM Technoworld Pte Ltd. Do not attempt to repair the device yourself.