

Honeywell H910P N95 Mask Donning Procedure

The H910 N95 mask is a vertical flat fold mask, and therefore the donning procedure varies slightly from other manufacturer designs. Please follow the steps below to ensure the best fit:

Step #1: Open the Mask

When taking it out of the pack and unfolding the mask ready for donning, ensure that the metal nose-bridge is flattened (refer to image). This is to take the point out that has been caused by the mask being folded flat.



Step #2: Present the Mask

Cup the respirator in your hand ensuring the mask is open with the straps hanging down.



Step #3: Don the Mask

Place the mask on the face ensuring the chin is covered. Pull the bottom strap over your head and place on the back of your neck. Then pull the top strap over your head and place on the crown of your head above your ears.



Step #4: Place the Mask


Place both hands on the mask and move up, down, right and left, until you find that the mask is covering the chin and the nose-bridge is not too high on the nose. For reference, the top of the mask should be roughly at eye level.



Step #5: Mould the Nose Bridge:

Starting at the top of the nose and using both hands, start to mould the nose-bridge to the shape of the wearers face, working down the entire length of the nose-bridge.

This can be repeated several times until the user is happy that the nose-bridge is sealing. Care must be taken not to pinch the nose-bridge as it can create a point which may cause the mask to leak.

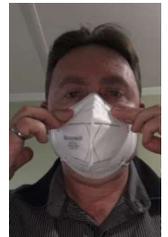

Step #6: Seal Check:

Place both hands on the side of the mask holding it in place. Care must be taken not to move it from its ideal position obtained by correctly donning. Breathe in sharply. Negative pressure should be felt inside the mask. This should be done every time the mask is donned.

If a seal is not reached, readjust the position on the face, readjust the straps and try again until a seal is reached.

