

1. AUTOCLAVE

Super Deluxe Series

Autoclave

The Series represents Sturdy's all purpose, top-of-line tabletop autoclave with versatility and flexibility to meet almost any situation that arises. This model uses an LCD panel along with 3 navigation buttons (up/down/return), which allow you to configure almost everything using the screen - this minimizes the buttons on the panel and allows the series more flexibility and functionality.



Autoclave

Super Deluxe Series

Features

- Class B
- LCD panel
- Pre & post vacuum
- Prion program
- Leakage, Helix, B.D. test program
- Customization program
- Data recorder (SD card) and printer
- Tank can be filled manually or piped-in (filtered water)
- Overheat protection
- Process evaluation system
- Pressure auto door lock
- Emergency stop button
- Pressure overload protection
- Double water tanks (Only for SA-300MB/302MB)
- Chamber volume: 24 L / 40L / 50L



Autoclave

Super Deluxe Series

A Pressure Gauge



B The yellow mark is full water level.



C Printer



D Easy Door Knob



Autoclave

Super Deluxe Series

SA-300MB/SA-302MB External View

A Pressure Gauge



B Door Latch



C Printer



Reliable Sterilization Partner

Super Deluxe Series

SA-260MB / 300MB / 302MB Control Panel

Up Button:
Used for setting and changing the function and increasing the parameters.

Unlock Button:
Press this button to release the lock status after completing a sterilization cycle or un-executing program.

Printer Button:
Print the prompt message of each sterilization steps automatically or re-print the last sterilization steps.

SD Slot:
To accept a formatted SD card for storing the process information of a sterilization cycle by text format (*.dat).



LCD Panel:
Display the information of sterilization status.

Enter Button:
Press this button to execute or confirm the input data.

Down Button:
Used for setting and changing the function and decreasing the parameters.

Cancel Button:
To abort the changing operation and return to previous step.

Emergency Stop Button:
Stop program and release pressure/vacuum.

Autoclave

Super Deluxe Series

Certifications

Sturdy is manufactured in compliance with the following directives and standards:

- 93/42/EEC, Class IIB (European Directive for Medical Devices)
- 2014/68/EU (Pressure Equipment Directive)
- EN 285 (Large Steam Sterilizers),
- EN 13060 (Small Steam Sterilizers)
- EN 61010-1/-2-040 (Safety Requirements for Electrical Equipment for Measurement Control And Laboratory Use – Part 1 and Part 2)
- EN 61326-1 (Electrical Equipment for Measurement Control And Laboratory Use – EMC Requirements – Part 1: General Requirements)
- ISO 13485 (Quality Management/Certification)

Super Deluxe Series

Specifications

SPEC/Model NO.			SA-302MB
Chamber Capacity (L)	—	—	50
Chamber Size Diam x Depth (mm)	—	—	300 X 710
Overall Dimensions W x H x D (mm)	—	—	620 X 489 X 906
Voltage (V)	—	—	230
Current(A) / Power(W)	—	—	14A / 3226W
Heater (W)	—	—	2300
Dry Heater (W)	—	—	826
Net Weight (Kg)	—	—	92
Water Fill*	—	—	Tank/Piped
Type	—	—	Tabletop
Water Tank Capacity (cc)	—	—	4200 / 8400
Water Consumption per cycle (cc)	—	—	700 / 1500

* "Tank" - Internal Water Tank with auto-chamber-fill, "Piped" - uses local water plumbing with auto-fill.

SPEC/Model NO.			SA-302MB
Temperature Options (°C)	—	—	121 or 134 105-135
Prion	—	—	S
Bowie Dick Test	—	—	S
Pump Test	—	—	S
Sterilization Timer	—	—	0-60 min.
Dry Timer	—	—	0-60 min.
Redry	—	—	S
Data Recorder	—	—	S
Printer	—	—	S
Liquid	—	—	0
Temperature Indicator	—	—	LCD
Pressure Indicator	—	—	Gauge / LCD Display
Progress Indicator (bar or lights)	—	—	S
Power Indicator	—	—	S
Pressure Auto Lock	—	—	S
Door Closed Indicator	—	—	S
Low Water Indicator	—	—	S
Reset	—	—	S
Program Selector	—	—	S
Error Indicator	—	—	S
Pre-Vacuum	—	—	S
Post Dry Vacuum	—	—	S

Note: "S" - Standard configuration. "0" - Optional configuration.