# HydroCone

By GC Aesthetics™

## Instructions for use

---

Model Name REF\_01KR

English (EN)

French (FR)

German (DE)


Italian (IT)

Portuguese (PT)

Spanish (ES)

Dutch (NL)

Polish (PL)

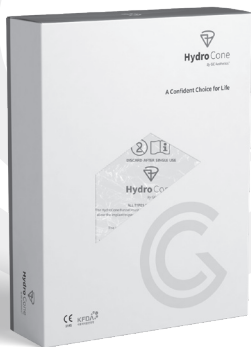2195

# Index

---

- Instructions for use** ..... Page 1  
English (EN)
- Consignes d'utilisation** ..... Page 9  
French (FR)
- Gebrauchsanweisung** ..... Page 17  
German (DE)
- Istruzioni per l'Uso** ..... Page 25  
Italian (IT)
- Instruções de utilização** ..... Page 33  
Portuguese (PT)
- Instrucciones de uso** ..... Page 41  
Spanish (ES)
- Gebruiksaanwijzing** ..... Page 49  
Dutch (NL)
- Instrukcja użytkowania** ..... Page 57  
Polish (PL)

# Instructions for use

---

English (EN)

## HydroCone description

---

HydroCone is a sterile, single-use instrument for the delivery of silicone gel implants. It has a cone-shaped sleeve constructed from a flexible, polymeric film and a hydrophilic coating on the inside.

**Note:** *This coating is not absorbed into the human body during the surgical operation procedure.*

## Single-use

---

This product is intended for single-use only. Do not attempt to use more than once.

## Product sterility

---

Each HydroCone is sterilised by ethylene oxide in a sealed, double-pouched configuration. Product sterility can only be maintained if the package seals are intact. Re-use or re-sterilisation should not be performed under any conditions as contamination or infection may occur. Attempts to re-use or re-sterilise may cause diminished product performance, including loss of lubricity, potentially causing implant damage, including rupture.

Please check the package is intact before using.

## Intended Use of HydroCone

---

HydroCone is a sterile single-use instrument for the delivery of silicone gel implants to assist with the delivery of silicone gel implants in breast implant surgeries.

## Contraindication

---

Pregnant women.

**Side effect:** Cross-infection. Single-use only to minimize the risk of cross-infection.

## How to open HydroCone?

---

HydroCone must not be exposed to lint, talc, sponges, towels, or other surface contaminants.

1. Remove HydroCone from its outer package in an **aseptic environment** using talc-free gloved hands.
2. Invert package over a sterile field, allowing the inner pouch to gently fall into the sterile field, or by presenting the inner peel pouch to the scrubbed person using a sterile technique.
3. Using sterile gloves, open the inner pouch and remove HydroCone placing it into the sterile field.
4. Unfold HydroCone and remove and discard the packaging insert from inside of HydroCone.
5. Do not place HydroCone outside of the sterile field at any point. If there are any concerns about the maintained sterility of the funnel please open a new one.

## Instructions for use

---

- Trim the small “distal” end of the funnel to an opening large enough so that when the inner surface of the funnel is hydrated, the implant can gently pass through the funnel and into the surgical pocket.
- Fully immerse HydroCone in a basin of sterile saline to ensure that the inner surface is slippery.
- Load the implant into the large “proximal” end of the funnel. With the tissue retracted, insert the distal end of the funnel into the tissue pocket. Squeeze the proximal end of the funnel behind the implant to gently guide the implant into the pocket. Once the implant is inserted, gently remove the funnel.

## 1 | HYDRATE

Fully immerse the funnel in a basin of sterile solution to ensure that the inner surface is slippery. When not in use, HydroCone should be laid flat on a sterile table so that the inner surface remains moist. The inner surface should be fully hydrated before using.

**Note:** *HydroCone should not be allowed to soak for longer than necessary in the basin of sterile solution. However, it should be kept moist to prevent the coating on the inner surface from drying as the lubricity of the hydrophilic coating is diminished when allowed to completely dry.*

---

## 2 | LOAD / Confirm Sizing

Insert the implant into the large proximal end of the hydrated funnel using a minimal or no-touch technique. Use one hand to support the distal end of the funnel, and the other hand to manipulate behind the implant. Use this hand movement to push the implant forward through the tapered funnel and eject the implant into a sterile field.

If the implant does not exit the small end of the funnel using gentle force, then remove the implant, re-trim, and repeat the test, gradually trimming larger opening sizes until the desired result is achieved.

Load the implant into the funnel ready for delivery into the tissue pocket. Using one hand, apply slight pressure behind the implant to remove any slack, verify positioning of the implant orientation within the funnel and advance the implant to approximately 1cm from the distal end.

### 3 | TRIM

It is recommended to use sterile straight edge blade scissors to trim the distal end. The HydroCone opening should be large enough to allow the implant to gently pass through without being too large, thus inadvertently enabling the implant to pass through.

**Note:** The length of the distal end of the funnel does not dictate the incision length required. Follow the implant manufacturers' guidelines for determining and making the optimum incision based on the specific implant being used.

#### Caution!

There are various manufacturers and types of implants, each with multiple variations of geometries, volumes, sizes, shapes, surface textures, cohesiveness, etc., as well as routine introductions of new models. The trim guidelines are only suggested **trim locations**. It is the responsibility of the user to assure the length of the distal funnel opening is adequate to insure the implant can pass through without any damage.

**Breast Implants:** The following trim guidelines are printed on the funnel, to accommodate various shapes and styles of implants based on their volume (cc's).

#### INFORMATION TO TRIM THE FUNNEL

SMOOTH ROUND	TEXTURED ROUND
Moderate & Moderate Plus: as shown	Moderate & Moderate Plus: +1.5cm larger
High Profile: +0.5cm larger	High Profile: +2cm larger
Ultra High Profile: +1cm larger	Ultra High Profile: +2cm larger
<b>Anatomical / Shaped implant +2cm larger</b>	

## 4 | PROPEL

Retract the incision site fully and continue to apply slight pressure behind the implant. Insert the distal end of the funnel approximately 1 cm through the incision. Use one hand to support the funnel and use the opposite hand to employ a pushing/squeezing technique behind the implant to gently propel the implant forward into the centre of the surgical pocket.

### Caution!

**Using excessive force may damage the implant.**

*If the implant does not gently advance through HydroCone into the surgical pocket, similar to the amount of force required during the sizing confirmation step, STOP! Ensure the distal end is not folded, pinched, wrinkled or overly constricted. Verify that both the incision and **surgical pocket** are large enough to accommodate the implant. Verify the end of the funnel is not inserted more than 1 cm, is directed toward the centre of the surgical pocket, and the pathway is not obstructed by muscle tissue or surgical instruments. DO NOT expose HydroCone to lint, talc, sponges, towels, or other surface contaminants.*

**Storage temperature limit:** Store at 1°C to 25°C.

This is a disposable product for single-use only.

**Shelf life:** 2 Years.

**Disposal:** The funnel should be disposed of according to local laws and regulations.

## Trouble shooting guidelines for HydroCone

---

**TROUBLE: HydroCone is sized properly, but the implant does not move through the funnel easily.**

**Question: Is the inner surface of HydroCone slippery or lubricious?**

**Considerations:**

1. Ensure the inner surface is hydrated as instructed in step #1.
2. Deliver the implants into the surgical pocket as soon as possible after hydration.

**TROUBLE: The funnel tore during the sizing step or during implant delivery.**

**Question: Was the end trimmed appropriately for the implant volume?**

1. Open and trim a new funnel according to instructions in step #3.
2. Sizing the funnel for textured shaped or high-profile implants may require a larger trim size.
3. Confirm the funnel is trimmed correctly and that the implant moves easily through the funnel before proceeding.

**TROUBLE: It is easier to deliver the implant on one side of the patient than the other.**

**Question: Is the surgeon right- or left-handed?**

**Considerations:**

1. For a Transaxillary approach such as for breast or pectoral implant procedures.
  - a. Right-handed surgeons can stand above the armboard when delivering the right implant and below the armboard when delivering the left implant.
  - b. Left-handed surgeons can stand below the armboard when delivering the right implant and above the armboard when delivering the left implant.
2. For most bilateral procedures.
  - a. Right-handed surgeons can stand on the patient's right side when delivering both left and right implants.
  - b. Left-handed surgeons can stand on the patient's left side when delivering both left and right implants.

## Symbols

	Manufacture date		Do not re-sterilise
	Lot number		CE Marking
	Consult instructions for use		Keep away from sunlight
	Reference number		Temperature Limit
	Ethylene Oxide Sterile		Warnings
	Manufacturer		Use By
	EU representative		Do not use if package is damaged
	Do not re-use		

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**Certification Experts B.V.**