

STADA

Product Title:
Savlon Advanced Healing Gel UK/IE

Pack Size: 50g

Mat.No: 92893751906_PIL

Dimensions: 160 x 210mm

Keyline Reference:

Barcode:

Pharmacode:

Datamatrix:

Reason for Change:
Change of ownership from GSK to T&R

1. Colour: ■ Black

2. Colour: ■ Keyline (Non-Printing)

3. Colour: ■ Stada Info Box Red

4. Colour: ■ Stada Info Box Blue

5. Colour: ■ Cirrus_Info_Box

6. Colour:

7. Colour:

8. Colour:

9. Colour:

10.Colour:

1. Fonts Used: Helvetica Neue LT Pro

Point Size Used: Min: 8ptMax: 8pt

2. Fonts Used:

Point Size Used: Min:Max:

Proof Number: 4Date: 23.09.2019

Operator: ECA Artwork Number: 1072043

CRN: 16193Suffix: C

Local Authority submission required: N/A

Approved by:Date:

Issued by:Date:

Wound treatment
An all round approach to wounds

Indications
Savlon Advanced Healing Gel is a hydro-active colloid gel for the treatment of minor wounds which supports the fast healing of skin by covering the wound and creating an optimal moist healing environment. Savlon Advanced Healing Gel is for use on minor wounds and superficial burns (e.g. grazes, cuts).

Properties
Savlon Advanced Healing Gel’s hydro-active colloid formulation can quickly help soothe the pain of minor wounds. The gel helps create optimal healing conditions, reducing the likelihood of scarring.

Savlon Advanced Healing Gel can be used for dry or wet wound:

- In case of a dry wound Savlon Advanced Healing Gel turns the dry wound into a moist one (“hydrogel” effect). The dry wound absorbs the water contained in Savlon Advanced Healing Gel helping reduce the likelihood of scarring.
- In case of a wet wound, the hydrocolloid of Savlon Advanced Healing Gel is activated and consequently Savlon Advanced Healing Gel absorbs excessive fluid from the wound and restores the water balance in the wound (“hydrocolloid” effect).

How to use
Do not use on extensive weeping or infected wound unless recommended by your doctor. Wash hands before and after treating the wound. Clean and dry the wound gently before applying the gel.

For minor wounds (e.g grazes, cuts) and superficial burns
Carefully apply Savlon Advanced Healing Gel to the wound (without rubbing) once or twice per day. If necessary, cover the wound with a plaster or dressing and change daily.

Removing the gel

- Wash hands before and after removing Savlon Advanced Healing Gel.
- Savlon Advanced Healing Gel can be removed simply and painlessly with wet gauze.

Precautions


- For external use.
- Side-effects such as irritations or allergic reactions are rare.
- Do not use Savlon Advanced Healing Gel on the eyelids or in the eyes. If Savlon Advanced Healing Gel comes into contact with the eyes rinse the eyes with running water and consult a doctor.
- On extensive weeping or infected wounds, consult a doctor before using Savlon Advanced Healing Gel because other complementary treatments might be necessary.
- If the wound becomes swollen, reddened or painful, or doesn’t start to heal within a few days seek medical advice.
- If you are pregnant or breast-feeding, ask a doctor before using Savlon Advanced Healing Gel.
- Savlon Advanced Healing Gel is a sterile single use medical device. Re-use imposes a risk of infection.
- Please do not use in case of known sensitivity to Savlon Advanced Healing Gel or any of its ingredients.

Storage conditions

- Store in a cool dry place and in the original packaging.
- Do not use Savlon Advanced Healing Gel after the expiry date shown on the pack.

Ingredients
Purified water: matrix
Propylene glycol: humectant
Carbopol: viscosity enhancer
Cross linked CMC: hydrocolloid particle that swell upon absorption
Tri ethanol amine: PH adjuster.

How supplied
50 g.

**Crest Medical Ltd, Chesford Grange,
Warrington, Cheshire WA1 4RQ. UK.**

Distributed in the UK by: Thornton & Ross Ltd., Huddersfield, HD7 5QH, UK.
Imported in Ireland by: Clonmel Healthcare Ltd., Waterford Road, Clonmel, Co. Tipperary E91 D768, Ireland.

CE

0459

2

STERILIZE

STERILE

R

EC REP

CMC Medical Devices & Drugs S.L. C/Horacio Lengo N°18,
CP29006, Malaga-Spain. info@cmcmedicaldevices.com

CP

CLONMEL
HEALTHCARE

STADA GROUP

T

+

R

Text prepared: 09/2019

GB 943737 – 92893751906



Product Title:
Savlon Advanced Healing Gel UK/IE

Pack Size: **50g**
Mat.No: **92893751906_PIL**

Dimensions: **160 x 210mm**






Keyline Reference:

Barcode:

Pharmacode:

Datamatrix:

Reason for Change:
Change of ownership from GSK to T&R

- 1. Colour:  **Black**
- 2. Colour:  **Keyline (Non-Printing)**
- 3. Colour:  **Stada Info Box Red**
- 4. Colour:  **Stada Info Box Blue**
- 5. Colour:  **Cirrus_Info_Box**
- 6. Colour:
- 7. Colour:
- 8. Colour:
- 9. Colour:
- 10.Colour:

1. Fonts Used: **Helvetica Neue LT Pro**
Point Size Used: Min: **8pt** Max: **8pt**
2. Fonts Used:
Point Size Used: Min: Max:
Proof Number: **4** Date: **23.09.2019**
Operator: **ECA** Artwork Number: **1072043**
CRN: **16193** Suffix: **C**
Local Authority submission required: **N/A**
Approved by: Date:

Issued by: Date:

