




E Z CLAMP II

UMBILICAL CORD CLAMP

REVOLUTIONIZES THE WAY UMBILICAL CLAMPS OPERATE

A unique Umbilical Cord Clamp that puts two clamps on the baby's cord and simultaneously cuts in between, in one simple and safe action.

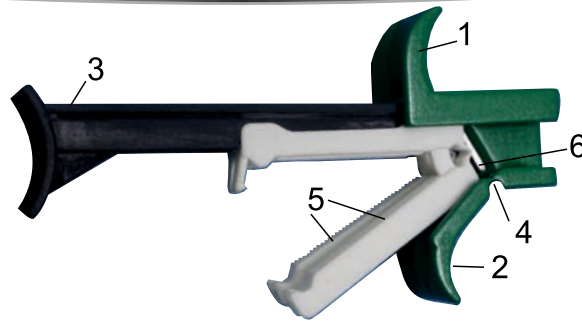

- 1 - Fixed handle
- 2 - Flexible handle
- 3 - Push handle
- 4 - Bending joint
- 5 - 2 Clamps
- 6 - Knife

ADVANTAGES

- Increases safety for infant and mother
- Separates to two regular clamps
- Convenient grip
- Four traditional stages in one easy action
- Reduces the danger of cross infection
- Enables a clear view of the inner veins
- Easy and fast to use with one hand
- Simple, fast & cost effective
- Can be used by either right and left hand
- No cutting point adjacent to clamp
- No need to scalpel or scissor near an infant
- No risk of blood
- Simple to operate for non-regular users
- Great for ambulances & clinics
- Disposable, for one time use

DESCRIPTION



The **E Z Clamp II** is composed of two regular Umbilical Cord Clamps and a Stainless Steel Knife combined into one instrument which looks, feels and operates as a regular Umbilical Cord Clamp. The **E Z Clamp II** will separate after use into three parts: a regular Umbilical Cord Clamp, put on the baby cord, a second clamp placed on the placenta. The third part will be the housing with the knife for disposal.

HOW IT WORKS

E Z Clamp II is ready to use in a sterile pack. The user simply holds the device in one hand and inserts the cord into the instrument as close as needed to the infant's stomach / body. User presses with his thumb on the pusher (3) (as with a syringe). As user start pushing clamp down, with holder (1&2) holder 2 will bend (4) and Umbilical Cord will be stabilized, and the umbilical clamps (5) will start closing and when it is closed it will cut the cord (knife 6), and automatically separate into two clamps. One will stay with the baby and the second will be attached to the placenta.

SIMPLE AND SAFE

Simpler, faster, and cost effective not to mention safer for the newborn and the health practitioner, one easy action saves four traditional stages of clamping twice, cutting and securing knife. Each E Z clamp is individually packed in a sterile peel pack tray; it is disposable and can be used only once.

CAT. No. 5655 E Z CLAMP II sterile in a pill pack.

CAT. No. 5650 E Z CLAMP II non-sterile bulk.

CAT. No. 5658 Umbilical Clamp only, non-sterile bulk.

CAT. No. 5659 Umbilical Clamp only, sterile.



EC, US, Chinese and others patents

All-in-one clamp

Once using the **E Z CLAMP II** the user will never return to the traditional procedure for clamping and cutting the umbilical cord. The traditional clamping and cutting procedure with five instruments, is replaced with one easy action to make births more safe and simple for both the parents, midwives, doctors and nurses.

E Z CLAMP II has been designed to minimize blood spatter and the chance of injuries. It is made of medical plastic and is a environment-friendly device. Does not contain latex.

E Z CLAMP II differs from other instruments in that it can be used by both right and left hand, from any direction and any side and it will split into 2 clamps on the infant clamp and on the placenta, after it cuts.

E Z CLAMP II Comes in a sterile tray and is a disposable obstetric instrument for clamping and cutting the umbilical cord at child birth.



Open peel pack and place the umbilical cord into the center white clamps (5) (normally 3-6 cm. from baby). To ensure proper use, it is recommended to clear blood from the umbilical cord by squeezing it gently away from the cutting point towards the placenta.

Press the black handle (3) with thumb and push down using the green holder, with the index and middle finger (as if using a syringe). As the green holder is pressed, the two white clamps (5) will start to close on the umbilical cord, pushing down on the two white clamps, which will close and then slide along the towards the blade (6).

As the user is pushing the black clamp (5) down with the holder (1&2), the umbilical white clamps will be locked, and the knife (6) will cut the cord. The clamps will be closed permanently and will be separated into two camps - one will remain on the baby, the second will remain on the placenta.



The World Health Organization (WHO) has emphasized in several recent publications regarding childbirth, various relevant recommendations on newborn healthcare procedures, including the treatment of the Umbilical Cord:

<http://www.who.int/rht/documents/MSM98-4/MSM-98-4.htm>

<http://www.who.int/rht/documents/MSM96-24/msm9624.htm>

http://www.who.int/rht/documents/MSM93-13/essential_newborn_care.htm


<http://www.who.int/rht/documents/MSM98-4/MSM-98-4.htm>

Umbilical Cord Clamp

Our **Double Grip Clamp** can be used and delivered separately in bulk or sterile. The unique design of clamp includes a novel and especially strong male-female grip. Lightweight, low cost, no need for special instrument to remove clamp. A regular scissors can be used for removal. Bulk or sterile in transparent color or any color requested.

CAT. No. 5658 - Bulk non sterile.

CAT. No. 5659 - Sterile each clamp in envelop.

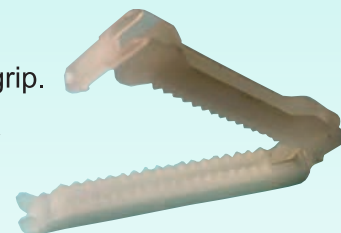

The **E Z Clamp II** is manufactured under **GMP** regulations. Certified according to **ISO 9001** and **ISO 13485**.

CE Marked

U.S. F.D.A. approved the device for marketing.

Clamps: **E Z Clamp II** clamps are manufactured from natural polyamide non-irritating and nontoxic. After use, in order to remove the clamp, there is no need for a special instrument and regular scissors can be used.

Blades: **E Z Clamp II** blades are manufactured from **stainless steel**.

PACKAGING: Each **E Z Clamp II** unit is packed in a sterile, transparent rigid peel-pack. Size of single unit pack: 18cm x 10cm.

There are 10 sterile **E Z Clamp II** units per box size: 23.5cm X 11cm. X 18.5cm

Master shipping case 10 X 10 (100each) size: 39cm. X 25cm. X 56cm.

Bulk packed – 200 per master.

Please see video how **E Z Clamp II** works in your language



in Spanish



in Italian



in German



in France



in English



KENCAP LTD.
KENYON INDUSTRIES, INC

kencap@kencapltd.com
www.kencapltd.com
www.kenyonindustries.com

EARDOC™

Ear Infections treatment

Relieves Ear Pain for Children and Adults

Solves the Problem and not the Symptom



The EARDOC™ is a non-invasive device designed to open the ear tube (Eustachian Tube) and release fluids and air trapped behind the eardrum, causing pain and infections in the middle ear (Otitis Media). In most cases, the pain is caused by the accumulated air & fluids pressuring on the ear-drum. About 75% of children aged 2 and up are affected by Otitis Media and annually 30-35 million visits to the doctor are related to ear pain and/or ear infections.

The EARDOC™ solves the problem causing the ear infections, instead of treating the symptoms. The fluids and air are drained by the EARDOC'S vibrations and the pain is relieved quickly and effectively. The EARDOC™ opens the ear tube without medication or ear tube surgery. The EARDOC™ relieves ear pain for children, adults, air travelers and divers that suffer from Otitis Media, trapped fluids in the middle ear and Eustachian tube dysfunction.

How does the EARDOC™ work?

EARDOC™ generates and transmits vibration waves to the middle ear and the Eustachian tube (view diagram on other side). The waves travel through the mastoid bone to the middle ear, vibrating & opening the Eustachian tube and draining fluids & air, in a non-invasive, safe and simple treatment. You can view a video about the EARDOC™ on our website – www.eardoc.info.

The EARDOC™ is a safe, non-invasive and effective treatment for:

- Otitis Media
- Trapped fluids & air in the middle ear
- Eustachian Tube Dysfunction
- Air travelers – releasing pressure during takeoff and landing
- Swimmers and Divers.

The EARDOC'S™ Benefits:

- Non-invasive
- Treats the problem not the symptom
- Easy to use
- Immediate relief
- Safe
- Children friendly
- Efficient
- Low cost, great value



<http://www.eardoc.info>

EARDOC™


EARDOC™ Test

The EARDOC™ was tested on patients complaining of ear pain and identified with fluids in the middle ear by tympanometry. Ear pain is often caused by pressure on the ear drum from the fluids & air trapped in the middle ear.

These patients were divided into 2 groups. (A & B)

Group A was not using the EARDOC™ and was getting regular treatment (Antibiotic and/or pain relief medication, or sit & wait for body to recover). Group B was using EARDOC™ for 3-8 minutes when feeling pressure or pain.

The patients in group B, treated with EARDOC™, were found to have a significant reduction of fluids in the middle ear, and patient reported of constant relief of pain and sense comfort after using the EARDOC™.

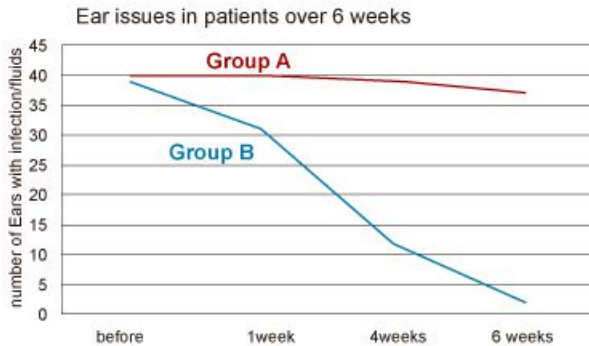


Patients using the EARDOC™ show significant pain relief and reduction of the middle ear fluids as well as ear drum functions.

— Group A: "Sit & Wait", Antibiotics and Painkillers
— Group B: Treated with EARDOC™


EARDOC™ Survey:

A satisfaction survey conducted with EARDOC™ customers has shown that a total of 94% of Eardoc users found pain relief when using the EARDOC™. 89% will use EARDOC™ again for ear infection & earache treatment.

After using the EARDOC™, which treatment would you recommend others?

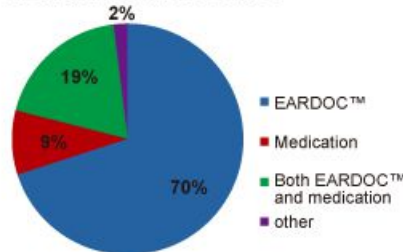


Did the EARDOC™ help?

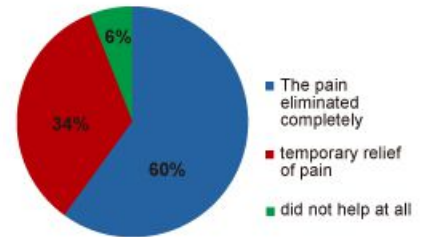

Ask your Doctor about the EARDOC™

Check out the EARDOC™ website: www.eardoc.info and see some of the testimonials customers have sent us out of the thousands of users that the EARDOC™ helped.

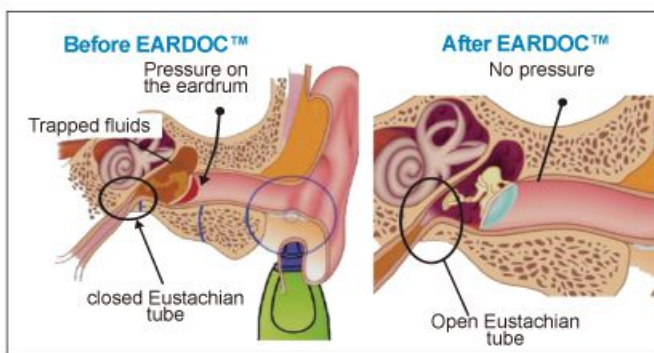

Each EARDOC™ is packed in a case. 10 cases in a box.
8 boxes in a master case (80 units per case)
MODEL 4055
PATENTED in Europe, USA, China and other countries

- 1) Soft head for a better feel and effects.
- 2) Led light indicating EARDOC™ is on.
- 3) ON button with 10 speeds available.
- 4) OFF button (3 second press to turn off).
- 5) Battery tray and cover (2 AA batteries).
- 6) The EARDOC's head curved shape is designed to fit behind the ear.
- 7) Dome design to transmit the vibration through the mastoid bone.



Learn more about the Eardoc and ear pain on:

www.eardoc.info



[Eardoc.info](http://www.Eardoc.info)



[@Eardoc](https://twitter.com/Eardoc)

or contact us via info@eardoc.info