

ref. 53907

IIR MEDICAL FACE MASK



Function

Disposable medical face mask, filtering more than 98% of the particles and allowing comfortable breathing during the work shift. Not sterile. Made of polypropylene, non-woven, 3 layer-filter material, and metal strip.

Technical specifications

Material	100% polypropylene fiber
Measures	17,5x9,5cm
Type	IIR Class I
Bacterial filtration efficiency	99,92 ± 0,07
Differential pressure	41,1 ± 0,5
Splash resistance pressure	21,3 kPa

Requirements:

- EN 14683: 2019 + AC: 2019.

Instructions

- ✓ Wash your hands beforehand with soap and water or a hydroalcoholic solution.
- ✓ Place the mask and adjust the clip on the nose, covering the ring.
- ✓ Do not wear the mask for more than 4 hours or touch it with your hand during use.
- ✓ Replace it if it is damp or damage.
- ✓ Remove it without touching the front and dispose of it in the appropriate plastic bag container
- ✓ Should not be placed on the forehead or chin during or after use.



Presentation

Bag of 50 units per carton

