BioPod

**BOUGER, RENFORCER,
PROGRESSER :**

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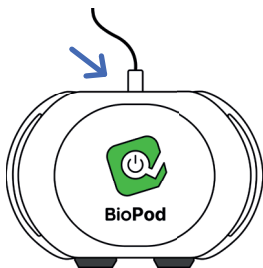
• Choose the programme you'd like in the left column, then click on the arrow  to start the programme. The programme will then start automatically on the BioPod device.

5 Chargement du BioPod Charging the BioPod

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• Use the supplied USB-C cable to charge the BioPod by connecting it to the back of the device.

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Bouton BioPod clignote orange
BioPod button flashes orange

En chargement
Charging

Bouton BioPod clignote vert
BioPod button flashes green

Appareil chargé
Fully charged device

Bouton BioPod vert fixe
BioPod button stays green

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Please note that when you plug the device in to charge, Bluetooth automatically disconnects.

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• A long push on the BioPod button allows you to stop the device. The button is then not lit up anymore.

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For further instructions on your BioPod device, the user's manual is available :

- On the website
<https://electroniquedumazet.com/index.php/manuels/>
- On the USB stick coming with the BioPod
- By scanning the QR code below

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For further instructions on the software, the user's BioStim manual is available by scanning the QR code.

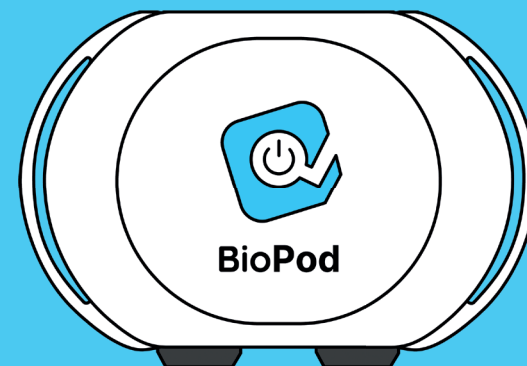
www.mazetsante.fr

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MANUEL D'INSTALLATION RAPIDE QUICKSTART MANUAL

BioPod

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BIOPOD START-UP AND INSTALLATION ON THE PATIENT

1 Branchement et fixation du BioPod sur le patient BioPod's plugging in and installation on the patient

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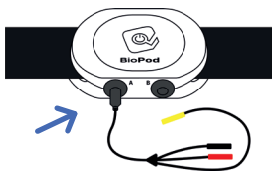
- Installation of the elastic strap on the BioPod : Pass the ends of the elastic strap through the two loops in the BioPod's housing (at the back of the device).

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
- Electrode's cable connection: clip the cable onto the A or B paths of the BioPod's housing.

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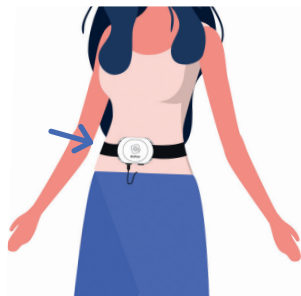- Installing the strap on the patient: the strap is placed on the patient's abdomen.

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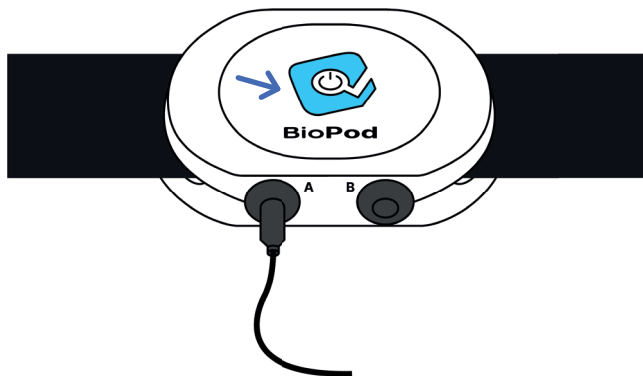
- Start the device up by holding the button located at the front of the BioPod.


On start-up, the button will flash blue, indicating connection search. The BioPod pairs automatically with the BioStim software through Bluetooth. One should launch the Biostim software and start-up the BioPod simultaneously to ensure optimal connection (if pairing does not work, please see the user's manual).

3 Vérification de la connexion du BioPod au logiciel BioStim Verifying the connection between the BioPod and the BioStim software

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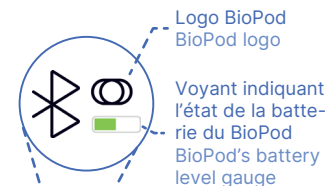- After the Biostim software has started up, click on the  logo; the connection happens automatically between the BioPod and the software. The Biopod's button will stay blue, and the BioPod icon will show up next to the Bluetooth one on the Biostim software. A battery level gauge is visible as well.

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- On the Biostim software welcome screen, by clicking "biofeedback", a program list opens. These are predefined programs with pre-saved settings.