BLACKROLL®

RECOVERY BASE

Instruction Manual

EN | DE | FR | NL | IT

Medical Device
Medizinprodukt
Dispositif Médical
Medische Product
Dispositivo Medico

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Intended use

The BLACKROLL® RECOVERY BASE is a foam mattress comprising multiple layers that can be tailored to accommodate individual weight and sleeping patterns. The CELLIANT® fibers on the surface of the cover facilitate the conversion of body heat into infrared energy, thereby improving local blood circulation and cellular oxygenation which will enhance performance, accelerate recuperation, and improve sleep. The mattress can be utilized with all bed frames. Install mattress to fit the bed frame and maintain the mattress by cleaning the mattress as required by the manufacturer.

Intended user and patients

The BLACKROLL RECOVERY BASE is a medical device because of the CELLIANT® fibers on the top of the mattress that enable conversion of body heat to infrared energy reflected back to the body to improve local blood circulation and cellular oxygenation which will enhance performance, accelerate recuperation, and improve sleep. No additional actions by the user are required.

The mattress is intended to be used by all kind of users, including lay persons.

Indications for use

An innovative textile that absorbs energy (heat) from the body and emits it as Infrared (IR) energy to muscles and tissues to help increase local blood circulation providing temporary relief of muscle soreness and joint pain; increases tissue oxygenation which enhances performance and strength; helps recovery and increase comfort and promotes restful sleep.

Indicated for all bodies that need restful sleep to improve their wellbeing.

Contraindications

There are no significant contraindications to the use of this product.

Precautions

Foam mattresses are generally durable, but they can be damaged if mishandled; sharp implements may puncture or tear the fabric cover.

Instructions for use

First, find the right configuration based on your body weight and sleeping habits. Adjust the mattress as needed. Then use it every night in the desired sleeping position.

If a serious incident occurs with this product, the BLACKROLL® RECOVERY BASE, please contact BLACKROLL AG, immediately on the contact details below:

shop.de@blackroll.com
+4971416962559

The instructions provided with the product have been validated by BLACKROLL AG to ensure correct installation and use of the product. It is the user's responsibility to ensure that the product is installed according to the recommendations in order to achieve the desired results.

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Hologenix LLC
Advena Limited,