



ASSEMBLY INSTRUCTIONS

HL Series
Pneumatic stool with back
AAP-CS610

Read these instructions before starting; follow assembly procedures carefully.

1. Install casters ⑥ into each of the legs of the base ⑤. Be sure each caster is completely inserted into leg for proper stability.
2. Insert the lift assembly ④ into base with tapered-end down. Secure lift assembly by applying a little downward pressure. Remove yellow plastic cap from top of lift assembly. **DO NOT PUSH ON BUTTON AT THE TOP!**
3. Attach back bars to back ③ using pan head screws ⑦, but do not tighten screws completely.
4. Insert back bars into left and right back bar support brackets on seat assembly and slide bars into brackets several inches. Secure back bars with knobs ⑧. Securely tighten all four screws to back.
5. Install seat with control assembly ② by aligning hole on the control with the the shaft of lift assembly. Push down firmly on seat. Handle on control can be pulled up to raise and lower set.
6. When sitting on seat, pull up on handle to lower seat. To raise seat, remove weight from seat and pull on seat handle.

GENERAL SAFETY INSTRUCTIONS

1. Do not stand on seat.
2. Stool must be on level surface to ensure proper stability.
3. Only use stool when fully assembled.
4. Periodically check fasteners and tighten securely if needed.
5. Never use stool if it does not perform according to this manual.

MAINTENANCE

Pneumatic lift in lift assembly is permanently lubricated at factory and needs no further lubrication. Never lubricate pneumatic lift.

Do not remove clip from bottom of lift assembly. Contact 1-800-733-4000 immediately if clip is missing or damaged.

PARTS LIST

① 1PC	② 1PC	③ 2PCS	④ 1PC	⑤ 1PC	⑥ 5PCS	⑦ 4PCS	⑧ 2PCS
--------------	--------------	---------------	--------------	--------------	---------------	---------------	---------------

