
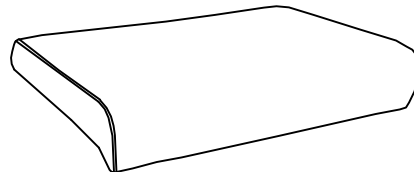
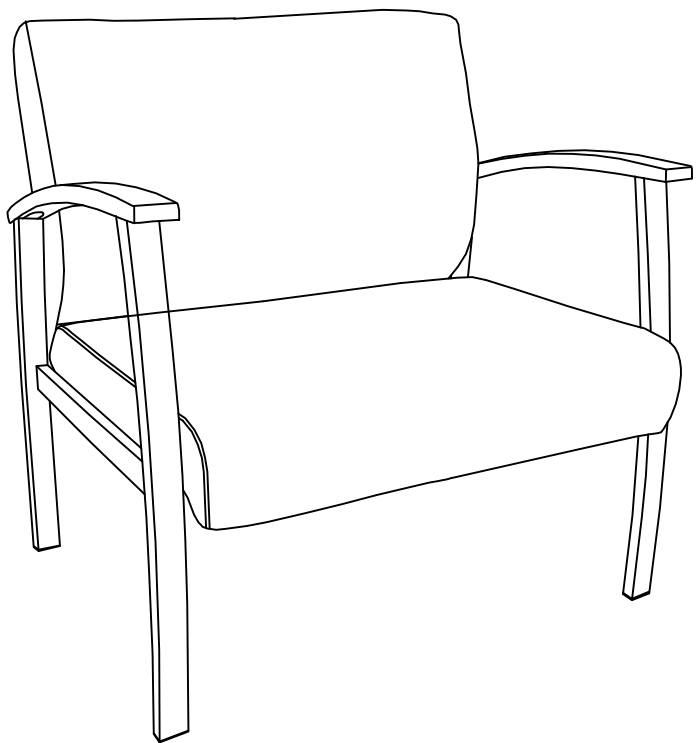
**Medical Chair**  
**Chaise ergonomique**  
**Silla de Uso Médico**


**LLR67001/LLR68557/LLR68558**

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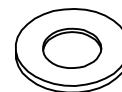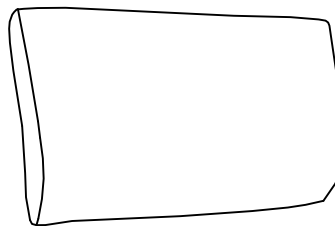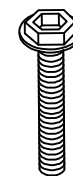
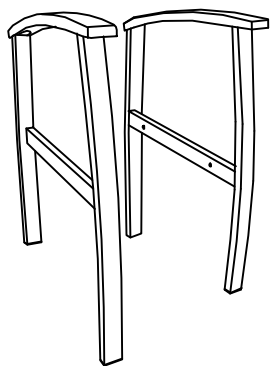
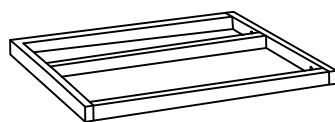
LLR67001/LLR68557/LLR68558

### ASSEMBLY INSTRUCTION

Remove all items from the carton. Verify all pieces before assembly.

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**STEP 2**  
**ÉTAPE 2**  
**PASO 2**

**STEP 3**  
**ÉTAPE 3**  
**PASO 3**

**DO NOT** tighten the screws of seat frame until the back cushion assembled.

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**STEP 4**  
**ÉTAPE 4**  
**PASO 4**

Set chair level on floor & tighten screws.  
If chair wobbles, loosen screws, re-level chair, then re-tighten screws.

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**STEP 5**  
**ÉTAPE 5**  
**PASO 5**

#### PREVENTIVE MAINTENANCE AND WARNING!

- Use this product only for seating one person at a time.
- Do not use this chair as a step stool/ladder.
- Do not use the chair on uneven floor surfaces.
- Do not use chair unless all bolts, screws and knobs are tightened. At least every six months check all bolts, screws and knobs to be sure they are tight.
- If any parts are missing, broken, damaged or worn, stop use of the product until repairs are made using factory authorized parts.
- Dispose of packaging properly. Plastic bag is not a toy. Do not use plastic bag as head covering - it may cause suffocation. Failure to follow these warnings could result in serious injury.
- Note: some screws may be preassembled. If preassembled, ignore assembly instructions.

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## Instructions

### **CALL US FIRST! DO NOT RETURN TO STORE.**

For immediate help with assembly or product information call our toll-free number: **1-888-598-7316** Mon. - Fri. 7am to 7 pm CST  
Our staff is ready to provide assistance. Damaged or missing parts ship from our facility in 1-2 business days.

Please provide the following information:

1. Full Name
2. Physical Address – do not include P.O. Box as our carriers cannot deliver to P.O. Boxes
3. Phone Number
4. Model Number
5. Part Number
6. Invoice #
7. Date of Purchase

Note: Our policy is to send all parts requests via normal ground transportation. If your requirements are to have these parts via overnight/next day, please provide your designated carrier and account number. Replacement panels are not available. If any damage has occurred during shipment please return the entire unit to place of purchase for credit and replacement. Original invoice may be required.

Before you begin:

Open, identify and count all parts prior to assembly. Lay out the parts on a non-abrasive surface such as carpet or blanket.

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