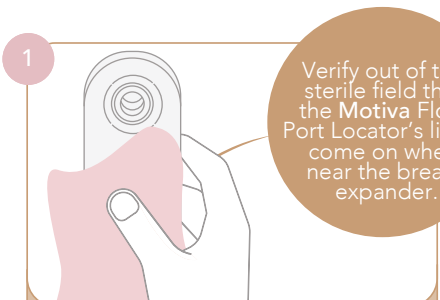
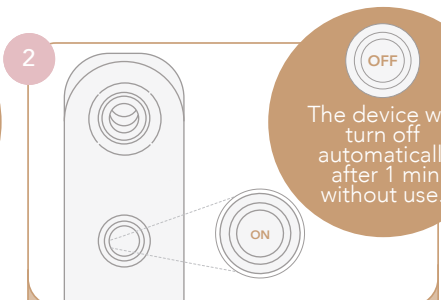


Motiva Flora[®] Port Locator


Quick Reference Guide


- 


Verify out of the sterile field that the Motiva Flora Port Locator's lights come on when near the breast expander.


Clean and disinfect using a lint-free cotton cloth and isopropyl alcohol. NEVER use abrasive products or solvents.
- 


The device will turn off automatically after 1 min without use.

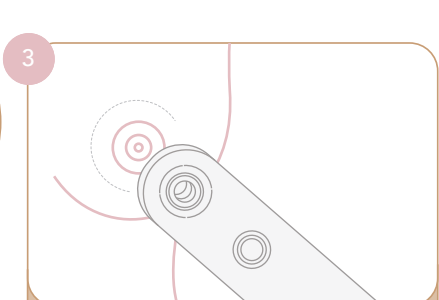Turn on the Port Locator by pressing the button and release once the white light ring is turned ON.
- 

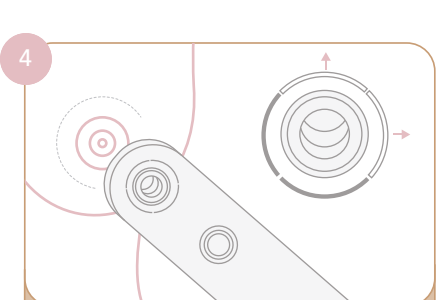Place the top part of the device near the breast and move it around looking for the injection port.
- 

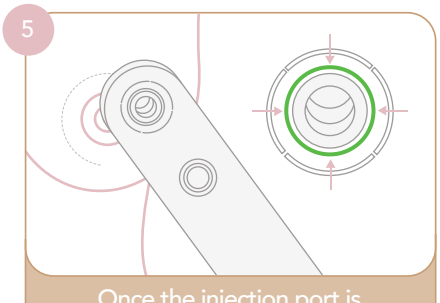

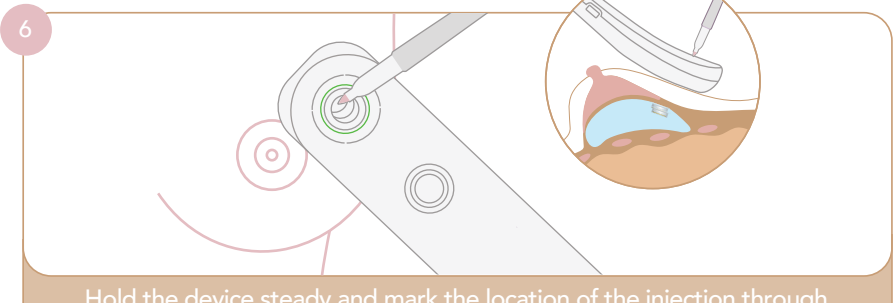

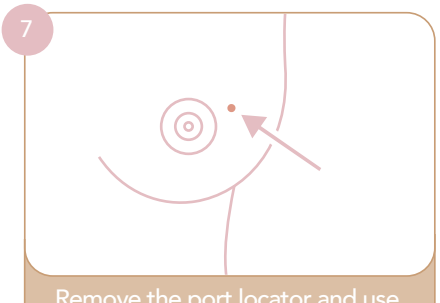









White lights will turn on guiding the location of the port.
- 

Once the injection port is located, a green light ring will turn on.
- 

Hold the device steady and mark the location of the injection through the opening of the device with a surgical marker perpendicular to the breast skin and/or pressing down firmly on the device to create a temporary indentation on the skin where the port is located.
- 

Remove the port locator and use the marked point or the center of the indentation on the skin as guidance for the injection site.

USER INTERFACE COLOR CODE

DEVICE ON & PORT LOCATION			BATTERY STATUS			RISK
ON	PORT LOCATION	PORT LOCATED	LOW BATTERY	CHARGING	CHARGED	ALERT
FIXED WHITE	FIXED WHITE	FIXED GREEN	BLINKING ORANGE	BLINKING BLUE	FIXED BLUE	FIXED RED
						
						

Reference: This is not a controlled document. This document is based on DOC-038: Directions for Use for Motiva Flora Port Locator.