


# HOW TO INSTALL LURACO WELLNESS CHAIR

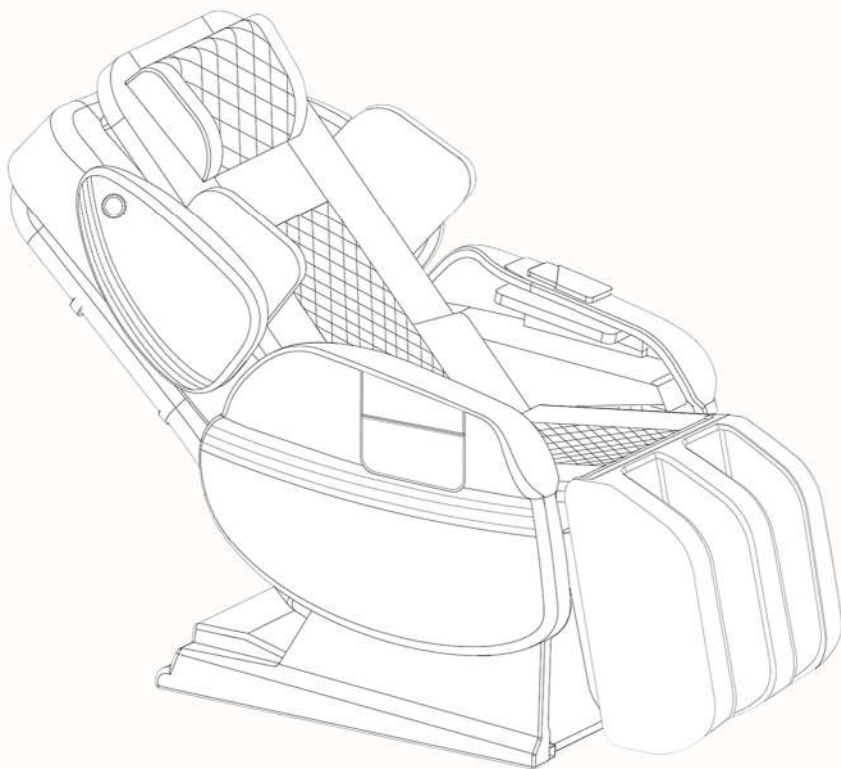

**FDA**  
REGISTERED

c **UL** us



The Power of American Engineering®



**Congratulations on your purchase of a new Luraco Medical  
Massage Chair!**

For more detailed instructions, please watch How to  
Install Luraco Wellness Chair video at  
[www.LuracoChairs.com](http://www.LuracoChairs.com). This video is also available  
on Luraco YouTube Channel.


For convenience, the complete User Manual is installed in  
the Wireless Controller and is available for download from  
website [www.LuracoChairs.com](http://www.LuracoChairs.com)

After reviewing guide, if you have any questions or  
comments, please call 1-800-483-9930 or contact our  
technical support department at [Support@Luraco.com](mailto:Support@Luraco.com)  
or visit our website at [www.LuracoChairs.com](http://www.LuracoChairs.com) for more  
information.

**ATTENTION: Do not turn on power until all  
installation is complete.**

## I. CHAIR PLACEMENT

There must be enough space around the chair for proper operation. There should be a clearance of 30 inches from the front of the chair to the front wall, 12 inches from the back wall to the backrest, and 18 inches from the wall to the closest armrest to allow for easy access.

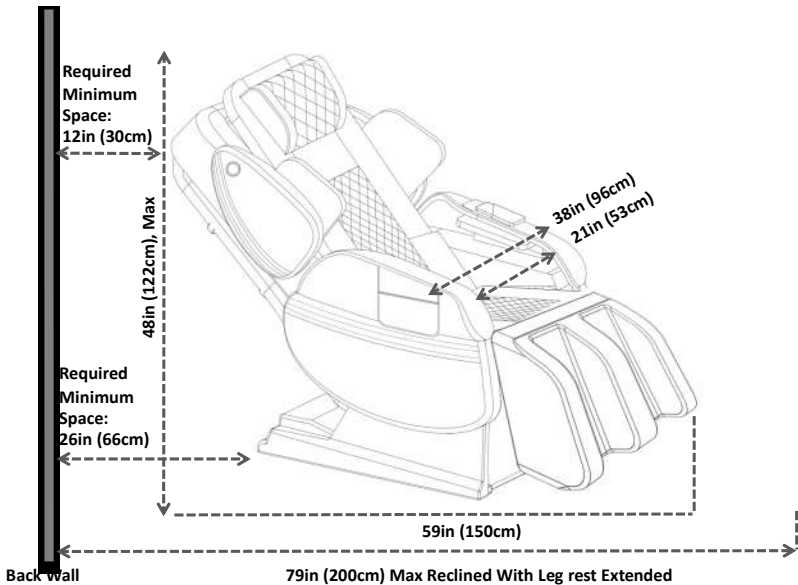

**Fig.2: Space Requirement**

## II. ASSEMBLE THE CHAIR

### **Step 1: Unpack the main box**

Carefully take out the chair main body, footrest, armrest packages from the main box.

**Note: Wellness chair requires at least two people to assist with assembly.**




### **ATTENTION:**

Be sure no children, pets, or objects are around chair. Allow enough space for installation.


### **Step 2: Assemble the Footrest**

Please refer to Fig.3 & 4 and follow the steps (1-2) below to install the Footrest.

1. Assemble the calf roller electrical connector (1pc) and air massage connector (1pc). **Make sure the connectors are fully locked (Fig.3, Step 3a & 3b).**
2. Install the Footrest hinges. Slowly align the footrest with the chair main body and install the lock-pins (2pcs) for the footrest hinges (**Fig.4**).

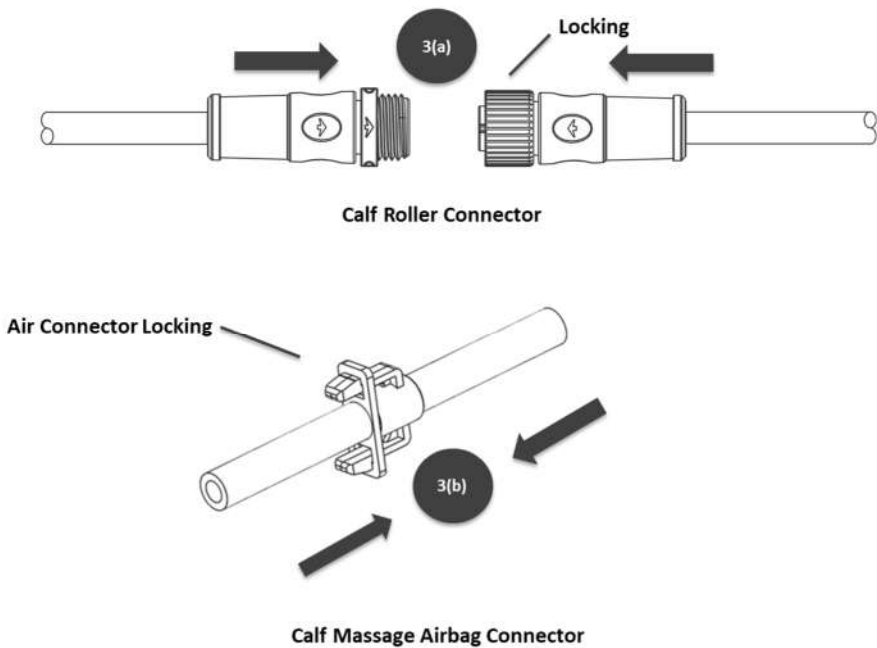

**Fig.3: Instruction for Footrest Calf Airbag and Roller Connector Assembly**

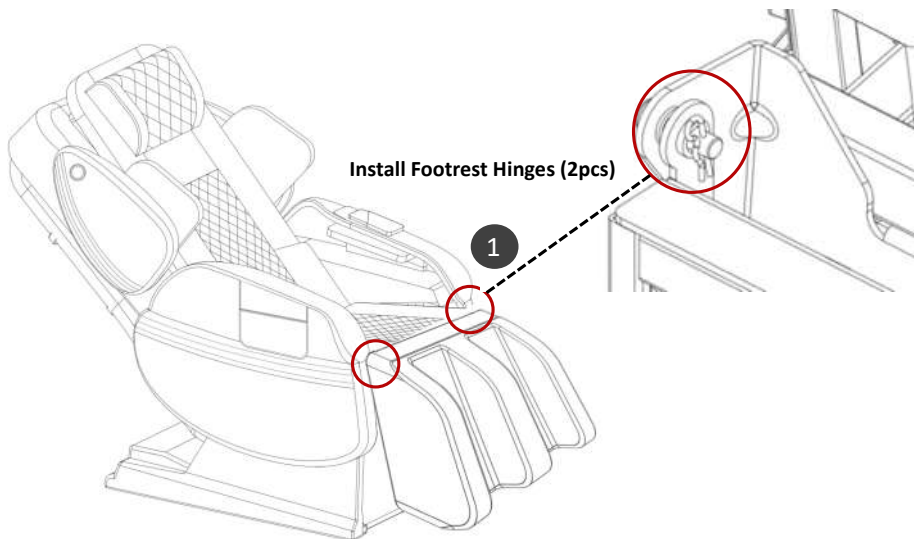


**Fig.4: Footrest Assembly**

### **Step 3: Assemble the Armrests**

Please refer to Fig.5-6 and follow the steps (1-5) below to install the armrests.

1. Insert the metal tab on the armrest into the located slot on the chair frame (**Fig.5, Step 1**).
2. Install 2 knob-screws to secure the armrest metal plate onto the seat frame (**Fig.5, Steps 2**).
3. Assemble armrest air connectors (2 pcs). **Make sure the color marks/numbers are matched (Fig.6, Step 3).**
4. Assemble USB connectors for wireless controller power and charging. **Make sure Male and Female fully inserted and secured (Fig.6, Step 4).**
5. Repeat steps (1 thru 4) above for the second armrest assembly.


**NOTE:** One USB connector needs to be connected for the wireless controller charging on the Right Armrest (Fig.6, Step 4).

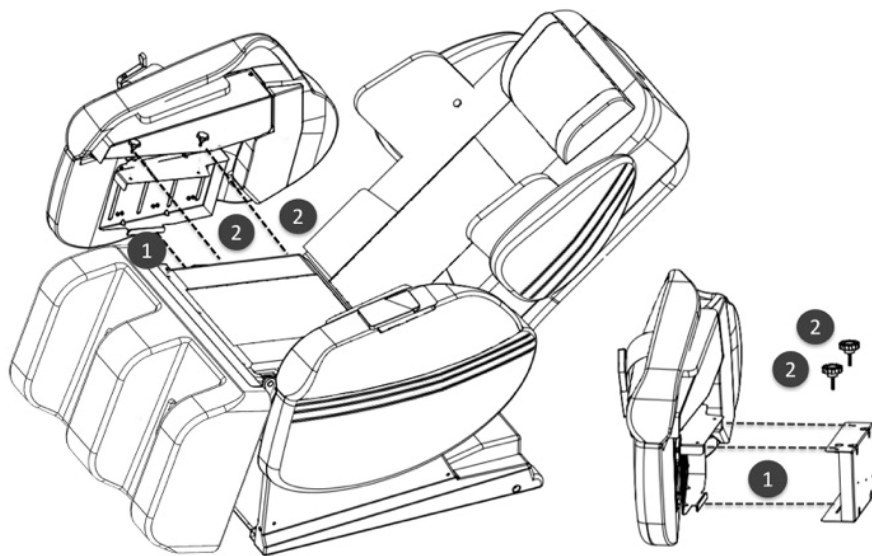

**Fig.5: Armrest Installation**



**ATTENTION: NEVER SIT ON ARMRESTS OR LEGRESTS!**

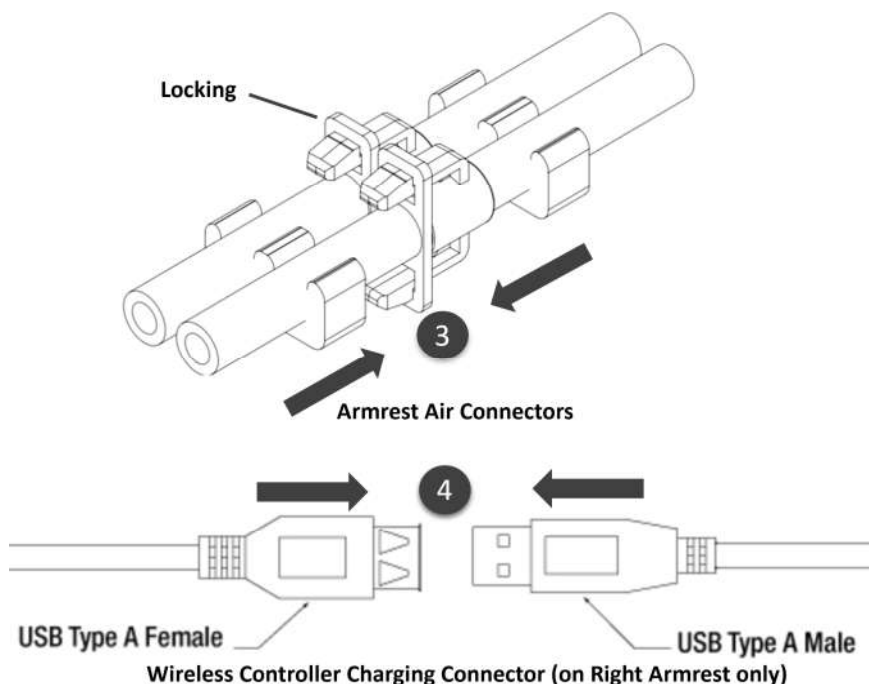

**Fig.6: Armrest Air and Electrical Connections**

**ATTENTION:** For more detailed instructions, please watch How to install Wellness Chair video at [www.LuracoChairs.com](http://www.LuracoChairs.com). This video is also available at Luraco YouTube channel.

**Customer Assistance:**

For assistance with this product, please visit:

[www.LuracoChairs.com](http://www.LuracoChairs.com) Or call: 1-800-483-9930, Monday – Friday, 8:30 am to 5:00 pm (CST).



**LURACO HEALTH & BEAUTY, LLC.**

**1140 107TH STREET**

**ARLINGTON, TX 76011, USA.**

**TEL: 817-633-1080**

**FAX: 817-633-1085**

**[WWW.LURACOCHAIRS.COM](http://WWW.LURACOCHAIRS.COM)**

**© All Rights Reserved, Sep 2024.**