HEPRO

SØSTERHJELP

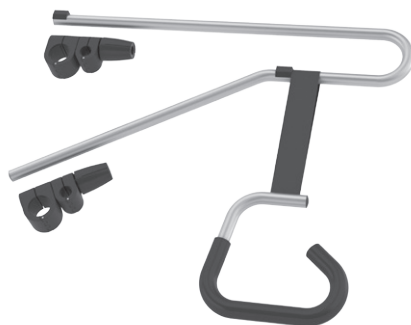
Manual

NORSK	2 - 3
SVENSKA	4 - 5
DANSK	6 - 7
SUOMI	8 - 9
ENGLISH	10 - 11
DEUTSCH	12 - 13
FRANÇAIS	14 - 15
ESPAÑOL	16 - 17
ITALIANO	18 - 19
NEDERLANDS	20 - 21

Montering av

SØSTERHJELP

Art. Nr. 35148

A

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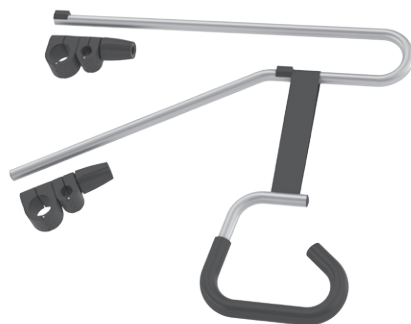
Art. No. 35148

A

Unwrap the received parts and make sure nothing is missing.

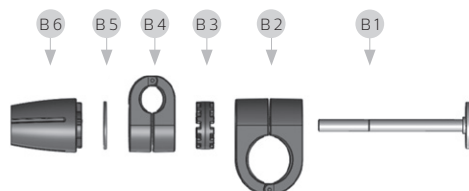
It should consist of:

- A1 One pole (trapeze bar tube)
- A2 Two universal connectors

B

The universal connector is disassembled, into six parts.

C

Attach the universal connector A2 to the Gripo pole. This is done by attaching the largest clamp B2 to the pole and then threading part B1 through, and attaching the remaining parts B3 to B6.

Angle part B4 so that the accessories fit.

Find the desired height and tighten so that the universal connector holds, but loosely enough that the accessories can be threaded on as described in the next step.

D

Then attach the second universal connector **A2** to the Gripo pole.

This is done the same way as in step C.

Thread on the accessories and tighten all universal connectors well unto **B6** so that the accessories cannot twist.

E

If the trapeze bar is supposed to have a hanging grip. Start by threading the strap over the trapeze bar and adjust to desired height before installing.

Make sure the universal connectors are sufficiently tightened so that the trapeze bar does not twist or come loose during use (maximum user weight is 110 kg).

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