# Medical Professionals Recommended VITAL INJECTOR 2<sup>™</sup>

# Surprisingly Vital

Introducing the All-Renewed Rejuvenating Injector

# Contents

- 1) What is Vital2 Injector?
- 2) Device Feature
- 3) Usage
- 4) Technology
- 5) Feature & Advantage
- 6) Why Vital Injector2?
- 7) Summary
- 8) Appendix Treatment Tip


# 1. What is Vital Injector2?

Enter the Vitality – A grandeur initiative we all share in search of precise and accurate beauty care for the people.



Vital Injector2 is a multi-use injector with a capacity of
loading interchangeable 1cc ~ 5cc syringes that is
versatile to various medical use.
The unique mechanism inside the device allows it to
repeatedly deliver low to high viscous formulations in
small yet highly accurate dosages.

# 2. Device Feature – Main Console

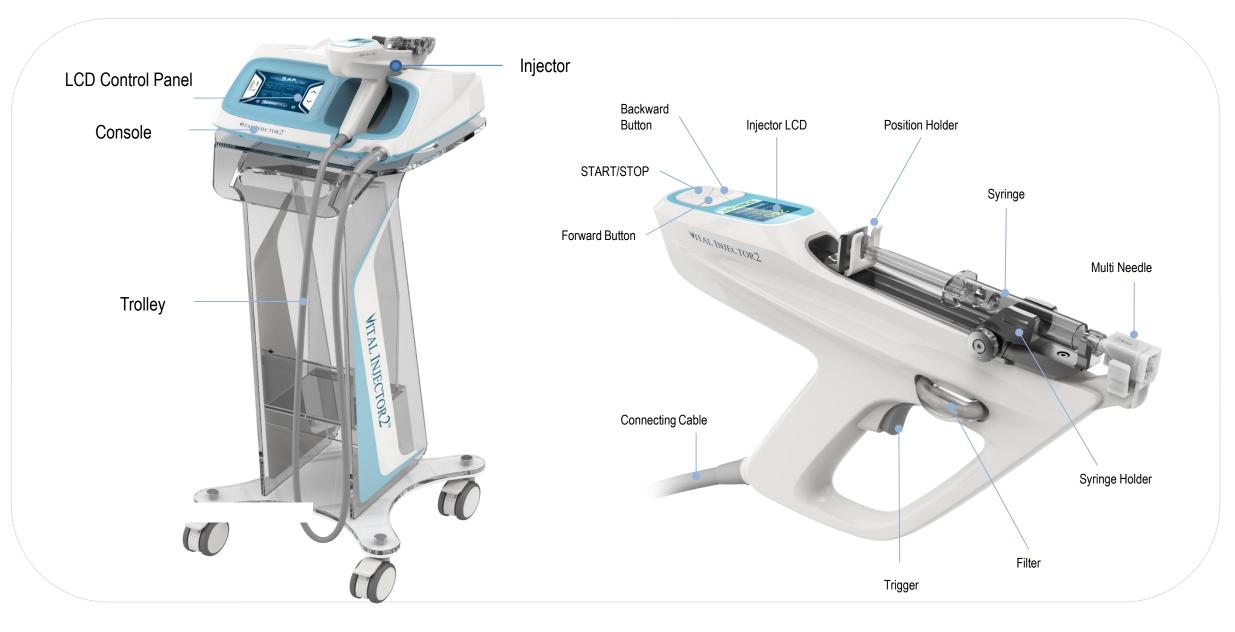

# 3. Usage


Vital Injector2 is developed to assist in the delivery of stabilized hyaluronic acid filler to restore hydrobalance in the skin and improve skin elasticity and tone to create a revitalized look.

Healthcare professional recommended skin care science that leverages proven formulation at full capacity.

### Improves

- Hydration
- Elasticity
- Roughness

- Smoothness
- Complexion
- Texture

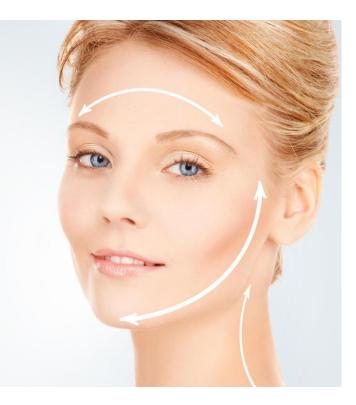

# 4. Technology Inside - Filter

Advanced capabilities.

Now, Featuring with Detachable Hygienic- Filter Technology

Built with advanced hygienic-centric

technologies inside, this small but highly

capable item effectively protect the system

to use it safer and cleaner – help you make reliable, confident treatment




# 4. Technology Inside - Needle

Incorporating the latest technology - vacuum assisted needle.

Pressure release-needle enables to inject all

needles beneath skin and subcutaneous tissue

- minimize drug loss.
- Patented Backflow Block Structure, effectively prevents from the contamination out of drug and blood.

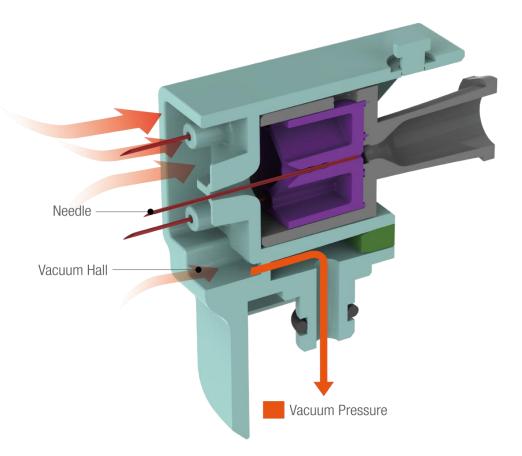


# 4. Technology Inside - Needle

Newly engineered needle delivers all the essential functionality to meet the clinical needs.

- Controlled depth adjustment, from 0mm to 5mm at 0.2mm interval, delivers customized treatment to accurately target sub dermis where most of collagen and elastin exists.
- 31G, 5 Pin multi needle provides dynamically faster

treatment.

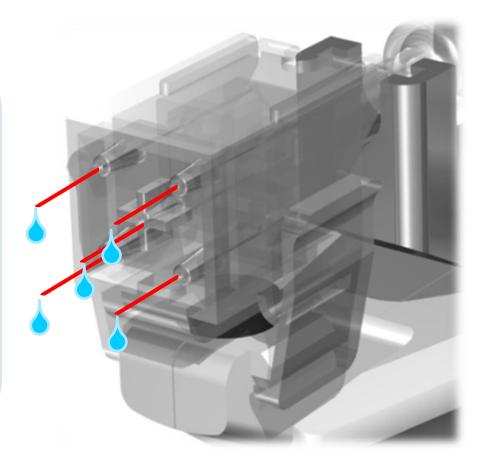

# 4. Technology Inside - Others

### Engineered for operational efficiency - Easy to use and maintain

- User-friendly interface simplifies and speeds operation.
- Various sizes of syringes are applicable (1 to 5cc) to select from.
- Variety of viscosity formulations is allowed.
- Convenient audible alarm signal activation initiated when drug within the syringe is almost used up



# 5. Feature & Advantage of Vital Injector2

- Ergonomic & Innovative design for increased working comfort and hand support.
- Accurate & Precise
   Injection
- Audible, Visual & Tactile
   Feedback
- Delivers small dosages
- Even distribution of

volume over large skin areas for predictable treatment results

- Easy treatment of versatile areas such as the face, neck,
  - decolletage and hands.
- Accommodates High Viscosity



# 6. Why you need Vital Injector2?

The new level of drug-delivery injection system which will be your cornerstone for the future. Help support your business confidence, too.

Buying Vital Injector2 represents a smart and strategic investment for the clinic so that you can gain a strong competitive differentiator.


- Engineered with unparalleled hygiene-centric system detachable single-use filter
- Praised for accurately adjusted depth control, powered by vacuum assisted technology.
- Reduced drug-loss during injection procedure.
- Acknowledged patented Backflow Block Structure system.

# 7. Specification

Device Type	Auto Injector		
Needle Information	31 G, 5Pin Multi Needle Type		
Needle Extension	0~5 mm(Interval 0.2mm)		
Syringe Dosage	1сс, 2сс, 3сс, 5сс	Voltage & Frequency	AC 110 ~ 240V, 50~60 Hz
Needle Attachment	Manually Attached	Dimension	<b>Injector</b> : 293 x 49 x 168
Needle Insertion	Manual	(WxDxH mm)	<b>Body</b> : 333 x 298 x 130
Needle Removal	Manual	Panel	Gun LCD : 1.7 Inch
Operation Made	Intermittent Continuous		Body LCD : 5.0 Inch, Touch Screen
Operation Mode	Intermittent, Continuous	Weight	4.7 Kg (Body + Injector)
Activation	Button(Trigger) Activated		

# Appendix Clinical Treatment

# Adjustable time to Speed

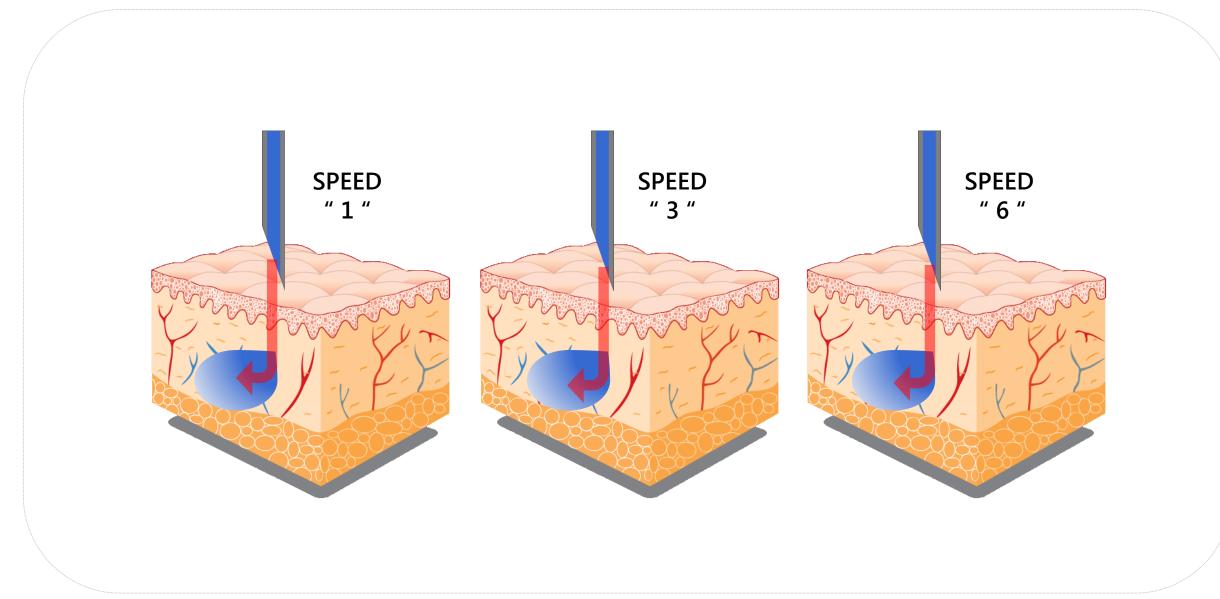

# Drug dosage per type of syringes

### Intermittent Mode [ Preset : 10 µl ]

Туре	Dose	Available Shot	Dose	Available Shot
Based on 1 cc Syringe	5 µl (1.0µl/pin)	200	12 µl (2.4µl/pin)	83
	6 µl (1.2µl/pin)	167	13 µl (2.6µl/pin)	77
	7 μl (1.4μl/pin)	143	14 µl (2.8µl/pin)	72
	8 µl (1.6µl/pin)	125	15 µl (3.0µl/pin)	67
	9 µl (1.8µl/pin)	111	20 µl (4.0µl/pin)	50
	10 µl (2.0µl/pin)	100	25 μl (5.0μl/pin)	40
	11 µl (2.2µl/pin)	90	30 μl (6.0μl/pin)	33

### Continuous Mode [ Preset : 1 cc/min ]

Туре	Dose
1 cc	1 cc/min, 2 cc/min, 3 cc/min
2 cc	
3 cc	1 cc/min, 3 cc/min, 5 cc/min
5 cc	

# Vital Injector2 Application on the face

