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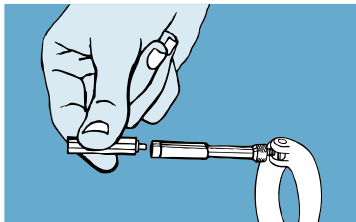
03

Add as many extension rods as necessary until **the device** matches your **personal size**. Once you have finished, replace the upper plastic support.

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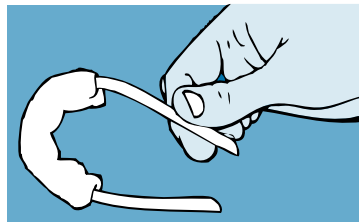
04

Thread the silicone band through the Androtop®. Next, insert each band end into the holes of the plastic support.

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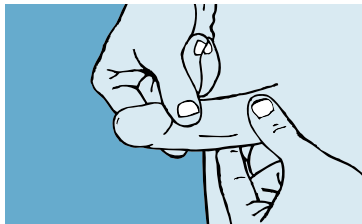
Your penis must be in flaccidity. Retract your foreskin to uncover your glans completely. We recommend covering the groove of the glans with a gauze, cotton wool and/or even the Androrring® (brown jelly ring).

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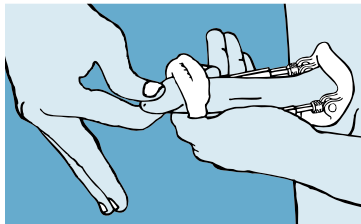
Next, insert your penis through the base ring (the thinnest part of the ring located upwards) and finally through the silicone band loop, resting your glans in the hollow part of the upper plastic support.

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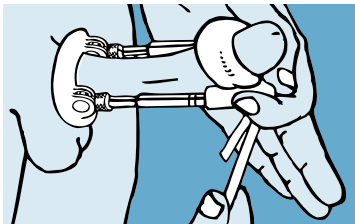
Keeping the band above your glans, shorten the loop by pulling on each end of the silicone band until the glans is completely held.

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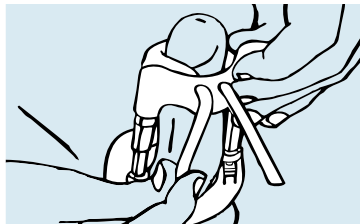
Finally, fit the ends of the silicone band into the grooves behind the upper plastic support carefully. You have to stretch the band while pressing into the grooves.

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COMPONENTS (fig. 1): 1-plastic base ring, 2-rounded hinge nut, 3-threaded bar, 4-threaded adjustable shaft, 5-metal shaft, 6-screw, 7-spring, 8-screw to secure the spring, 9-large 4 cm rod (1.6"), 10-medium 2 cm rod (0.8"), 11-small 0.5 cm rod (0.2"), 12-smallest rod 0.3 cm (0.1"), 13-upper plastic support, 14-silicone band, 15-protective cushion, 16-silicone rings.

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www.andropenis.com

Increase penis size with traction therapy and treat micropenis.

www.andropeyronie.com

Reduce penile curvature and treat Peyronie's disease.

www.androvacuum.com

Improve erections with vacuum therapy and treat erectile dysfunction.

andromedical.com/shop

Spare parts and accessories

www.andropharma.com

PENIS

Increase the own testosterone levels and improve penile lengthening.

VIGOR

Increase men's sexual desire and achieve long lasting erections.

CURVATURE

Inhibit the fibrous plaque of Peyronie with natural anti-inflammatories.

Further information, updates, privacy policy, return-refund policy, disclaimer and conditions of warranty can be found at: www.andromedical.com

© Copyright 1995-2025 Andromedical®, Madrid (Spain).