Instructions for Use (IFU)

1

IFU LANGUAGES

www.androvacuum.com/ifu/

VIDEOS

www.andromedical.com/videos

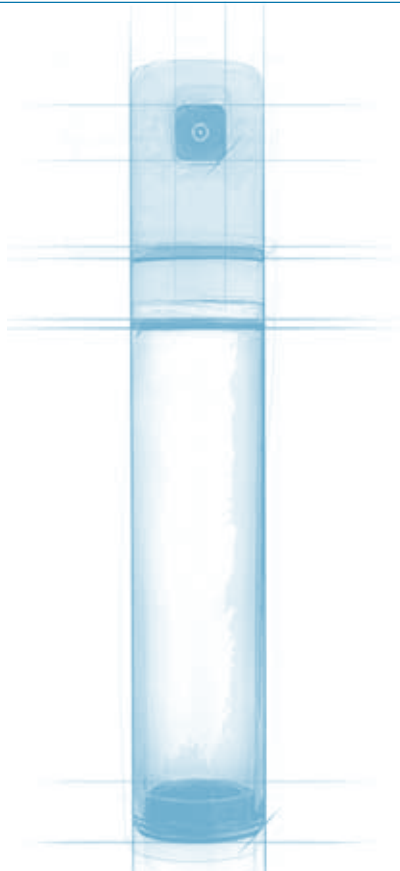
FAQs – QUESTIONS

www.androvacuum.com/faqs


MEDICAL CONSULTATION (online, chat & email)

www.androclinic.com

01

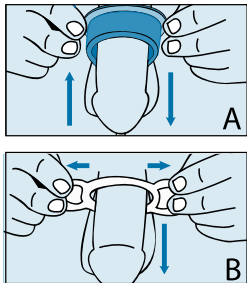Please, choose the components according to the therapy and your penis size. For erectile dysfunction treatment, choose the cushioning ring (fig. A) and the constriction ring (fig. B) which fits your penis width better.

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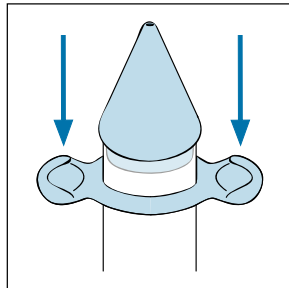
02

Fit the cone at one end of the cylinder. Apply lubricant gel to the cone of the cylinder and slide on the constriction ring until the ring snaps on to the cylinder.

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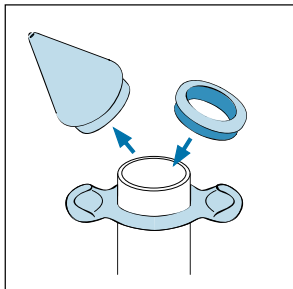
03

Remove the cone of the cylinder. Snap rings in place and place in the selected cushioning rings after applying Gel Lubricant.

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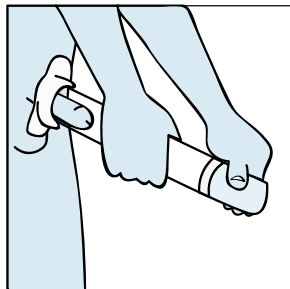
04

On the other end of the cylinder, snap in the electric motor, when fitted correctly there will be no movement and there will be a perfect seal.

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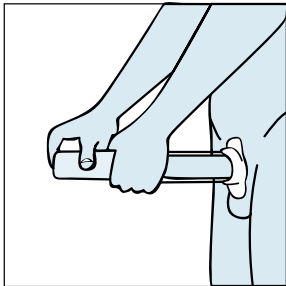
05

Insert the penis into the cylinder where the ring buffers are in place, insert the penis all the way until the pubis. I may be easier to create a vacuum if the hair is trimmed or shaved on the pubis where it is in contact with the cylinder.

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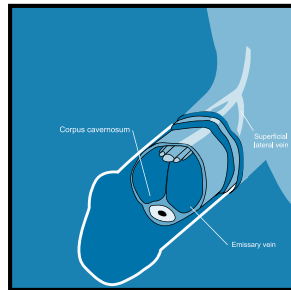
06

With one hand, hold the cylinder tight against the pubis, with the other hand press the motor button and start the motor. For manual model, grasp the motor with your hand and make up-and-down movements.

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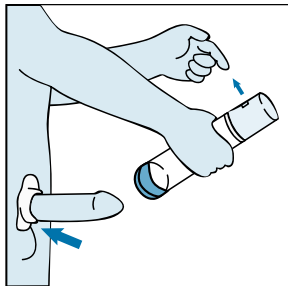
07

After achieving a full erection, slide the correct size of constriction ring down the outside of the cylinder and set at the base of the penis. Stop the vacuum by pressing the red button on the electric motor to release air back into the cylinder.

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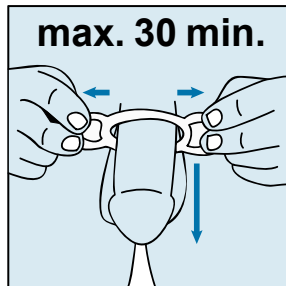
08

The constriction ring will keep blood in the penis and therefore hold the erection. It can be worn for up to 30 minutes.

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PORTFOLIO

www.andropenis.com

Increase penis size with traction therapy and treat micropenis.

www.andropeyronie.com

Reduce penile curvature and treat Peyronie's disease.

www.androvacuum.com

Improve erections with vacuum therapy and treat erectile dysfunction.

andromedical.com/shop

Spare parts and accessories

www.andropharma.com

PENIS

Increase the own testosterone levels and improve penile lengthening.

VIGOR

Increase men's sexual desire and achieve long lasting erections.

CURVATURE

Inhibit the fibrous plaque of Peyronie with natural anti-inflammatories.

Further information, updates, privacy policy, return-refund policy, disclaimer and conditions of warranty can be found at: www.andromedical.com

© Copyright 1995-2025 Andromedical®, Madrid (Spain).