Instructions for Use (IFU)

1

IFU 30 LANGUAGES

www.andropenis.com/ifu/

VIDEOS

www.andromedical.com/videos

FAQs - QUESTIONS

www.andropenis.com/faqs

FAQs - QUESTIONS

www.andropeyronie.com/faqs

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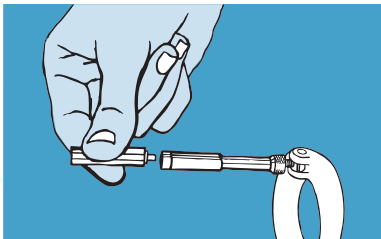
Add as many extension rods as necessary until **Andropenis®** matches your **personal size**. Once you have finished, replace the upper plastic support.

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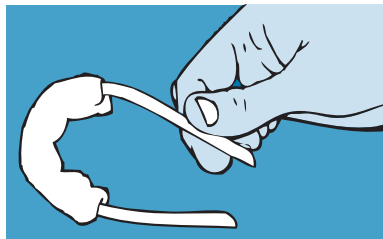
Thread the silicone band through the Androtop®. Next, insert each band end into the holes of the plastic support. For Andropeyronie®, Androsurgery® and Andropenis Mini®, please read the ADDITIONAL GUIDE FOR THE UPPER PLASTIC SUPPORT.

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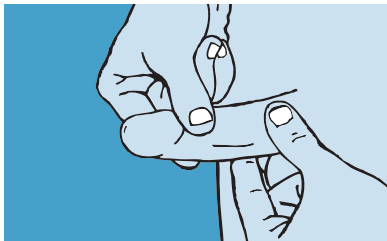
Your penis must be in flaccidity. Retract your foreskin to uncover your glans completely. We recommend covering the groove of the glans with a gauze, cotton wool and/or even the Androring® (brown jelly ring).

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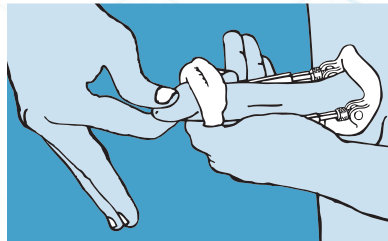
Next, insert your penis through the base ring (the thinnest part of the ring located upwards) and finally through the silicone band loop, resting your glans in the hollow part of the upper plastic support.

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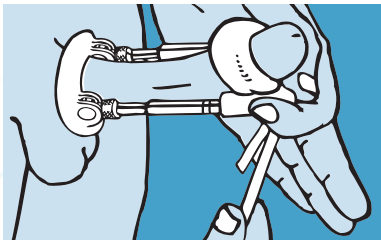
Keeping the band above your glans, shorten the loop by pulling on each end of the silicone band until the glans is completely held.

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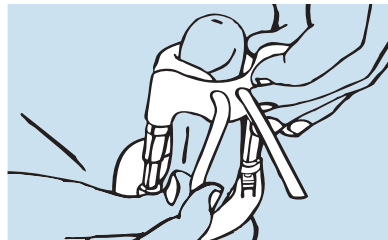
Finally, fit the ends of the silicone band into the grooves behind the upper plastic support carefully. You have to stretch the band while pressing into the grooves.

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