

BULB SELECTION GUIDE FOR PRECISION VISION ETDRS® ILLUMINATOR CABINETS

IMPORTANT INFORMATION

- **ONLY BULBS MANUFACTURED BY PRECISION VISION MAY BE USED IN PRECISION VISION ETDRS® ILLUMINATOR CABINETS**
- FLUORESCENT BULBS CAN ONLY BE USED IN V1,V2 AND V3 CABINETS
- LED BULBS CAN ONLY BE USED IN V4 CABINETS

BULB RECOMMENDATIONS FOR CLINICAL TRIALS

To maintain compliance in an active clinical trial, follow these standards:

- **Replacements:** All fluorescent ETDRS® Illuminator Cabinet bulbs must be replaced annually and LED ETDRS® Illuminator Cabinet bulbs are recommended for replacement every 2 years.
- **Spare Bulbs:** Sites should maintain a complete backup set of bulbs at all times.
- **Pair Replacement:** Bulbs must be replaced in sets of two (2).
- **Pre-Burn Requirement:** All bulbs must be pre-burned for 96 hours before use.
- **Documentation:** Pre-burned bulbs are supplied with a certificate of completion. Retain a copy of this certificate for site staff or sponsor documentation.

Precision Vision®

ETDRS® ILLUMINATOR CABINETS

VERSION 1 ETDRS® ILLUMINATOR CABINETS **SHOULD NOT** BE USED, ENROLLED, OR GRANDFATHERED IN ANY NEW CLINICAL TRIALS OR SCIENTIFIC STUDIES.

VERSION ONE (1) - FLUORESCENT 100-120 VOLT

REQUIRE TWO (2) UNITS OF
PART ID 2427D-BI-120V1

Precision Vision®
944 First Street • La Salle, IL 61301 • U.S.A.
Phone (815) 223-2022 • FAX (815) 223-2224

120V. 60Hz. A.C. Only .75 A.
CHART ILLUMINATOR CAT. NO. 2425

Precision Vision®
944 First Street • La Salle, IL • U.S.A. • precision-vision.com

120v. AC
50/60 Hz
0.54A

ETDRS Illuminator Cabinet™
Cat. No. 2425
Model: 120-E-P

LOT

Precision Vision®
944 First Street • La Salle, IL 61301 • U.S.A.
Phone (815) 223-2022 • FAX (815) 223-2224

100V. 50/60Hz. A.C. Only .800 A.
CHART ILLUMINATOR CAT. NO. 2425

Precision Vision®
944 First Street • La Salle, IL • U.S.A. • precision-vision.com

100v. AC
50/60 Hz
0.75A.

ETDRS Illuminator Cabinet™
Cat. No. 2425
Model: 100-M-P

LOT

LIGHTHOUSE ENTERPRISES
111 East 59th St.
New York, NY 10022
1-800-829-0500

120 V. 60Hz. A.C. Only .75 A.
CHART ILLUMINATOR CAT. NO. L9130

VERSION ONE (1) - FLUORESCENT 230 VOLT

REQUIRE TWO (2) UNITS OF
PART ID 2427D-BI-230V1

Precision Vision®
Precision Vision, Inc.
944 First Street
La Salle, IL 61301 USA
1.815.223.2022

EC REP QNET BV
Hommertweg 286
6436 AM Amstenrade The Netherlands

CE

230V ~
.3A
50 Hz
CLASS I

ETDRS Illuminator Cabinet™
Cat. No. 2425E
Model: 230-M-P

LOT

Precision Vision®
944 First Street • La Salle, IL 61301 • U.S.A.
Phone (815) 223-2022 • FAX (815) 223-2224

230V. 50Hz. A.C. Only .300 A.
CHART ILLUMINATOR CAT. NO. 2425

AVS
BAJA VISIÓN
Ponzano, 25-28003 Madrid (Spain)
Teléfono, 91 442 39 29
Fax: 91 399 43 57

230 V. 50 Hz. A.C. Only .300 A.
CHART ILLUMINATOR CAT. NO. 2425

Produced Exclusively by: **Precision Vision®**


Precision Vision®

ETDRS® ILLUMINATOR CABINETS

VERSION TWO (2) - FLUORESCENT

100-120 VOLT


REQUIRE TWO (2) UNITS OF
PART ID 2427D-BI-120V2

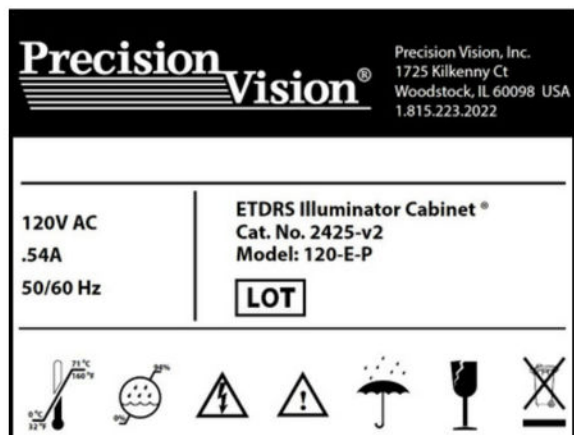

VERSION THREE (3) - FLUORESCENT

100-230 VOLT


REQUIRE TWO (2) UNITS OF
PART ID 2427D-BI-V3

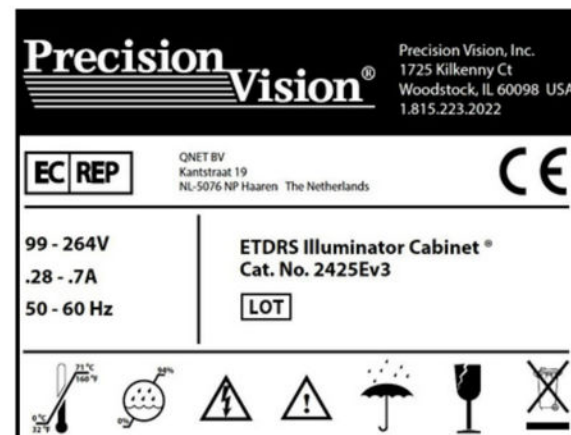

VERSION TWO (2) - FLUORESCENT

230 VOLT

REQUIRE TWO (2) UNITS OF
PART ID 2427D-BI-230V2

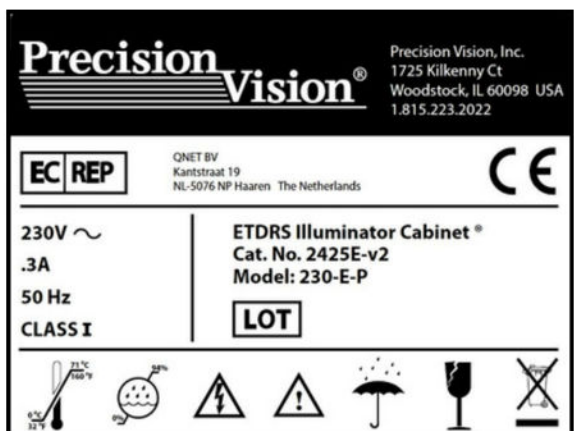

VERSION FOUR (4) - LED

90-265 VOLT

REQUIRE TWO (2) UNITS OF
PART ID 2427D-v4

