

FFP2

Filtration face mask

Size XS



Technical information

Product name: Filtering half mask
Classification: FFP2 - NR (non-reusable)
Model: STC-F2-01
BFE: > 98 %
PFE: > 94 % (>98% NaCl aerosol sprays)
Size: 138 mm x 97 mm. Size XS
Standard: EN 149:2001 + A1:2009
Composition: 66.7% non-woven fabric, 33.3% melt-blown non-woven fabric
Allergy warning: Non-woven textile
Expiration time: Two years if stored under the specified conditions

Fitting instructions

1. Attach the size regulators to the mask straps as shown in the picture.
2. Hold the mask with one hand and place it on the face, fully covering the nose and mouth. The nose clip should be over the nose.
3. Place the lower strap behind your head. Next, pass the top strap over your head until it is at the crown of your head.
4. Adjust the size of the straps with the size regulators for a better fit. Fasten the nose clip to the nose with both hands and make sure the edges of the mask are in direct contact with the face.
5. Fit testing for leaks. Allows you to check that the mask is fitted correctly. To do this, cover the mask with both hands and exhale with force. If any leaks are detected at the edges, you will need to adjust the position of the mask against the face, readjust the nose clip and/or the tensile position of the elastic straps using the size regulators. Repeat this process until no leaks are detected.



Specifications

Neck straps for maximum comfort
 Designed for **face perfect fit**
High breathability
Doble welding that guarantees the resistance of the cords
 up to 25 newtons
 With cord stopper for **size regulation**
Ear saver included
Individual packaging including:

- Reusable mask bag with ziplock
- Time tracker to accurately control of the maximum hours of use
- Name tag for the owner
- Ventilation system

CE 0370

EN 149:2001 + A1:2009

FFP2

Filtration face mask

Size XS



Important information

- NR equipment must not be used for more than one shift.
- Use in accordance with the manufacturer's recommendations, following the instructions on the label.
- These masks do not provide oxygen.
- Not to be used in environments with low oxygen concentration or poor ventilation.
- The presence of special facial features, abundant facial hair or glasses could hinder correct fitting of the mask.
- Keep away from heat sources.
- Indoor storage with less than 80% humidity, free from corrosive gases and well ventilated.

Individual bag specifications

Quantity: 1 unit/bag
 Dimensions: 130 x 205 mm
 Weight: 12 g

SKU: MASK-ST-00031.0001



10 pcs bag specifications


Quantity: 10 units (individual bags)/bag
 Dimensions: 210 x 132 x 40 mm
 Weight: 130 g

SKU: MASK-ST-00031.0010



Master carton specifications

Quantity: 60 box/master carton
 Total quantity: 600 units
 Dimensions: 485 x 415 x 444 mm
 Weight: 10.2 kg

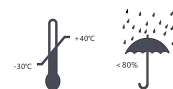

CE 0370

EN 149:2001 + A1:2009