

# COLPOFIX®

## 10 | VAGINAL APPLICATORS

### INDICATION

Prevention and treatment of cervical lesions caused by Human Papillomavirus, thanks to the control of physiological conditions of the transformation zone of the cervico-vaginal mucosa.

### DESCRIPTION

**Colpofix®** is an innovative preparation in terms of formulation and method of use. It is a gel with thixotropic characteristics that due to the force of mechanical compression, turns into a liquid when applied, and returns to gel form after application. The exclusive and ergonomic shape of the applicator allows a uniform and effective distribution of the mucoadhesive preparation directly, and in depth, on the cervico-vaginal mucosa, avoiding the spills.

**Colpofix®** contains as functional substances: Polycarboxophil and Carboxymethyl beta-glucan. Particularly, Colpofix is effective in the proposed indications by several mechanisms of action:

- Form a film protecting the cervico-vaginal mucosa from the deleterious action of external microbial agents, thanks to its capacity of bioadhesion.
- Promote re-epithelialisation and regression of cervical lesions, thanks to the control of physiological conditions on the cervico-vaginal transformation zone by hydration.
- Restore and maintain the vaginal microbiota thanks to the pH regulation and by a prebiotic effect.

### COMPOSITION

Water, Sorbitol, Carboxymethyl beta-glucan, Crosslinked acrylic polymer (Polycarboxophil), Imidazolidinyl urea, Sodium Dehydroacetate.

### WARNINGS

The colour of the gel may undergo variations due to the presence of the natural ingredients, this change is not altering the quality of the preparation. **The product is for vaginal use. Do not swallow.** Do not use the product without your physician or pharmacist recommendation, in case of confirmed or presumed pregnancy or concomitant local pharmacological therapies. The product is not intended for paediatric use. Keep the product out of sight and out of reach of children. Prolonged use of the product, as with all topical products, can give rise to phenomena of local sensitization or irritation; if this happens stop the treatment and consult your attending physician or pharmacist. Do not use the product if the package or the applicator is damaged, or not perfectly intact at time of first opening. Do not use the product after the expiry date. Use the product in accordance with the instructions for use. The cannula is disposable and should not be reused.

### STORAGE


Store in a dry place, away from light and heat sources. The expiry date indicated refers to the intact and properly stored product.

---

## INSTRUCTIONS FOR USE

### Do not shake the product before use.

Remove a vaginal applicator from its protective cover. Remove the cap from the spray cover. Insert the applicator into the dispensing nozzle of the spray. Begin the application by depressing the vacuum dispenser button until the product appears. Insert the applicator deep into the vagina while lying on your back with your knees bent. Perform 5 sprays per application. Discard the applicator after use. Close the spray with cap.

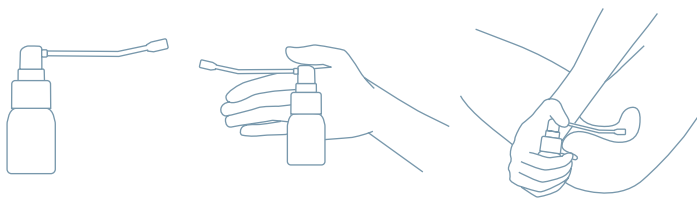

## POSOLOGY

It is advisable to carry out one application (5 deliveries) per day, for 10 days, in the evening before the night's rest.

To renew according to the recommendations of your doctor.

1 bottle corresponds to 10 days of use.

## SIDE EFFECTS

Do not use in case of hypersensitivity and incompatibility with any component of the product. The tests performed on the product to date have not shown any sensitisation phenomena or other side effects. In the event of persistent discomfort, consult a doctor.

## PRESENTATION

Box containing a 20 ml bottle and 10 individually packaged disposable applicators.

Each applicator is for single use. Fragrance-free.

Colpofix® is a Class IIa Medical Device in compliance with the requirements of Directive 93/42/EEC and subsequent updates.

This product should not be disposed of in waste water nor in household rubbish. Ask your pharmacist how to dispose of healthcare products that you no longer use. This will help to protect the environment.

Date of last revision: September 2020



Uriach Italy, S.r.l. A SOCIO UNICO  
Assago Milanofiori, Str. 1, Palazzo F6 - 20057 Assago (MI)



XXXXXXXXXX