


**TAMPER EVIDENT**

# Rainbow Trays™



Colour coded Anaesthetic trays


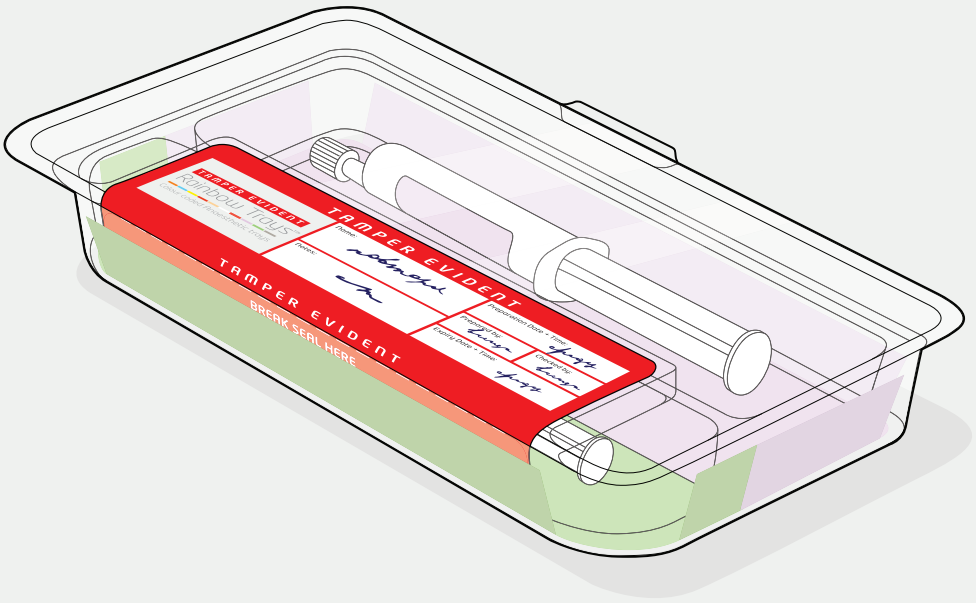


**INSTRUCTIONS FOR USE**

# LOADING THE TRAY

1

Place the tamper evident tray into the base tray

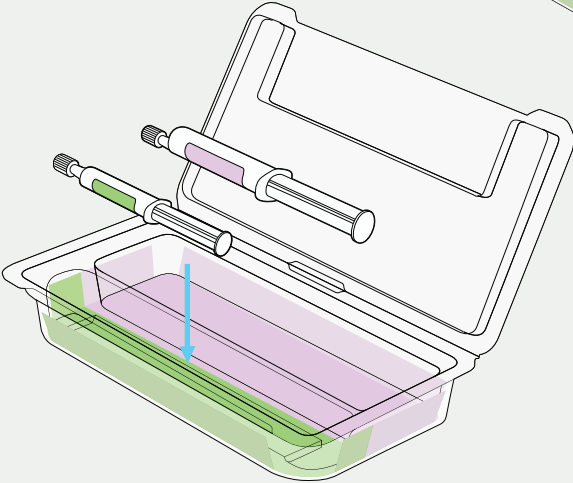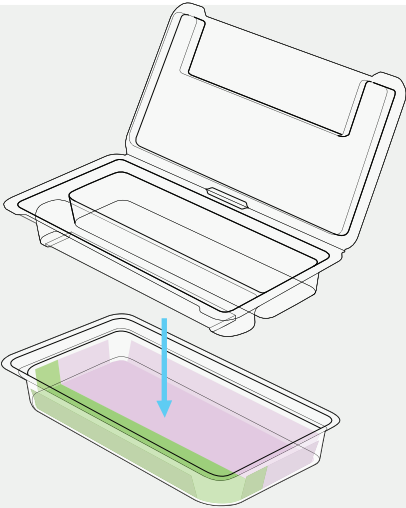


2

Place syringes into colour coded compartments

3


Click tray shut

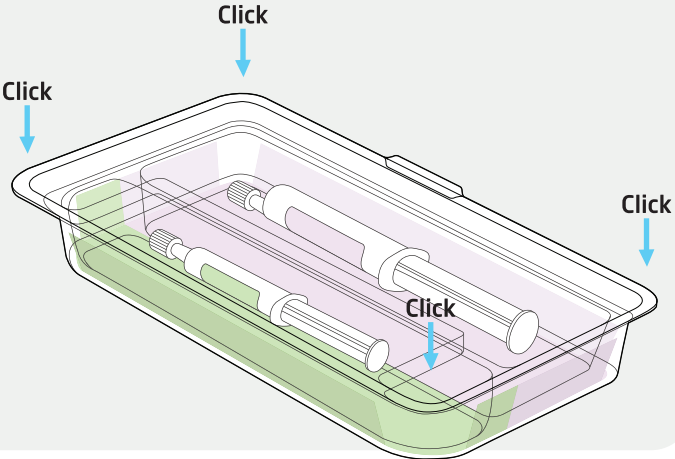

# SECURING THE TRAY

4

Fill in the label

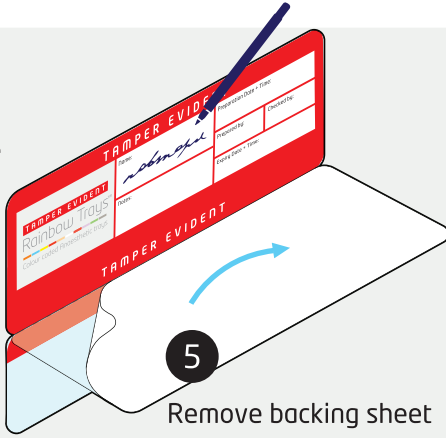


5

Remove backing sheet


6

Stick the adhesive flap over the text

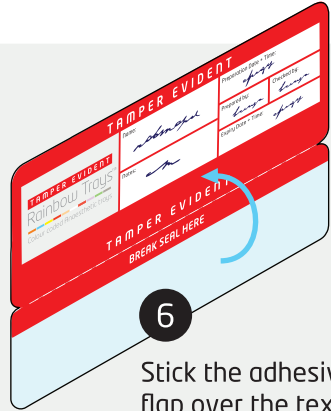


7

Remove the tray from the base and align the label with the top edge of the square area of the tray

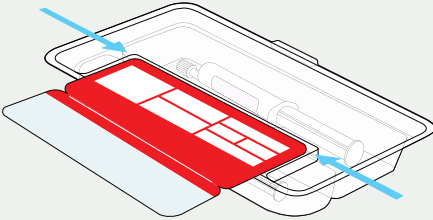


8

Wrap label around the tray and stick to underside

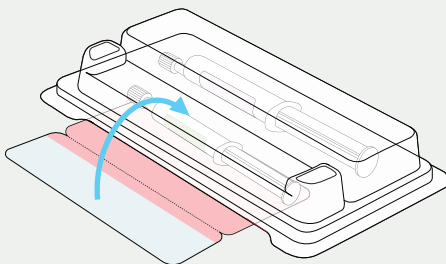

9



Place the tray back into the base. Your anaesthetic drugs are now secure in the tamper evident tray and ready for use

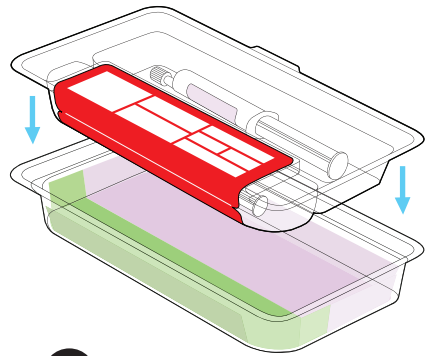

# USING THE TRAY

10

Remove the tamper evident tray from the base using the tab

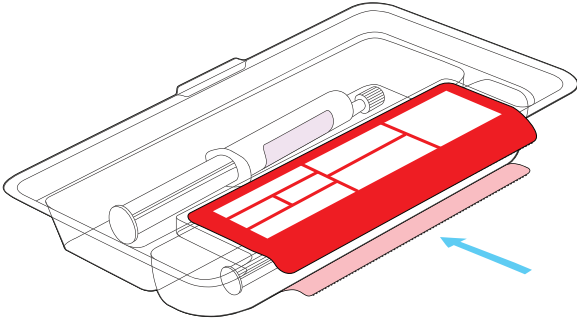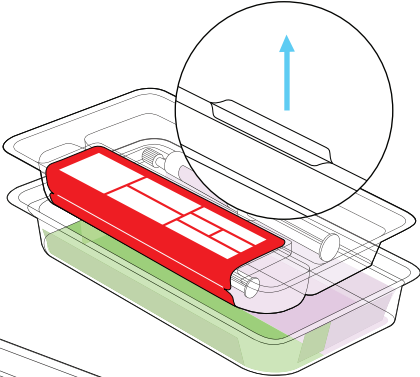

11

Break the seal in the middle of the perforations

12

Open lid – ready for use

