

BleedSolv™ Haemostatic Gauze

TGA approved haemostatic gauze for the topical control of bleeding from wounds; including trauma to face, head, neck and in body cavities.

First wherever bleeding occurs

BleedSolv™ forms a critical part of effective first aid and emergency management. Available in a range of products and sizes for any situation and safe for use on any part of the body. BleedSolv™ is particularly beneficial in remote locations distant from advanced care and in high-risk situations. BleedSolv™ gauze has bacteriostatic properties, conforms to the shape of any wound and is easy to apply wherever bleeding occurs.

BleedSolv™
Stop bleeding fast
Topical Haemostatic Dressings

A haemostatic material applied directly to the source of bleeding to control bleeding in trauma and wounds of all kinds



Technology saving lives

BleedSolv™ is a collagen-like natural substance created from cellulose. On contact with blood, BleedSolv™ expands 3-4 times absorbing exudates (fluids) from the blood and converting to a gel plug at the ends of damaged capillaries. BleedSolv™ quickly spreads clotting factors throughout the wound, reduces prothrombin release time, increases platelet aggregation reducing the loss of platelets and whole blood then degrades simply into polysaccharides and water. Following haemostasis, any excess unabsorbed BleedSolv™ may be rinsed gently from the wound using saline or water without disturbing the clot.




Meets Australian Resuscitation Council guidelines for the control of bleeding (ARC Guidelines, First Aid, 2016)

Effective Care

BleedSolv™ is effective in front-line trauma and first aid care, as well as for dentistry, obstetrical and gynaecological procedures. BleedSolv™ is compact, light and does not require special storage or training. Ideal to carry on your person or in remote or mobile situations.

Graph depicting haemostatic gauze success rates against other dressings for significant radial arterial haemorrhage on porcine sample (Adapted from Quick & Barnes, 2011)

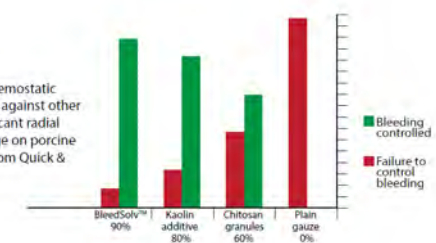

Ahead of time

For your nearest Stockist:

Phone: (08) 9409 9773

Mobile: 0415 359 968

Email: service@bleedsolv.com

Product Information Sheet Jan 2016