

BleedSolv™ Haemostatic Gauze

TGA approved haemostatic gauze for the topical control of bleeding from wounds; including trauma to face, head, neck and in body cavities.

First wherever bleeding occurs

BleedSolv™ forms a critical part of effective first aid and emergency management. Available in a range of products and sizes for any situation and safe for use on any part of the body. BleedSolv™ is particularly beneficial in remote locations distant from advanced care and in high-risk situations. BleedSolv™ gauze has bacteriostatic properties, conforms to the shape of any wound and is easy to apply wherever bleeding occurs.

BleedSolv™

Stop bleeding fast

Topical Haemostatic Dressings

A haemostatic material applied directly to the source of bleeding for the temporary control bleeding in trauma and wounds of all kinds



Technology saving lives


BleedSolv™ is derived from regenerated cellulose. On contact with blood, BleedSolv™ expands 3-4 times absorbing exudates (fluids) from the blood and converting to a gel plug at the ends of damaged capillaries. BleedSolv™ quickly spreads clotting factors throughout the wound, reduces prothrombin release time, increases platelet aggregation reducing the loss of platelets and whole blood. BleedSolv™ degrades into polysaccharides and water. Soften and loosen any excess, unabsorbed BleedSolv™ with saline or water for 1-2 minutes then rinse or wipe away without disturbing the clot.



Meets Australian Resuscitation Council guidelines for the control of bleeding (ARC Guidelines, First Aid, 2017)

Effective Care

BleedSolv™ is effective in front-line trauma and first aid care, as well as for dentistry, obstetrical and gynaecological and ear/nose/throat procedures. BleedSolv™ is compact, light and does not require special storage or protracted training. Ideal to carry on your person or into remote / mobile situations.

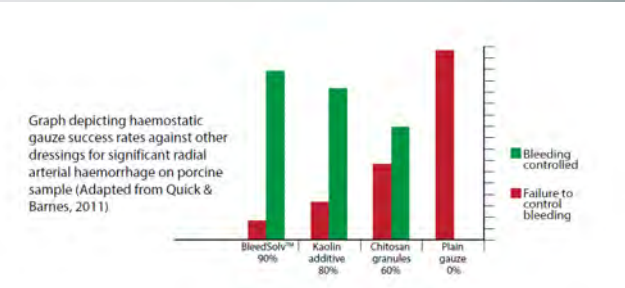

For your nearest Stockist:

Phone: (08) 9409 8773

Mobile: 0415 359 968 Email:

service@bleedsolv.com