

Clinical Results



Before

Age/Sun Spots



After



Before

Facial Veins



After



Before

Hair Removal



After



Before

Leg Veins



After

Photos courtesy of Kathleen P. Hutton, M.D.; Jonathan Crane, D.O.; E. Victor Ross, M.D.; Mark Taylor, M.D.; Mitch Chasin, M.D.

Science. Results. Trust.

Science. Results. Trust.

Syneron and Candela – two extraordinary brands, one global leader.

Syneron has a remarkable ability to innovate the newest technologies in aesthetic medicine.

Candela brings four decades of industry-leading scientific expertise.

Together, these two brands offer “gold standard” technologies for advanced aesthetic solutions.



GentleMax Pro[®]

Laser Hair Removal and Skin Treatment

Schedule a personal consultation with
your practitioner today!

www.syneron.com | www.candelalaser.com

A Laser for Everyone and Every Condition

Trust GentleMax Pro to remove unwanted hair, as well as sun and age spots and facial and leg veins efficiently and effectively on all skin types.

GentleMax Pro uses laser technology to deliver an intense but gentle burst of energy onto targeted areas of the skin, delivering a controlled amount of therapeutic heat to the targeted area safely and effectively.

What can I expect during laser treatment?

Treatment will take place in your practitioner's office. Your practitioner will use a small handpiece to deliver each laser pulse. A treatment could take as little as a few minutes to as long as an hour, depending on the treatment area and the condition being treated.

Does it hurt?

Some patients may feel a slight discomfort with each laser pulse, sometimes described like the snapping of a rubber band on the skin. This discomfort resolves quickly and is minimized by the patented Dynamic Cooling Device™ (DCD™) or air cooling. For some patients, a topical anesthetic may be used prior to treatment. Be sure to discuss this with your practitioner.

How many treatment sessions will I need?

The number of sessions depends on the treatment you require and the condition of your skin. For hair removal, multiple treatments are required. Other conditions may clear up in only one treatment, but many factors impact the efficacy of treatments, including the severity of the condition being treated.

How do I prepare for treatment?

Hair removal patients should avoid plucking, electrolysis or waxing for six weeks prior to treatment. If you have a history of herpes simplex virus, prophylactic antiviral therapy may be prescribed. Skin should be clean-shaven and thoroughly cleansed, removing any makeup, creams, oils, topical anesthetics or self-tanning products before laser treatment. Do not expose the intended treatment area to sunlight or tanning booth light prior to or immediately after treatment. Apply sunblock (SPF 30 or above) before treatment.

What can I expect after treatment?

A cool compress or gel pack may be applied for

comfort following your treatment. Any redness or swelling should resolve within a few hours.

What precautions should I take after treatment?

After your laser treatment, prevent sun exposure by applying sunblock (SPF 30 or above). In the first few days after treatment, don't scrub or rinse the treated areas with abrasive skin cleansers. Your practitioner may advise additional precautions.

Are there side effects?

Permanent side effects are rare, and may include redness, swelling, hyperpigmentation, hypopigmentation, blistering and purpura (a laser bruise) in the treated area—these usually do not last more than a few days. Be sure to discuss potential side effects with your practitioner.

Are these treatments covered by insurance?

Most insurance companies do not offer reimbursement for cosmetic procedures. Some carriers will, however, cover the treatment of disfiguring birthmarks such as hemangiomas. Consult your insurance carrier for conclusive information.

How much do treatments cost?

Cost depends on the area and condition being treated. Discuss your personal needs with your practitioner to learn more.